



Front Rigging Hangers and Hardware For 70NHD (Sector Style)

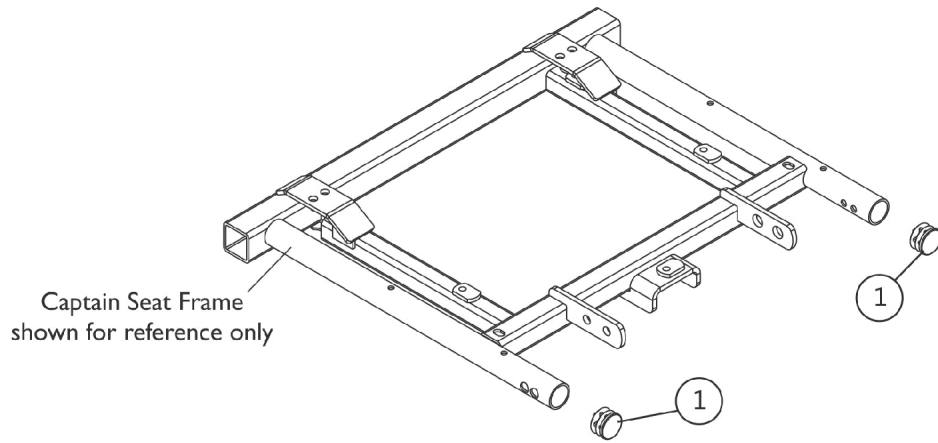

Swingaway Front Rigging Non-Hanger Hardware
(When Front Rigging Is Excluded From Chair)
Or
When Center Mount Riggings are used

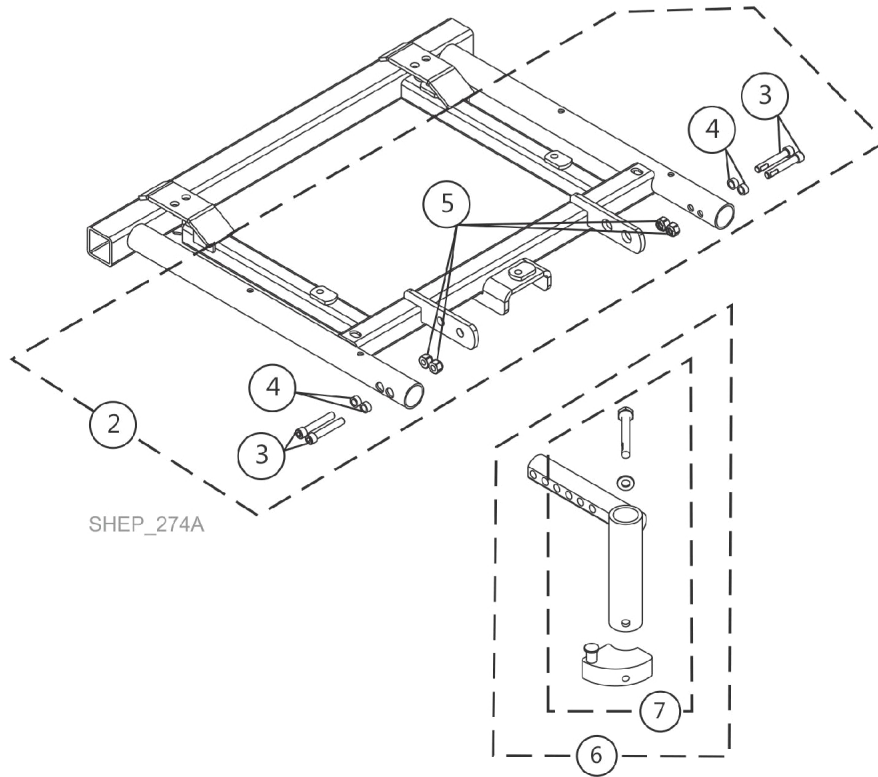

Front Rigging Hangers and Hardware For 70NHD (Sector Style)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1134425	Package, Plug Button, Black (1")	10	1
2	A	60104885	Kit, Front Rigging Hanger Hardware, Captain Seat	1	1
			Screw, Socket Head Cap w/ Patch (1/4-20 x 1-1/2")	1	4
			NLA - Spacer (1/4 x 3/8 x 3/16")	1	4
			Locknut (1/4-20)	1	4
6	B	1143416	Front Rigging Hanger Assembly w/ Sector Block, Black - Right	1	1
6	B	1143417	Front Rigging Hanger Assembly w/ Sector Block, Black - Left	1	1
7	C	1054571	Kit, Sector Block - Right	1	1
7	C	1054572	Kit, Sector Block - Left	1	1
NOTE: A - Includes items 3-5 B - Includes item 8 C - Includes Sector Block, Screw and Washer					