


Captain Seat - "ELOW" - Elevate, Main System Mounting and Interface Hardware

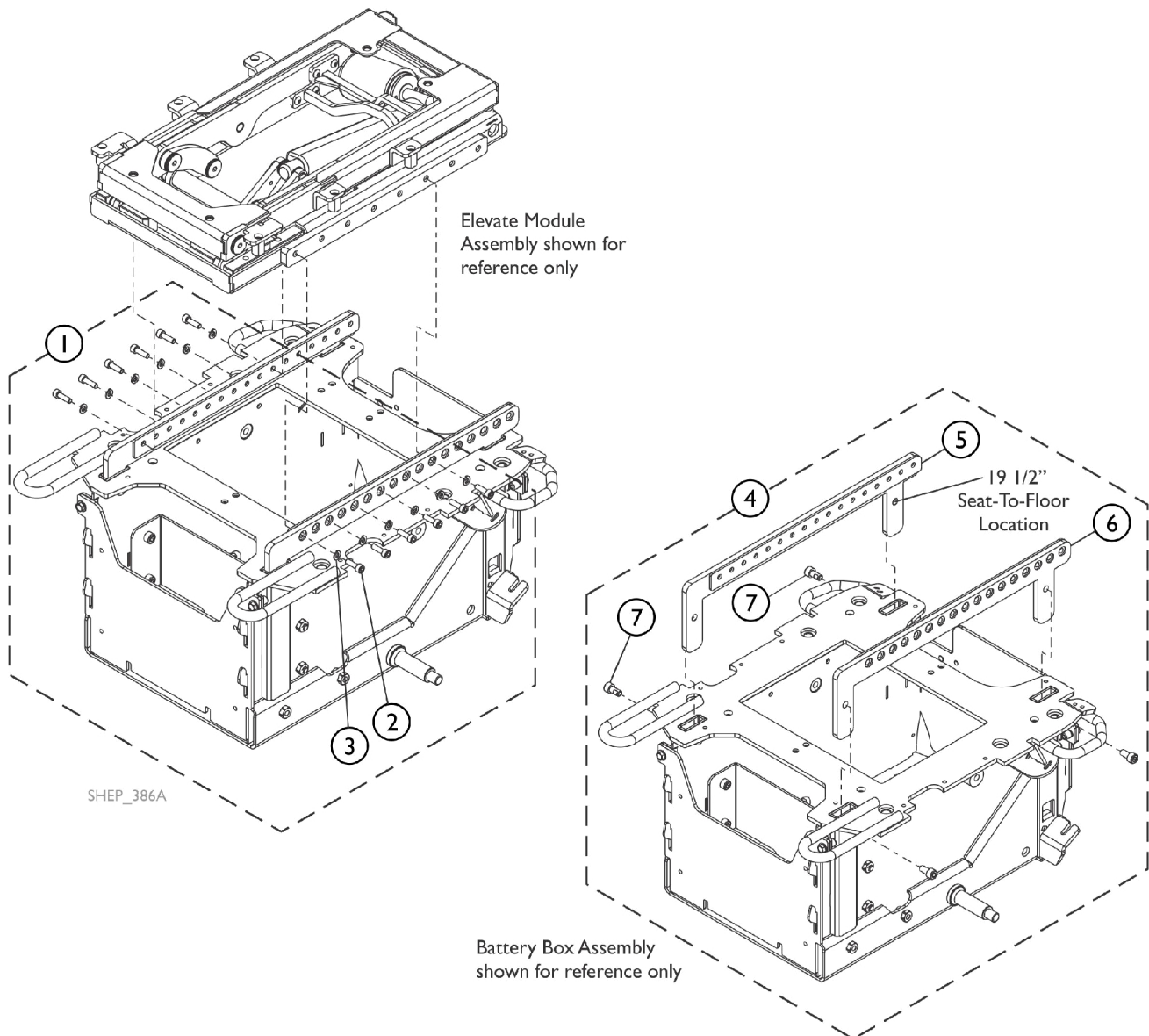

Captain Seat - "ELOW" - Elevate, Main System Mounting and Interface Hardware

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	60124362	Kit, Seat Mounting Hardware, Low	1	1
2			Screw, Socket Head Cap (M6 x 1 x 20mm)	1	12
3			Washer, Split Lock, Black Zinc (M6)	1	12
			Sheet, Insert Parts Service	1	1
4	B	60124359	Kit, Seat Mounting Bracket Hardware, Low	1	1
5			Bracket, Seat Mounting, Low - Right	1	1
6			Bracket, Seat Mounting, Low - Left	1	1
7			Screw, Socket Head w/ Patch (M8 x 1.25 x 25mm)	1	4
			Sheet, Insert Parts Service	1	1
NOTE: A - Includes items 2 and 3 B - Includes items 5-7					