


# Captain Seat Frame & Mounting Hardware, Non-Elevate, Non-TRRO or TRRO

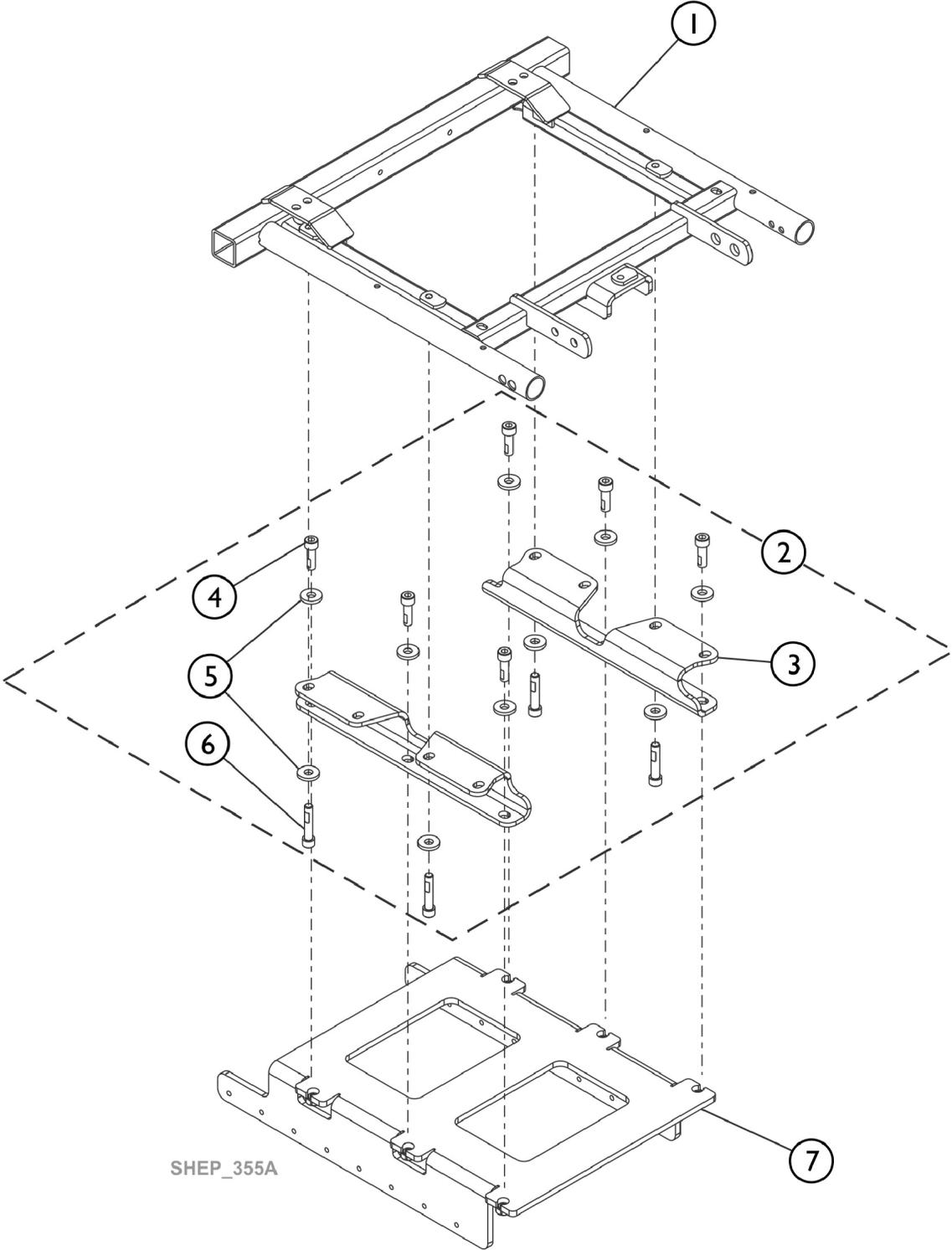

SHEP\_355A

## Captain Seat Frame & Mounting Hardware, Non-Elevate, Non-TRRO or TRRO

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		60104873	Kit, Frame, Captain Seat, Black (16" W)	1	1
1		60104874	Kit, Frame, Captain Seat, Black (18" W)	1	1
1		60104875	Kit, Frame, Captain Seat, Black (20" W)	1	1
1		60104876	Kit, Frame, Captain Seat, Black (22" W)	1	1
1		60108354	Kit, Frame, Captain Seat, Black, SP2 (24" W)	1	1
1		60104869	Kit, Frame, Captain Seat, Black, TRRO (16" W)	1	1
1		60104870	Kit, Frame, Captain Seat, Black, TRRO (18" W)	1	1
1		60104871	Kit, Frame, Captain Seat, Black, TRRO (20" W)	1	1
1		60104872	Kit, Frame, Captain Seat Black, TRRO (22" W)	1	1
1		60108353	Kit, Frame, Captain Seat, Black, TRRO, SP2 (24" W)	1	1
2	A	60108592	Kit, Captain Seat Frame Mounting Hardware (Elevate)	1	1
3			Bracket, Captain Seat Mounting, Elevate, Black	1	2
4			Screw, Socket Head w/ Patch (M8 x 1.25 x 25mm)	1	6
5			Washer, Flat (21/64 x 53/64 x 7/64")	1	10
6			NLA - Screw, Socket Head w/ Patch (5/16-18 x 1-1/2")	1	4
7		60124944	Kit, Non Power Seat Mounting Bracket	1	1

NOTE: A - Includes items 3-6.