


Crossbraces and Hardware - Heavy Duty

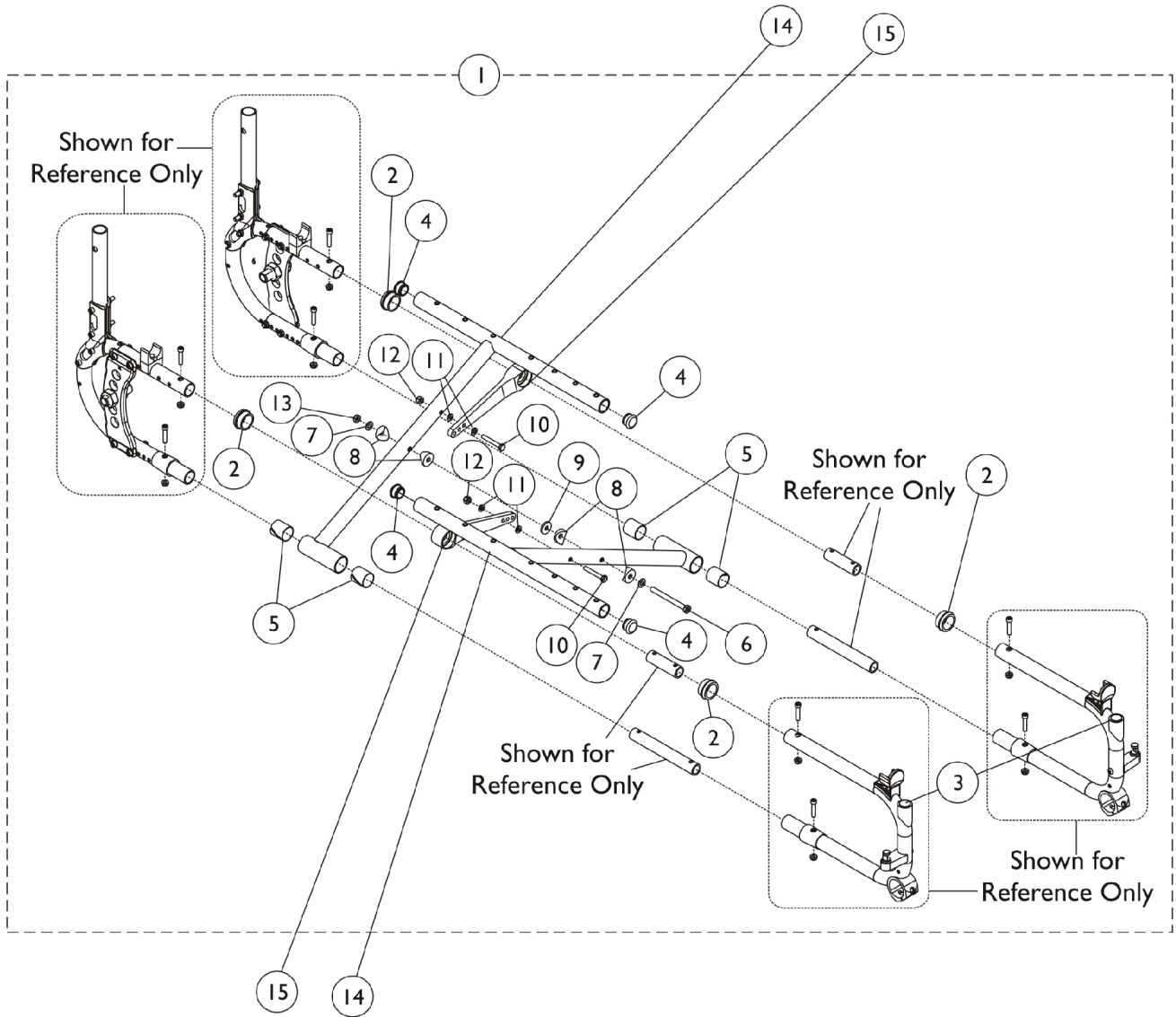

Crossbraces and Hardware - Heavy Duty

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1136670	Hardware, Crossbrace, Heavy-Duty	1	1
2		1134175	Bearing, Link Pivot	1	4
4		1056328	Plug Button, Black (1")	1	4
5		1018687	Sleeve, Bearing	1	4
6		1025705-NLA	NLA - Screw, Hex Head (5/16-24 x 3-1/4")	1	1
7		1025704	Washer, Flat (5/16 x 5/8 x 1/16")	1	2
8		1016263	Spacer, Coved (3/4 x 1 x 1/2")	1	1
9		1016301	Washer, Nylon (5/16 x 7/8 x 1/8")	1	1
10		1055348	Screw, Hex Head (1/4-28 x 1-3/4")	1	2
11		1026106-NLA	NLA - Washer (1/4 x 5/8 x 1/16")	1	4
12		1025765	Locknut (1/4-28)	1	2
13		1025730	Locknut (5/16-24)	1	1
14	A	See Chart	#14 Crossbraces - Heavy Duty	1	1
15		1142172	Link, Pivot, Black (18 - 20" W)	1	1
16		1142173	Link, Pivot, Black (20 - 22" W)	1	1

NOTE: A - Includes items 1-13.

#14 Crossbraces - Heavy Duty

Chair Depth	18" Wide	19" Wide	20" Wide	22" Wide
18" - Right	1136630	1136636	1136642	60125715
18" - Left	1136631	1136637	1136643	60125716
19" - Right	1136632	1136638	1136644	60125717
19" - Left	1136633	1136639	1136645	60125718
20" - Right	1136634	1136640	1136646	60125720
20" - Left	1136635	1136641	1136647	60125719
21" - Right	N/A	N/A	1136648	60125721
21" - Left	N/A	N/A	1136649	60125722
22" - Right	N/A	N/A	1136650	1136652
22" - Left	N/A	N/A	1136651	1136653