


Wheel Locks

Push/Pull to Lock

Wheel Locks can be converted from Push to Pull or Pull to Push. See Prospin X4 Owner's Manual (1125104) for instructions.

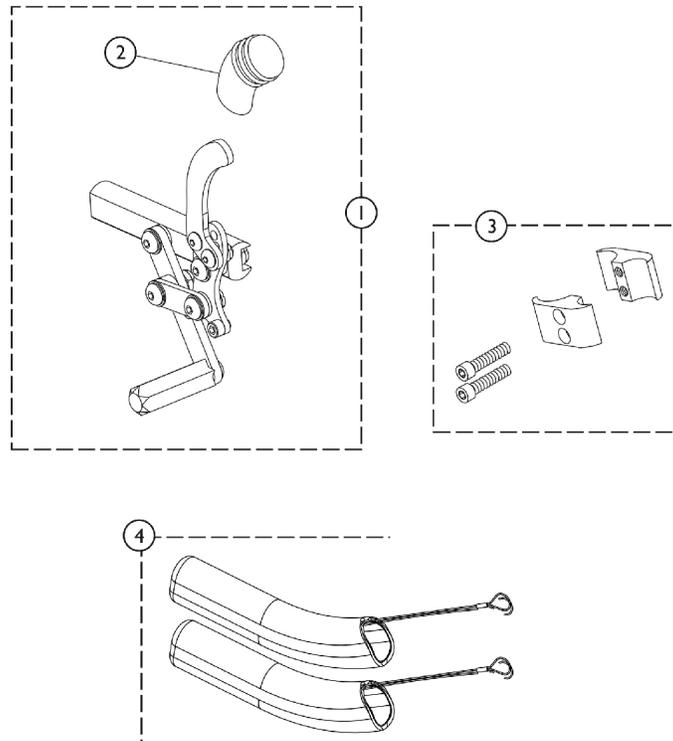

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A,C	1125459	Wheel Lock Assembly, Push/Pull - Right	1	1
1	A,C	1125460	Wheel Lock Assembly, Push/Pull - Left	1	1
1	B,C	1134129	Wheel Lock Assembly, Push/Pull for Amputee - Right	1	1
1	B,C	1134130	Wheel Lock Assembly, Push/Pull for Amputee - Left	1	1
2		1130584	Package, Cap, Brake Lever End, Black	10	1
3	D	1068210-NLA	Kit, Wheel Lock Attaching Hardware (1" Tubing)	1	1
			Screw, Allen Head (1/4-20 x 1")	1	2
			Clamp, Half, Non-Threaded	1	1
			Clamp, Half, Threaded	1	1
4		1146498	Handle, Wheel Lock Extension, Black (6") (Pair)	1	2

NOTE: A - Chairs without Amputee Attachment. Push/Pull Wheel Locks serve as replacements for High Mount Push to Lock that is No Longer Available. Order in pairs to match.
 B - Chairs with Amputee Attachment
 C - Wheel Locks can be converted from Push to Pull or Pull to Push. See Owner's Manual for instructions.
 D - Includes 2 ea. half clamps and 2 ea. screws