
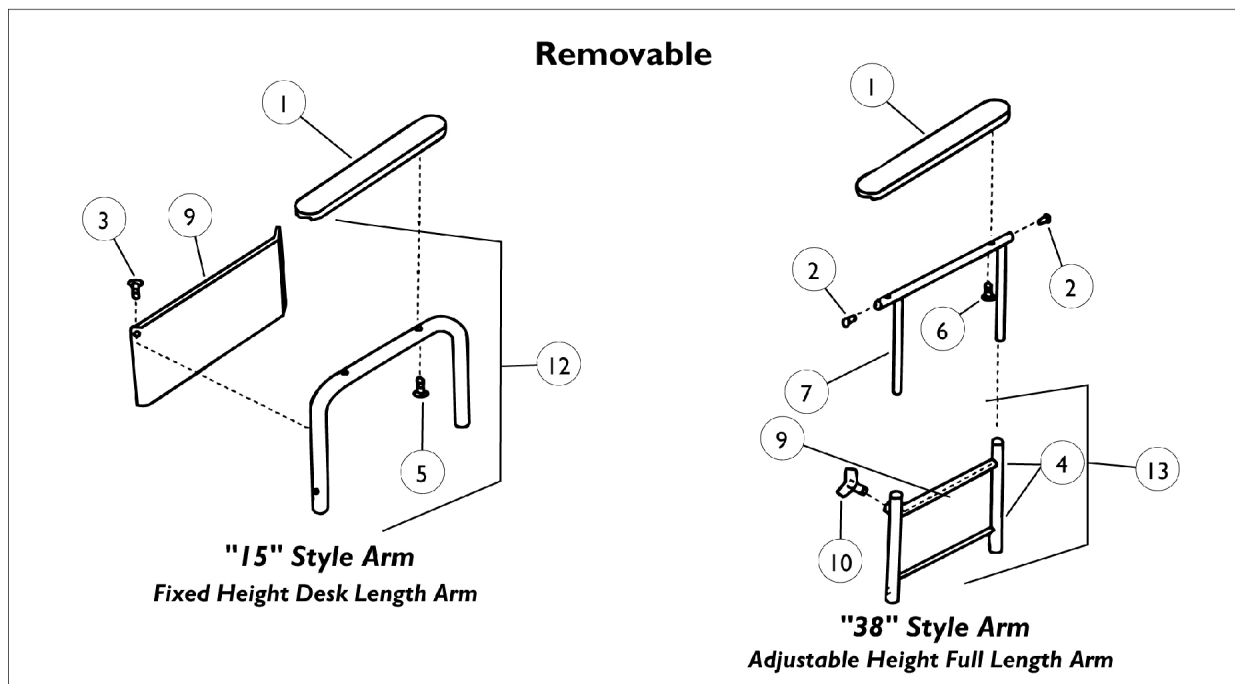


Permanent and Removable Arms

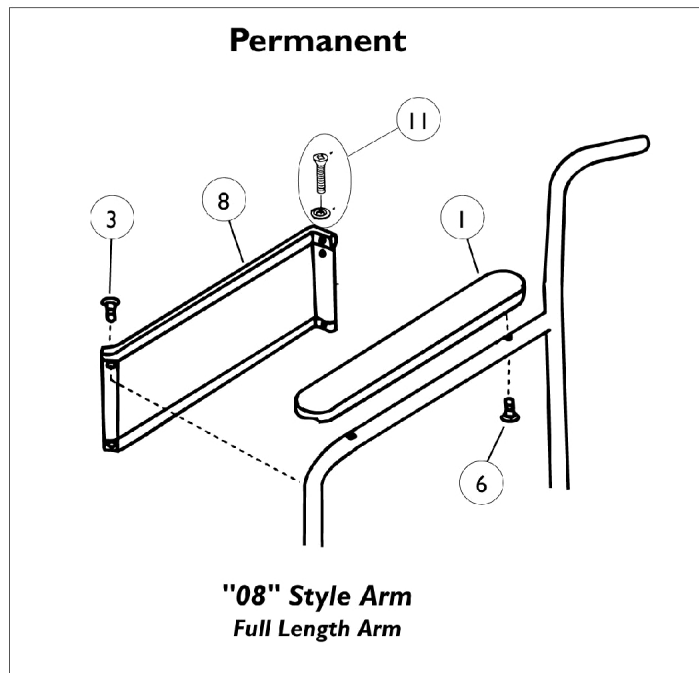

Permanent and Removable Arms

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	8881053570	Kit, Arm Pad w/ Screws, Desk Length (Pair)	1	1
1	A	8881053571	Kit, Arm Pad w/ Screws, Full Length (Pair)	1	1
2		1027095	Plug Button, Black (7/8")	1	2
3		1025729	Screw, Phillips Pan Head Tap (#8-32 x 3/8")	1	1
4		21243X000-NLA	NLA - Screw, Phillips Pan Head (#6-20 x 3/8")	1	4
5		11008X002-NLA	NLA - Screw, Phillips Oval Head (#10-32 x 1-1/4")	2	2
6		1034083-NLA	NLA - Package, Screw Phillips Round Head (#10-32 x 1-1/8")	2	2
7		1036198	Arm Tube Assembly w/ Plug Buttons, Black - Right/Left	1	1
8		47234X001-NLA	NLA - Guard, Clothing, Full Length, Fixed Arm, Chrome	1	1
8		47234X009-NLA	NLA - Guard, Clothing, Permanent, Full Length	1	1
9		47234X053-NLA	NLA - Guard, Clothing, Removable, Desk Length	1	1
9		47234X015-NLA	NLA - Guard, Clothing Removable, Full Length	1	1
10		40358X007-NLA	NLA - Knob, Locking	1	1
11		1076283	Screw, Phillips Oval Head w/ Washer (#10-16 x 7/8")	1	1
11		1041541	Package, Screw Phillips Csunk Oval Head w/ Washer (#10-16 x 7/8") (12 ea.)	12	1
12		1036193	Arm Base Assembly, Desk Length w/o Arm Pad - Right	1	1
12		1036194	Arm Base Assembly, Desk Length w/o Arm Pad - Left	1	1
13		04702X036-NLA	NLA - Arm Base Assembly, Full Length w/o Armpad - Right	1	1
13		04702X037-NLA	NLA - Arm Base Assembly, Full Length w/o Armpad - Left	1	1

NOTE: A - Specify upholstery color, see page titled "Packaging and Upholstery Colors"