
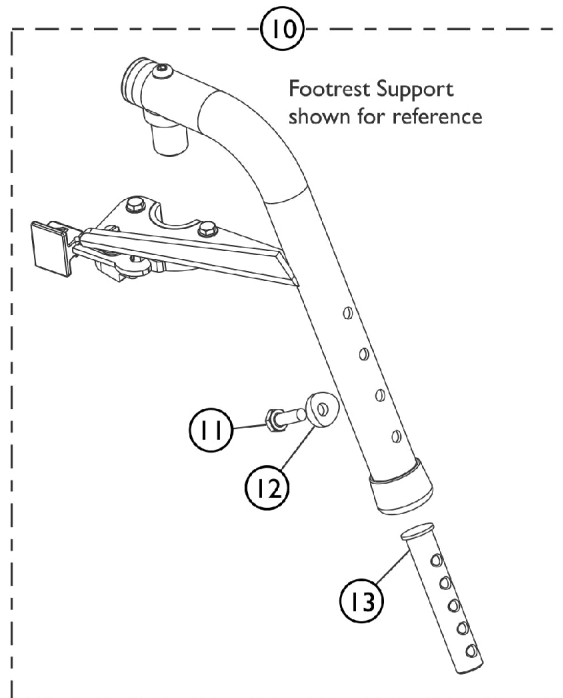


# Footrest Support Assembly 70NHD (Sector Style)

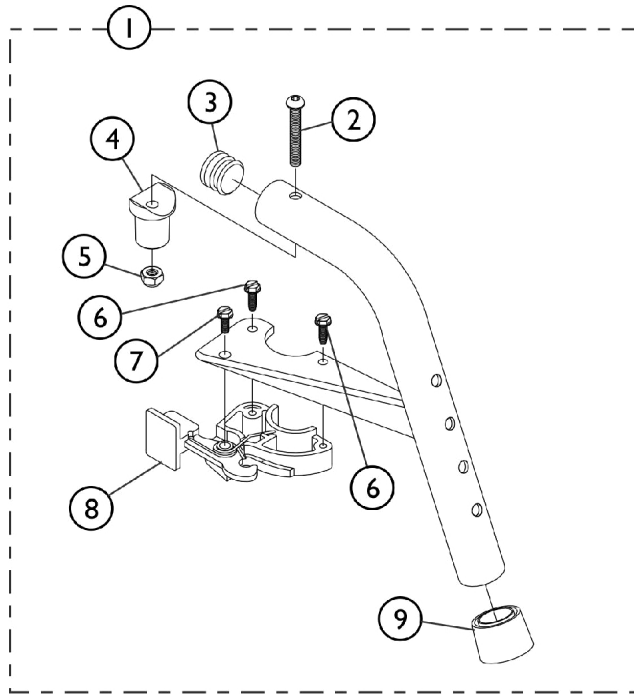

SHEP\_275A

## Footrest Support Assembly 70NHD (Sector Style)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A,B	8881053149	Footrest Support Assembly, 70 Degree Non-Tapered Heavy Duty, Sector, Color - Right	1	1
1	A,B	8881053148	Footrest Support Assembly, 70 Degree Non-Tapered Heavy Duty, Sector, Color - Left	1	1
2		1001762	Screw, Socket Button Head (1/4-20 x 1-3/4")	1	1
3		1029501	Plug Button, Black (1")	1	1
4		1017148	Pivot, Footrest	1	1
5		1025736	Locknut (1/4-20)	1	1
6		1025451	Screw, Slotted Hex Washer Head (#10-32 x 5/8")	1	2
7		1017423	Screw, Slotted Hex Washer Head w/ Patch (#10-32 x 1/2")	1	1
8		1025454	Housing, Latch - Right	1	1
8		1025455	Housing, Latch - Left	1	1
9		1026124	Collar, Anti-Rattle	1	1
10	C	60108343	Kit, Pivot/Slide Tube Mounting Hardware (Pair)	1	1
11			Screw, Hex Head w/ Patch (1/4-20 x 3/4")	1	2
12			Spacer, Coved (1/4 x 11/16 x 1/2")	1	2
13			Plug, Slide Tube Threaded Insert	1	2
	D	1026987	Guard, Impact (8" L)	1	1

NOTE: A - Specify 24P Wet Black frame color when ordering.  
B - Includes items 2-9  
C - Includes items 11-13  
D - Not Shown