
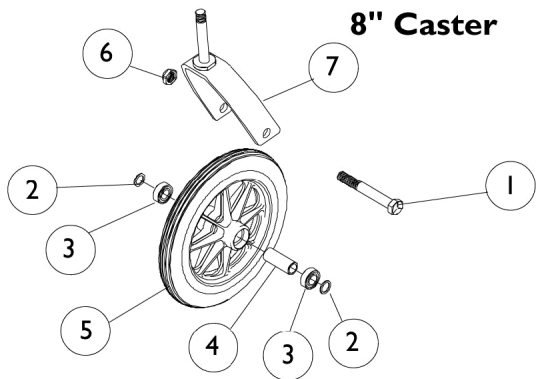


Front Casters and Mounting Hardware

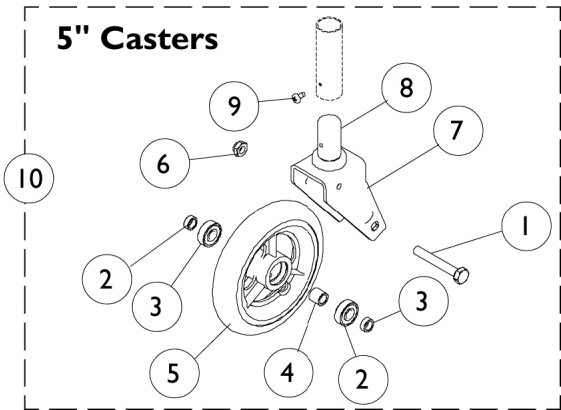

Front Casters and Mounting Hardware

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1031891	Screw, Hex Head (5/16-24 x 2")	1	1
1	B	57667P007	Screw, Hex Head (7/16-20 x 3-1/4")	1	1
2	A,C	1018027-NLA	NLA - Spacer, Axle, 5" Casters (5/16 x 5/8")	1	2
2	D	47183X047	Spacer (5/16 x 1/2 x 3/16")	1	2
2	E	1026101	Spacer (7/16 x 5/8 x 3/16")	1	1
2	F	1016032	Washer, Steel (15/32 x 5/8 x 3/64")	1	1
3	A	40117X026	Bearing, Flange (5/16")	1	2
3	B	1006049	Bearing (7/16")	1	2
4	A	1026172	Spacer (5/16 x 7/16 x 15/32")	1	1
4	E	1018982	Spacer, Hub Caster (7/16 x 5/8 x 1-1/2")	1	1
4	G	1020592	Spacer, Bearing (7/16 x 5/8 x 1-3/4")	1	1
5	H	See Chart	See Chart	1	1
6	A	1025730	Locknut (5/16-24)	1	1
6	A	1107321	Nut, Low Cap (5/16"-24)	1	1
6	B	20225X000	Locknut (7/16-20)	1	1
7		1086613-NLA	NLA - Fork and Stem (5" Caster)	1	1
8		1016682	Bearing, Sleeve Caster	1	1
9		020647	Screw, Phillips Pan Head Tap (#8-32 x 5/16")	1	1
10		1086615-NLA	NLA - Caster Assembly (5")	1	1
	I	See Chart	Front Casters	1	1

NOTE: A - 5" Casters
 B - 8" casters
 C - Before 3-9-99
 D - After 3-8-99
 E - 8 x 7/8" Casters
 F - All other 8" Casters
 G - 8 x 1-1/4" and 8 x 1-3/4" Casters
 H - See chart below titled "#5 Front Casters"
 I - Not Shown

#5 Front Casters

Part Number	Description	Tire	Package Tube/Insert	Quantity
1029496 See Note(s) D, C, A	Caster, Urethane (5")	N/A	N/A	1
1058137 See Note(s) A	Wheel Assembly, Solid Rubber (8" x 1")	1040144	N/A	1
1009481 See Note(s) A	Caster (8x1-1/4") Pneumatic	40176X040	40176X041	1
1030773 See Note(s) A	Pneumatic w/Flat Free Insert	40176X040	1010904-NLA	1
1070811 See Note(s) E	Semi-Pneumatic (8x2")	N/A	N/A	1
1071666 See Note(s) F	Semi-Pneumatic (8x2")	N/A	N/A	2

Front Casters and Mounting Hardware

NOTE: A - Includes bearings and bearing spacers
C - Order in Pairs (#1086615) for chairs manufactured Before 3/9/99.
D - After 3/8/99
E - 1 piece
F - 1 pair