


Internal Assembly

Sieve Beds, 3-Way Valve, Capacitor, Exhaust Muffler, Heat Exchanger

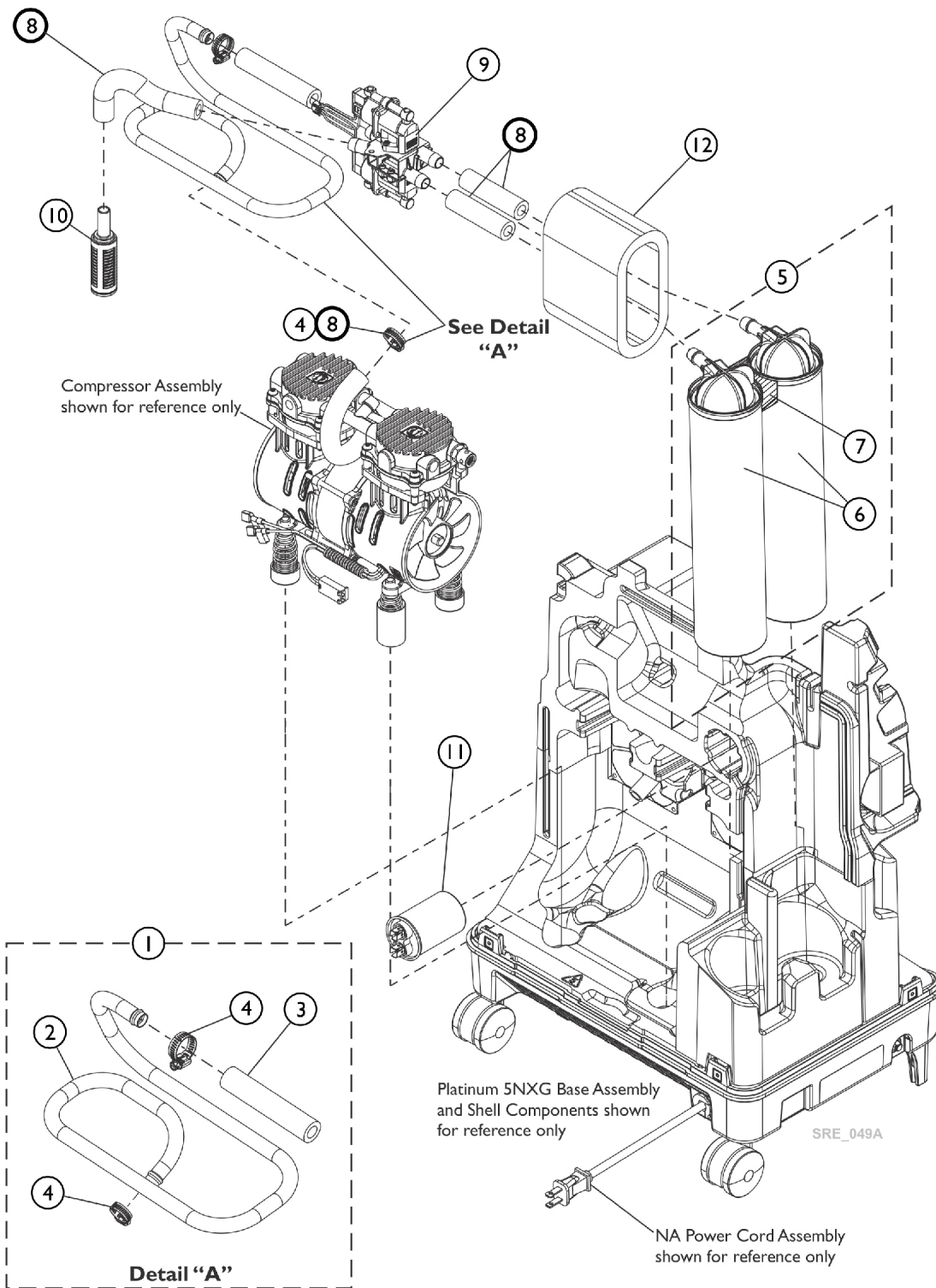

Internal Assembly

Sieve Beds, 3-Way Valve, Capacitor, Exhaust Muffler, Heat Exchanger

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	60127139	Kit, Heat Exchanger Assembly, Platinum 5NXG	1	1
2			Heat Exchanger, Platinum 5NXG	1	1
3			Tube, Silicone (7/16 x 3-29/32" Long)	1	1
4			Clamp, Hose	1	2
			Sheet, Insert Parts Service	1	1
5	B	60127137	Kit, Sieve Bed Assembly (2 ea.)	2	1
6			Sieve Bed Assembly	1	2
7			Foam, Sieve Bed	1	1
			Plug, Sieve Bed Silicone	1	2
			Cap, Sieve Bed Silicone	1	2
			Sheet, Insert Parts Service	1	1
8		60127150	Kit, 3-Way Valve Tubing	1	1
			Tube, Silicone (7/16 x 3" Long)	1	2
			Tube, Valve to Muffler	1	1
			Clamp, Hose	1	1
			Sheet, Insert Parts Service	1	1
9		60127149	Kit, 3-Way Valve Assembly, SMC	1	1
			3-Way Valve Assembly, SMC	1	1
			Sheet, Insert Parts Service	1	1
10		60127151	Kit, Exhaust Muffler	1	1
			Exhaust Muffler	1	1
			Sheet, Insert Parts Service	1	1
11	C	60127145	Kit, Capacitor, 22 MFD, NA	1	1
			Capacitor, 22MFD, NA	1	1
			Sheet, Insert Parts Service	1	1
12		60127497	Kit, Around Valve Foam, 5NXG	1	1
			Foam, Around Valve, 5NXG	1	1
			Sheet, Insert Parts Service	1	1

NOTE: A - Includes items 2-4.
 B - Includes items 6,7, Sieve Bed Silicone Plugs and Caps (2ea.) (not shown)
 C - For North America (NA) models.