


Cabinet Shroud w/ Filters and Retractable Handle Assembly

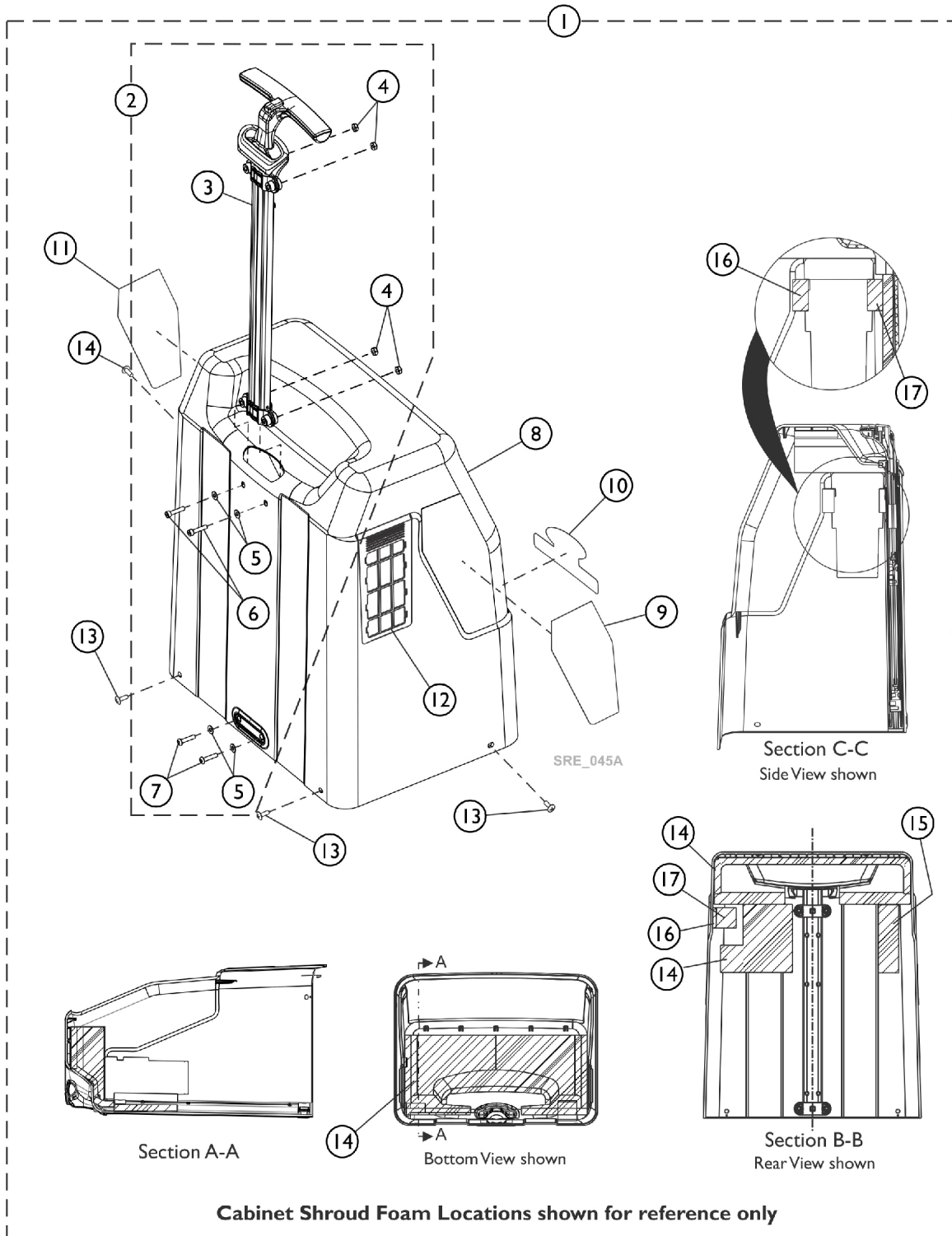

Cabinet Shroud w/ Filters and Retractable Handle Assembly

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	60127160	Kit, Cabinet Shroud and Retractable Handle Assembly, Platinum 5NXG	1	1
2			Retractable Handle Assembly	1	1
8			Shroud, Cabinet, Platinum 5NXG	1	1
9			Label, QSG Platinum NXG	1	1
10			Label, Brand LG Platinum NXG	1	1
11			Label, Troubleshooting, Platinum NXG	1	1
12			Gross Particle Filter Assembly, Platinum 5NXG	1	2
13			Screw, Phillips Truss Head Self Tap (#8-32 x 5/8")	1	4
14			Foam, Cabinet Shroud, Platinum NXG	1	1
15			Foam, Wall, Self-Adhesive, P5NXG (6 x 1-1/2 x 1-1/2")	1	1
16			Foam, Side	1	1
17			Foam, Back	1	1
			Sheet, Insert Parts Service	1	1
2	B	60127162	Kit, Retractable Handle Assembly	1	1
3			Retractable Handle Assembly	1	1
4			Locknut (10-32)	1	4
5			Washer (#10 x 7/16 x 1/32")	1	4
6			Screw, Socket Button Head Cap (10-32 x 7/8")	1	2
7			Screw, Socket Head Cap (10-32 x 7/8")	1	2
			Sheet, Insert Parts Service	1	1
12		60127161	Kit, Gross Particle Filter Assembly, Platinum 5NXG (Pkg. of 2)	2	1
			Gross Particle Filter Assembly, Platinum 5NXG	1	2
			Sheet, Insert Parts Service	1	1

NOTE: A - Includes items 2, 8-17.
B - Includes items 3-7.