
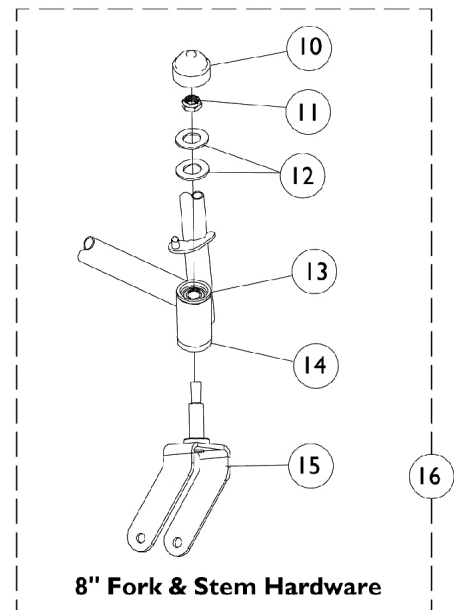
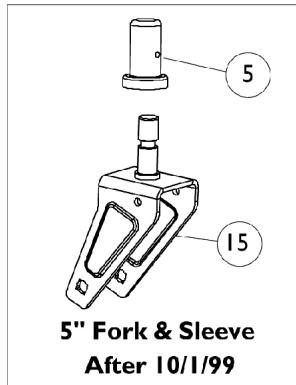


Forks and Hardware

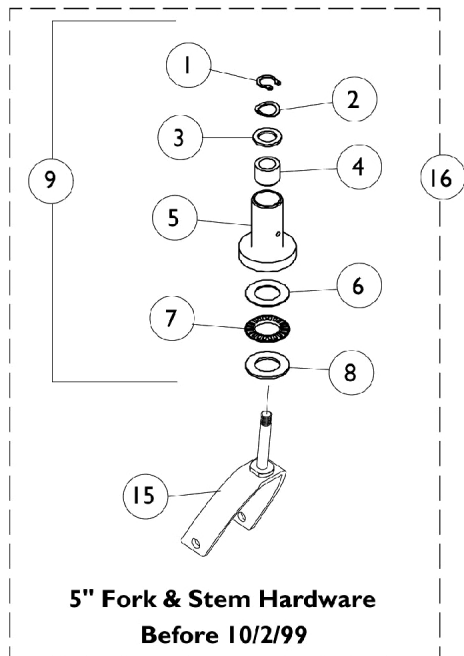

Forks and Hardware

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		40314X002	Ring, Retaining	1	1
2		10141X006	Washer, Wave (1/2 x 5/8 x 1/32")	1	1
3		40157X005	Washer, Nylon (1/2 x 3/4 x 1/16")	1	1
4		40117X008-NLA	NLA - Bearing, Radial	1	1
5	B	57768X020-NLA	NLA - Sleeve, Caster	1	1
5		1016682	Bearing, Sleeve Caster	1	1
6		40157X007	Washer, Steel (5/8 x 1-3/32 x 1/32")	1	1
7		40117X007	Bearing, Thrust Needle	1	1
8		40157X006-NLA	NLA - Washer (5/8 x 1-3/32 x 1/16")	1	1
	A	1027095	Plug Button, Black (7/8")	1	1
9		01008X007-NLA	NLA - Bearing Assembly	1	1
10		1001843	Cap, Head Tube, Black	1	1
10		1131189	Package, Head Tube Cap	10	1
11		20225X000	Locknut (7/16-20)	1	1
12		40157X015-NLA	NLA - Washer, Nylon (9/16 x 1 x 5/8")	1	2
13		1002626-NLA	NLA - Bearing	1	1
14		04268X022-NLA	NLA - Bearing, Lower	1	1
15	D	01007X004-NLA	NLA - Fork and Stem 5" Caster	1	1
15	D	1086613-NLA	NLA - Fork and Stem (5" Caster)	1	1
15	E	04540X264-NLA	NLA - Fork and Stem (8 x 1" and 8 x 1-1/4" Casters)	1	1
15	F	04540X247	Fork and Stem Assembly, Chrome	1	1
16	C	1086614-NLA	Fork and Stem Assembly (5")	1	1
16		04540X265-NLA	NLA - Fork and Stem Assembly (8")	1	1
NOTE: A - Not Shown B - Due to product improvement when replacing 57768X020 sleeve, a pair of no. 1086613 and no. 1016682 Caster sleeves should be ordered. C - Fork and Stem assembly NLA, see page titled "Front Casters and Hardware" item number 10 for replacement. D - 5" Caster E - 8 x 1" and 8 x 1-1/4" Casters F - 8 x 2" Caster					