


Caster - 8" Front - Single Sided Fork

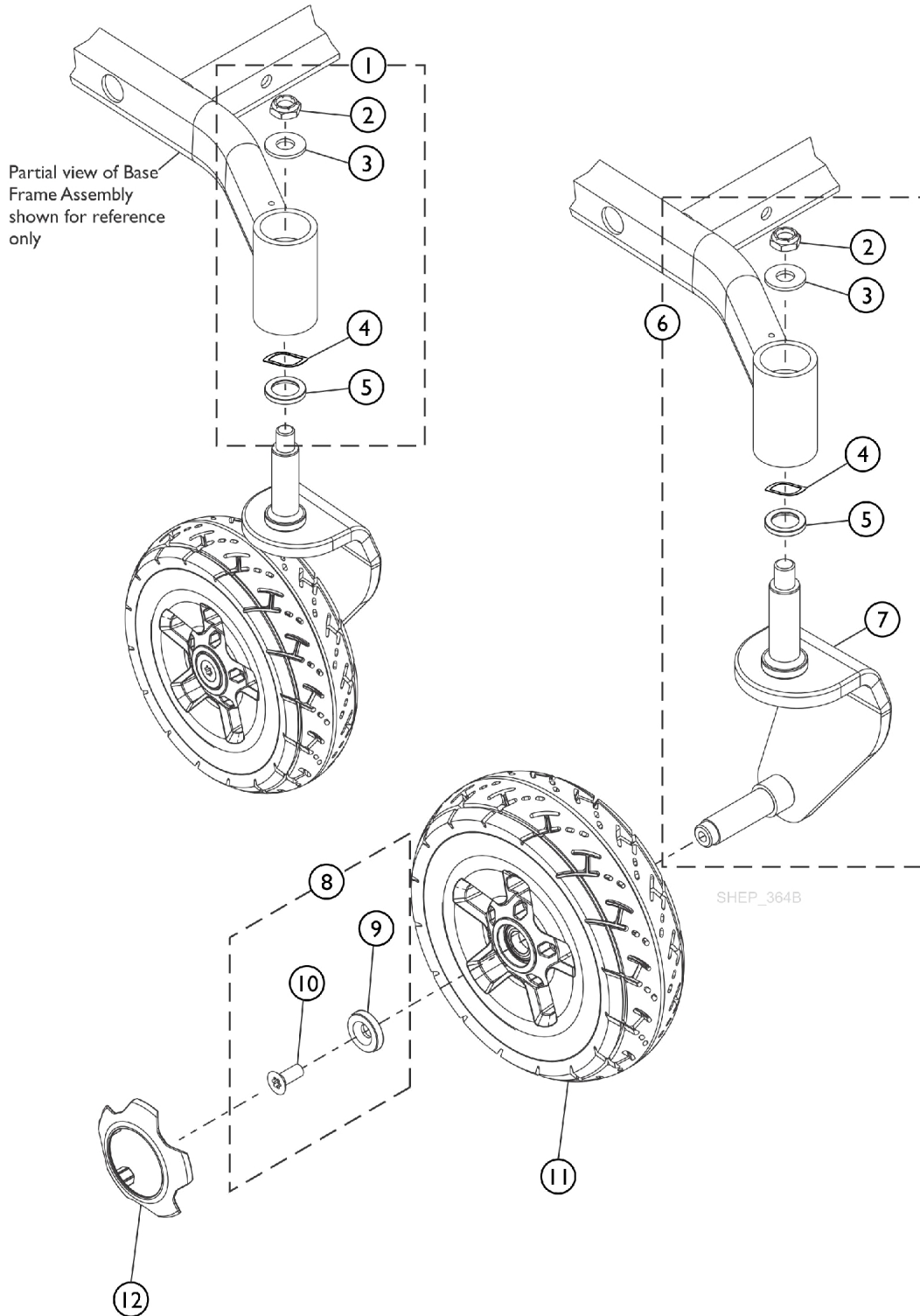

Caster - 8" Front - Single Sided Fork

Item Number	Note	Part Number	Description	Quantity Per		
				Package	Assembly	
1	A	60127931	Kit, Fork Mounting Hardware	1	1	
2			Locknut (1/2-20)	1	1	
3			Washer DIN 125-013 zc	1	1	
4			Washer, Wave (25/32 x 1 x 1/64")	1	1	
5			Spacer, Bronze Fork	1	1	
			Sheet, Insert Parts Service	1	1	
6	B	60127898	Kit, Fork Assembly, Single Sided w/ Hardware (8") - Right	1	1	
2			Locknut (1/2-20)	1	1	
3			Washer DIN 125-013 zc	1	1	
4			Washer, Wave (25/32 x 1 x 1/64")	1	1	
5			Spacer, Bronze Fork	1	1	
7			Fork, Single Sided 8" - Right	1	1	
					Sheet, Insert Parts Service	1
6	B	60127899	Kit, Fork Assembly, Single Sided w/ Hardware (8") - Left	1	1	
2			Locknut (1/2-20)	1	1	
3			Washer DIN 125-013 zc	1	1	
4			Washer, Wave (25/32 x 1 x 1/64")	1	1	
5			Spacer, Bronze Fork	1	1	
7			Fork, Single Sided 8" - Left	1	1	
					Sheet, Insert Parts Service	1
8	C	60124367	Kit, Caster Mounting Hardware	1	1	
9			Spacer, Black (9 x 27 x 5.5mm)	1	1	
10			Screw, DIN7991SVZ TX PB (M8 x 20 x 12.9mm)	1	1	
			Sheet, Insert Parts Service	1	1	
11	D	60124366	Kit, Wheel Assembly, Flat Free, Black (8x2.50-4) Right and Left (Pair)	2	1	
12			See Chart	Item #12 Caster Rim Cap Kit	1	1
12			See Chart	Item #12 Caster Rim Cap Kit (continued)	1	1

NOTE: A - Includes items 2-5
 B - Includes items 2-5 and 7
 C - Includes items 9 and 10
 D - Right and Left side Wheel Assembly sold as pair only.

Item #12 Caster Rim Cap Kit

Description	Obsidian Black (170PR)	Ruby Red (171PR)	Peacock Blue (172PR)	Pink Flamingo (173PR)	Purple Passion (174PR)	Denim Navy (175PR)
Caster Rim Cap Kit	60129184 See Note(s) A	60129185 See Note(s) A	60129186 See Note(s) A	60129187 See Note(s) A	60129188 See Note(s) A	60129189 See Note(s) A

NOTE: A - Includes 1 ea. Caster Rim Cap

Item #12 Caster Rim Cap Kit (continued)

Caster - 8" Front - Single Sided Fork

Item #12 Caster Rim Cap Kit (continued)

Description	Lime Twist (176PR)	Artic Blue (177PR)	Banana Split (178PR)	Orange Marmalade (179PR)	Morning Mist (180PR)
Caster Rim Cap Kit	60129190 See Note(s) A	60129191 See Note(s) A	60129192 See Note(s) A	60129193 See Note(s) A	60129194 See Note(s) A

NOTE: A - Includes 1 ea. Caster Rim Cap