



# Drive Wheels and Mounting Hardware (14" x 3")

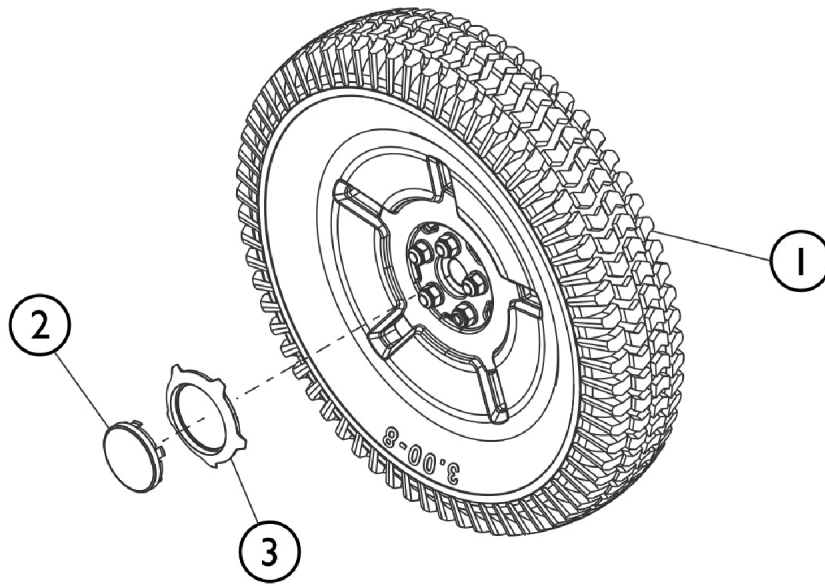

Motor/Gearbox Assembly  
shown for reference only

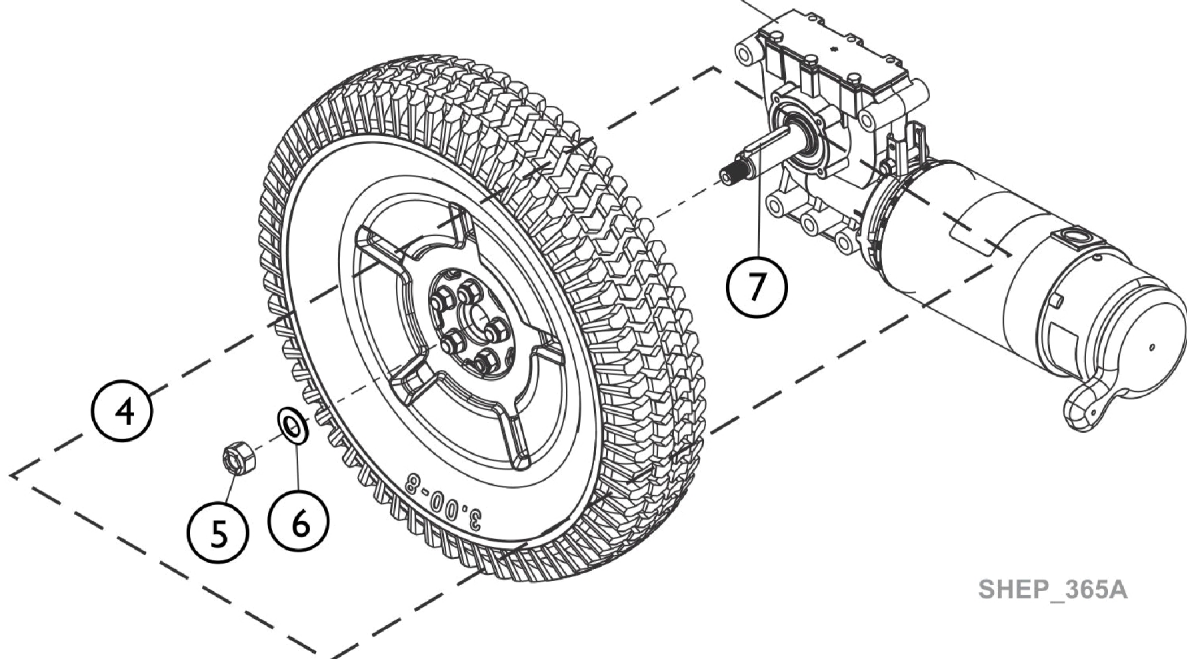

SHEP\_365A

## Drive Wheels and Mounting Hardware (14" x 3")

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	60124365	Kit, Drive Wheel, Flat Free w/ Hub Assembly, Black (14" x 3")	1	1
1	A	60127943	Kit, Drive Wheel, Pneumatic w/ Hub Assembly, Black (14" x 3")	1	1
2		60129164	Kit, Drive Wheel Hub Cap, Black	1	1
3		See Chart	Item #3 Drive Wheel Rim Cap Kit	1	1
3		See Chart	Item #3 Drive Wheel Rim Cap Kit (continued)	1	1
4	B	60129154	Kit, Drive Wheel Mounting Hardware	1	1
5			Locknut (M12 x 1.5)	1	1
6			Washer, Belleville (12 x 23 x 1.5mm)	1	1
7			Key (6mm x 6mm x 40mm)	1	1
7			Sheet, Insert Parts Service	1	1
7		1023264	Key (6mm x 6mm x 40mm)	1	1

NOTE: A - Sold Complete.  
B - Includes items 5-7.

### Item #3 Drive Wheel Rim Cap Kit

Description	Obsidian Black (170PR)	Ruby Red (171PR)	Peacock Blue (172PR)	Pink Flamingo (173PR)	Purple Passion (174PR)	Denim Navy (175PR)
Drive Wheel Rim Cap Kit	60129173 See Note(s) A	60129174 See Note(s) A	60129175 See Note(s) A	60129176 See Note(s) A	60129177 See Note(s) A	60129178

NOTE: A - Includes 1 ea. Drive Wheel Rim Cap.

### Item #3 Drive Wheel Rim Cap Kit (continued)

Description	Lime Twist (176PR)	Artic Blue (177PR)	Banana Split (178PR)	Orange Marmalade (179PR)	Morning Mist (180PR)
Drive Wheel Rim Cap Kit	60129179 See Note(s) A	60129180 See Note(s) A	60129181 See Note(s) A	60129182 See Note(s) A	60129183 See Note(s) A

NOTE: A - Includes 1 ea. Drive Wheel Rim Cap.