


Compressor Rebuild Kit - NA Models

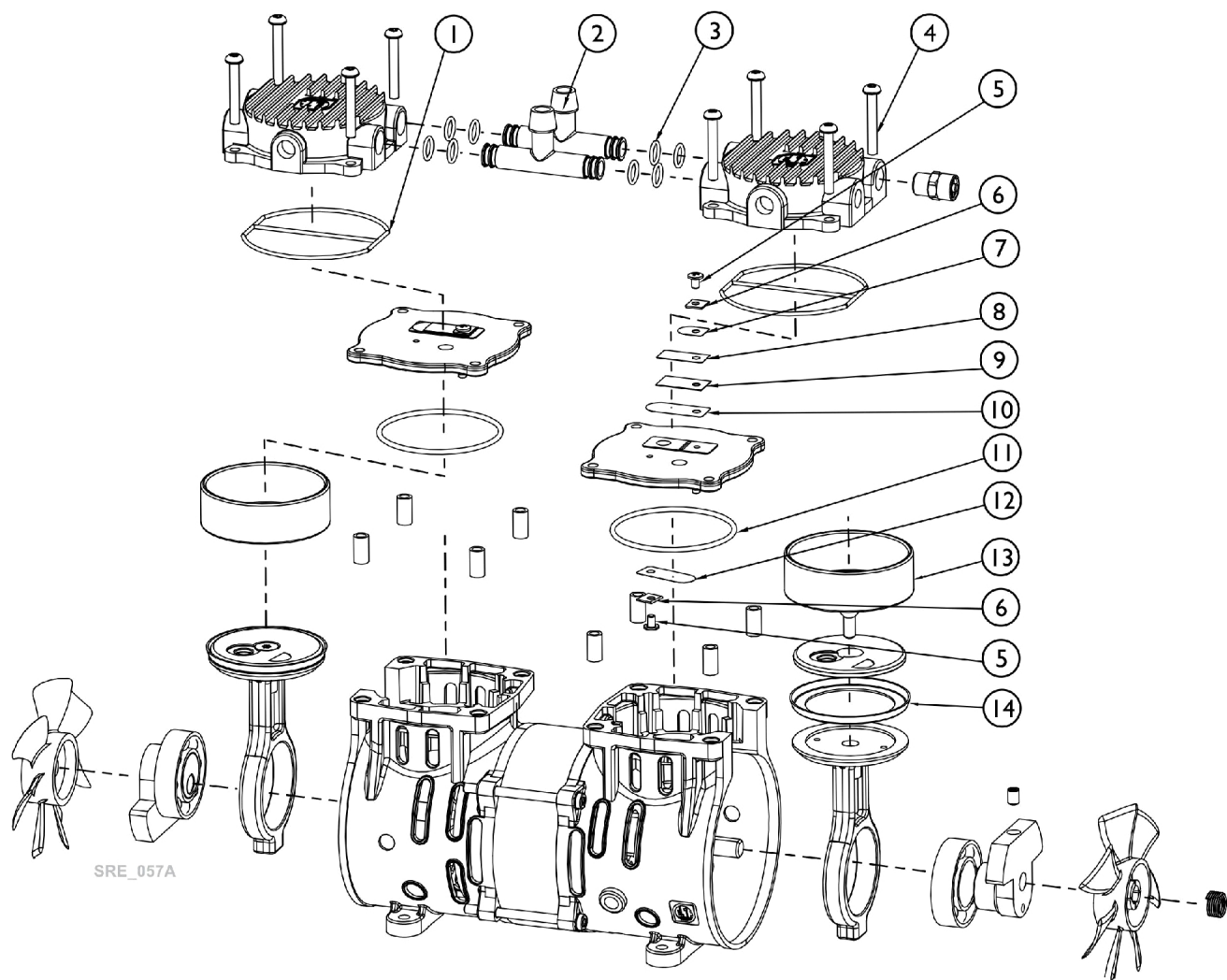

Compressor Rebuild Kit - NA Models

NOTE: Individual Kit Components are Non-Saleable. Description of components provided for reference only.

Item Number	Description	Quantity	P5NXG North America (NA) Model
1-14 See Note(s) A	Kit, Compressor Rebuild, P5NXG	1	60127500
1.	Head Seal, O-Ring	2	X
2.	Tube	2	X
3.	Tube, O-Ring	8	X
4.	Screw, Head	8	X
5.	Screw, Valve Flapper	4	X
6.	Washer, Square	4	X
7.	Valve Keeper Strip, Short	2	X
8.	Valve Keeper Strip, Long	2	X
9.	Valve Keeper Strip, Plastic	2	X
10.	Valve Flapper, Top	2	X
11.	Valve Plate, O-Ring Round	2	X
12.	Valve Flapper, Bottom	2	X
13.	Cylinder	2	X
14.	Piston Cup Seal	2	X
NOTE: A - Kit includes items 1-14.			