


Aquatec Ocean Shower Commode

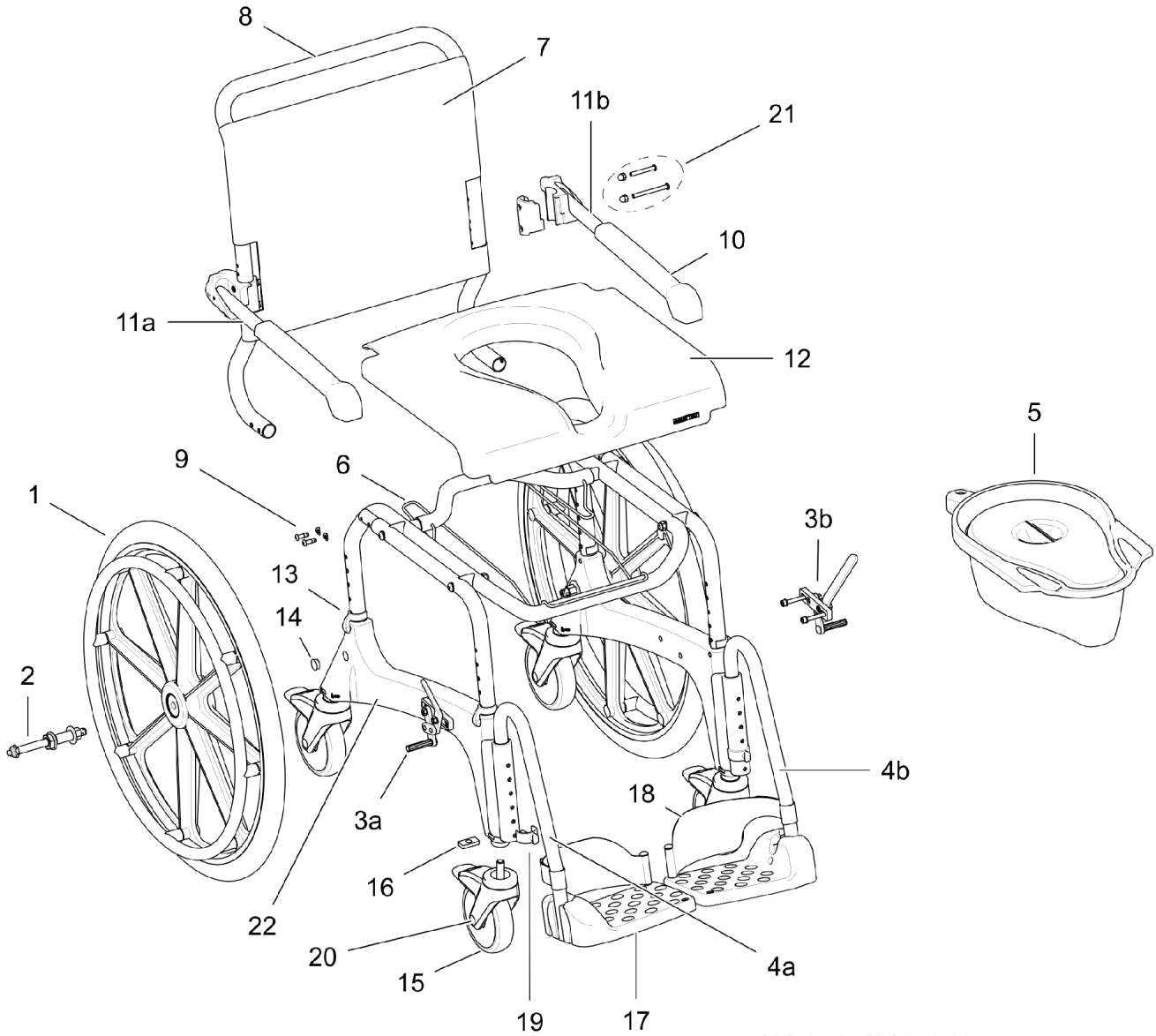

SPC_075A - SPS1542894

Aquatec Ocean Shower Commode

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		16336	24 In Wheel W / Handrim for Ocean/Ergo SP	1	2
2		16418	Cpl Set Hub and QR Axle (16338 and 16337)	1	2
1-2		16379	Self Propelling Wheels Kit complete including brakes	1	1
3a		16340	Rt. Brk. Lever Ocean / Ergo Commodes	1	1
3b		16339	Lt. Brk. Lever - Ocean / Ergo Commodes	1	1
4a	A	SP1643542	Footrest cpl., Right - Ocean (XL) / VIP / Ocean Ergo Series	1	1
4b	B	SP1643541	Footrest cpl., Left - Ocean (XL) / (Dual-)VIP / Ocean Ergo Series	1	1
5		10230	Collection Pan with Lid	1	1
9		1542900	Bkrst Hdwr. - Ergo / Ocean (Dual-)VIP / XL	1	1
10		16472	Armrest Pads Incl. Screws Kit	1	2
11a		1470693	Armrest assembly, Right	1	1
11b		1470694	Armrest Assembly Left	1	1
13		16256	Side Frame Adj. Clip	1	1
15		16259	Caster, Locking, Red Tab	1	4
17		1470689	Footrest Plate	1	2
18		16334	Heel Loop (Pair)	2	1
21		1471052	Ocean Armrest Kit Parts	1	1
NOTE: A - Replaces 1470696 B - Replaces 1470695					
Items not listed are non-saleable					