


# Wheel

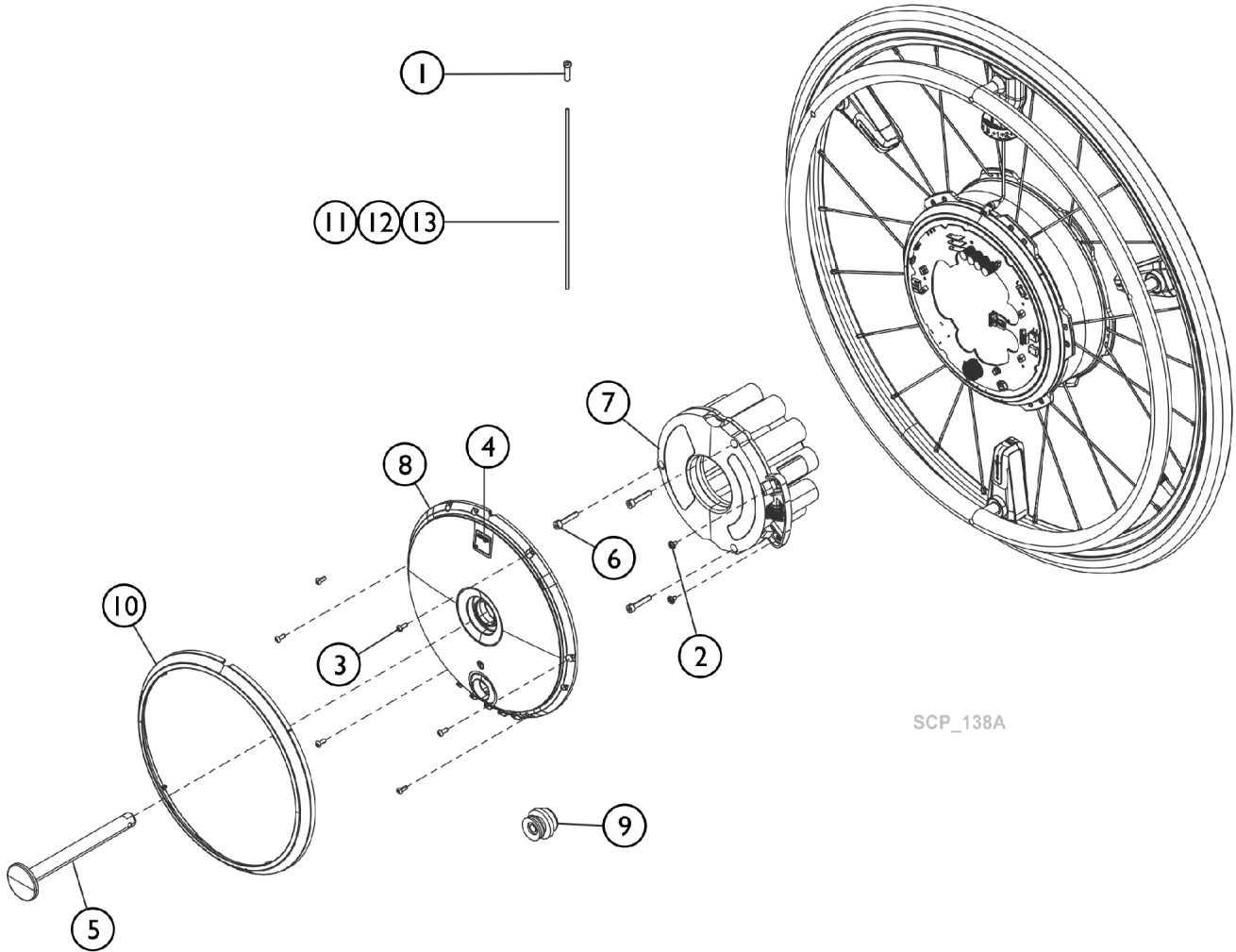

SCP\_138A

# Wheel

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1467215	Nipple: brass Polyax 14mm for 2mm spoke, silver	1	--
2		1467526	Talow drop screw Torx M 3 x 5 - 8.8	1	--
3		1467743	ISO 7380 - M 3 x 8 - 8.8 - ISR vz	1	--
4		1490801	Alber Logo 17 x 17mm	1	--
5		1593669	Stub axle e-motion M25 with signal ring	1	--
6		1565957	Cylinder-head screw (DIN 912) ISO 4762 M4x22-8.8 A2K	1	--
7		1591781	Battery (Corepack) M25 4,3Ah (red)	1	--
8		1592381	Cover mounted	1	--
9		1592420	Charging connector	1	--
10		1592588	Ring mounted	1	--
11		1592647	Spoke set M25 24 „consisting of 5 spokes + nipple	1	--
12		1592648	Spoke set M25 25“ 2,0mm 5 spokes + nipples	1	--
13		1592827	Spoke set M25 22“ 2,0mm	1	--