


Anti-Tippers until 2019

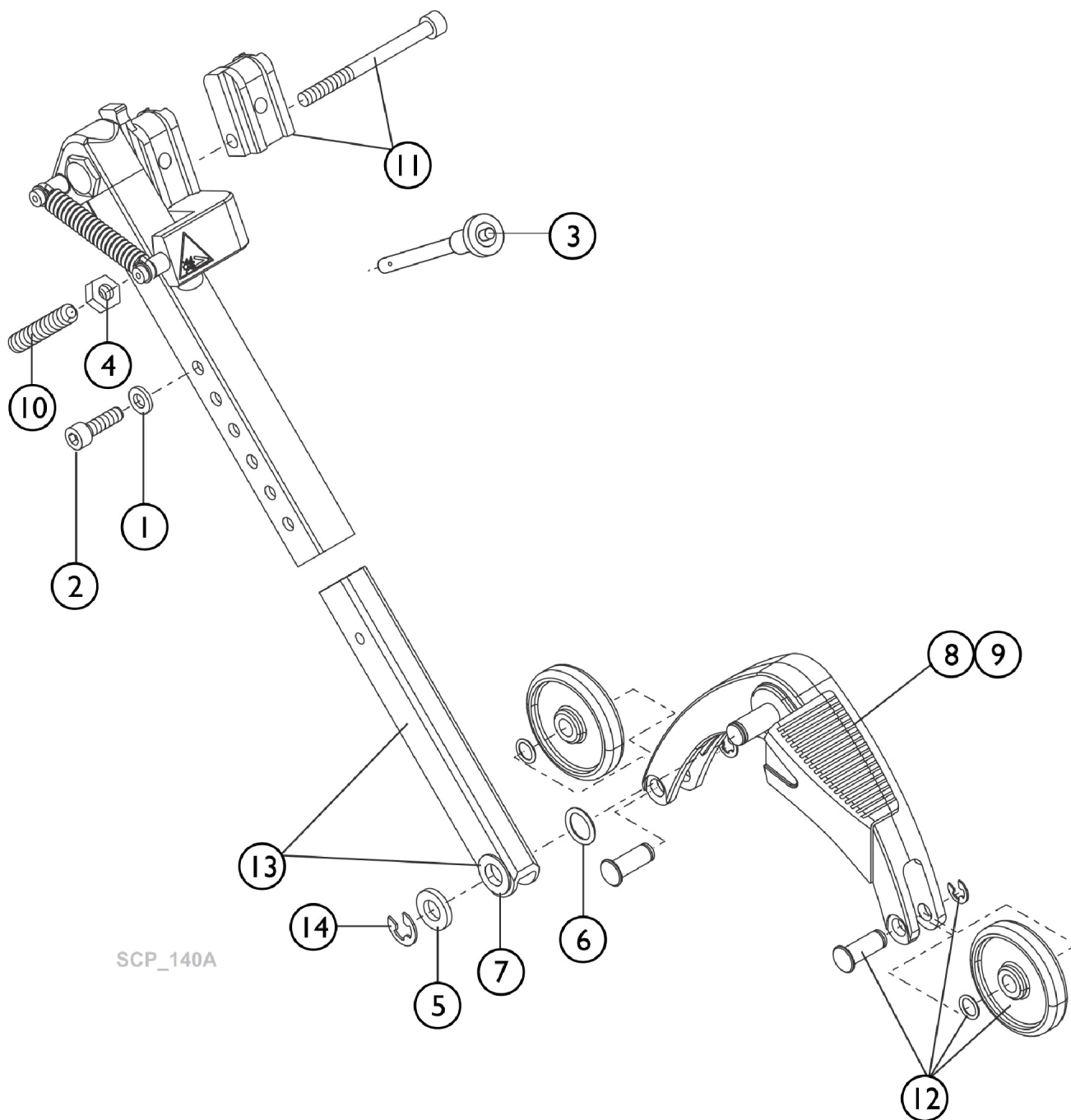

SCP_140A

Anti-Tippers until 2019

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1456183	Washer DIN 125 - A 6,4	1	--
2		1456233	Cylinder head screw M 6 x 20	1	--
3		1456406	Quickpin quick release bolt	1	--
4		1459636	Hexagon nut ISO 4032 - M8 vz	1	--
5		1459944	Washer DIN 988 - 10 x 16 x 1	1	--
6		1463698	DIN 988 - 10 x 16 x 0,3 - St	1	--
7		1467811	Friction bearing Igus form F GFM-1212-035	1	--
8		1467845	Support angle right for anti-tippers	1	--
9		1467846	Support angle left for anti-tippers	1	--
10		1467943	Threaded pin M 8 x 45 DIN 913 - 45 H	1	--
11		1489221	Hardware set for anti-tippers expanded version 4 x 12mm shoe inc. screws	1	--
12		1489365	Wheel for anti-tippers complete	1	--
13		1489366	Slide rod anti-tippers mounted	1	--
14		1519603	Securing disk DIN 6799-Gr.8 vz	1	--