



# Shrouds, Rear Top and Lower Rear, Polycarbonate

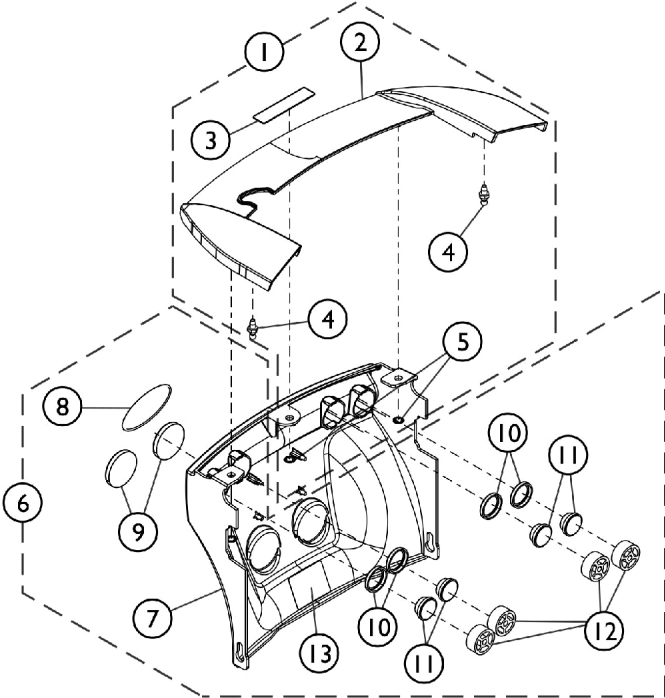

SHEP\_323A

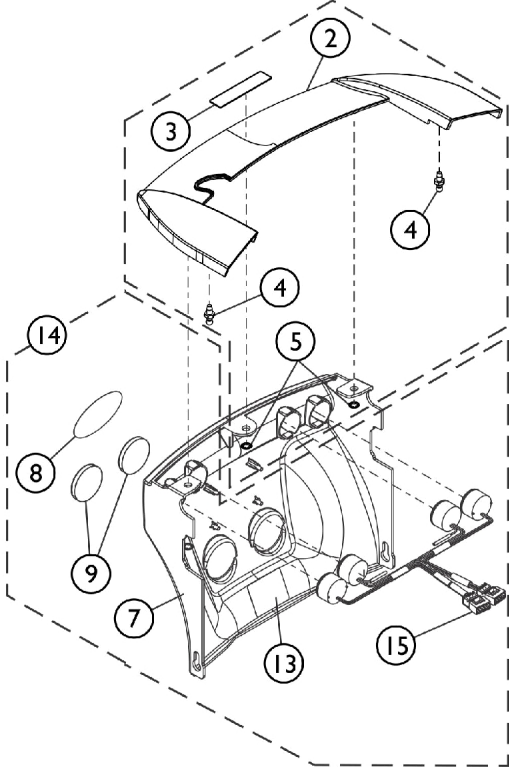

## Shrouds, Rear Top and Lower Rear, Polycarbonate

Item Number	Note	Part Number	Description	Quantity Per			
				Package	Assembly		
1	A	See Chart	Item #1 Rear Top Shroud w/ Hardware Kit	1	1		
1	A	See Chart	Item #1 Rear Top Shroud w/ Hardware Kit (continued)	1	1		
6	B	60130235	Kit, Shroud, Rear Lower w/o Lights	1	1		
7			Shroud, Rear Lower	1	1		
8			Decal, Invacare Medallion, Reflective - Large	1	1		
9			Reflector, Red	1	2		
10			Reflector, Ring	1	2		
11			Reflector, Red	1	2		
12			Spacer, Reflector	1	2		
13			Label, Warning Motor/SSD Gearbox Replacement Sheet, Insert Parts Service	1	1		
14			C	60130236	Kit, Shroud, Rear Lower w/ Lights	1	1
7					Shroud, Rear Lower	1	1
8					Decal, Invacare Medallion, Reflective - Large	1	1
9					Reflector, Red	1	2
13					Label, Warning Motor/SSD Gearbox Replacement Sheet, Insert Parts Service	1	1
15	Harness, Rear Lights, AVIVA FX	1			1		
	Sheet, Insert Parts Service	1			1		

NOTE: A - Includes items 2-5  
 B - Includes items 7-13  
 C - Includes items 7-9, 13 and 15.

### Item #1 Rear Top Shroud w/ Hardware Kit

Description	Obsidian Black	Ruby Red	Denim Navy	Peacock Blue	Purple Passion	Lime Twist
Rear Top Shroud w/ Hardware Kit	60130237	60130238	60130239	60130240	60130241	60130242
	See Note(s) A	See Note(s) A	See Note(s) A	See Note(s) A	See Note(s) A	See Note(s) A

NOTE: A - Includes items 2-5.

### Item #1 Rear Top Shroud w/ Hardware Kit (continued)

Description	Pink Flamingo	Arctic Blue
Rear Top Shroud w/ Hardware Kit	60130243	60130244
	See Note(s) A	See Note(s) A

NOTE: A - Includes items 2-5.