


Caster - 8" Rear - Single Sided Fork, Polycarbonarbonate

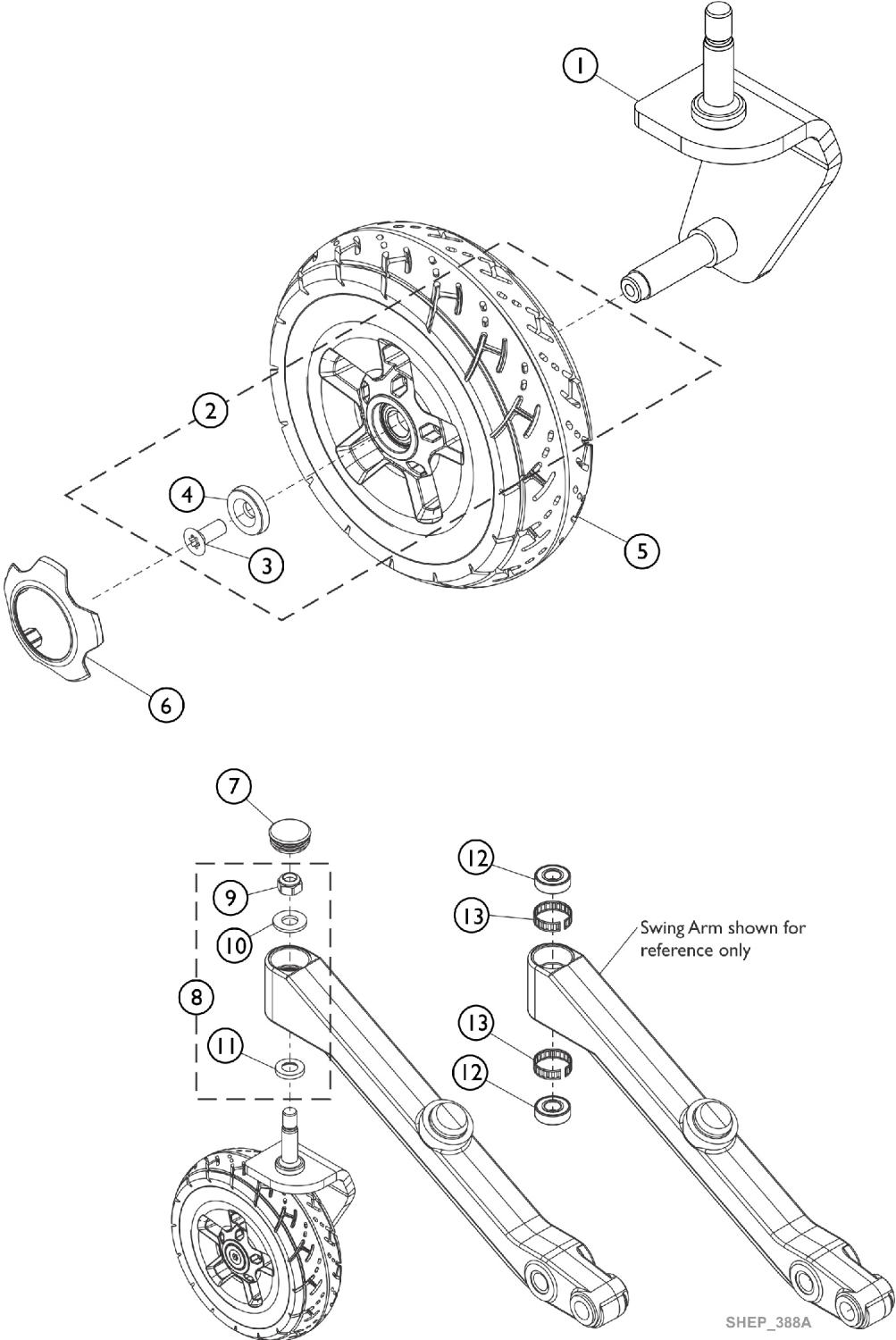

Caster - 8" Rear - Single Sided Fork, Polycarbonate

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		60124368	Kit, Single Sided Fork Assembly - Right	1	1
1		60124369	Kit, Single Sided Fork Assembly - Left	1	1
2	A	60124367	Kit, Caster Mounting Hardware	1	1
3			Screw, DIN7991SVZ TX PB (M8 x 20 x 12.9mm)	1	1
4			Spacer, Black (9 x 27 x 5.5mm)	1	1
5	C	60124366	Kit, Wheel Assembly, Flat Free, Black (8x2.50-4) Right and Left (Pair)	2	1
6		See Chart	Item #6 Caster Rim Cap Kit, Polycarbonate	1	1
6		See Chart	Item #6 Caster Rim Cap Kit (continued), Polycarbonate	1	1
7		1110709	Cap, Head Tube, Black	1	1
8	B	60124370	Kit, Fork Mounting Hardware	1	1
9			Locknut (M14 x 2mm)	1	1
10			Washer, Nylon (5/8 x 1-5/16 x 1/8")	1	1
11			Spacer, Stem (5/8 x 1-1/8 x 3/16")	1	1
12		40117X006	Bearing (5/8")	1	2
13		1110717	Ring, Tolerance	1	2

NOTE: A - Includes items 3 and 4.
B - Includes items 9-11.
C - Right and Left side Wheel Assembly sold as pair only.

Item #6 Caster Rim Cap Kit, Polycarbonate

Description	Obsidian Black	Ruby Red	Peacock Blue	Pink Flamingo	Purple Passion	Denim Navy
Caster Rim Cap Kit	60129184 See Note(s) A	60129185 See Note(s) A	60129186 See Note(s) A	60129187 See Note(s) A	60129188 See Note(s) A	60129189 See Note(s) A

NOTE: A - Includes 1 ea. Caster Rim Cap

Item #6 Caster Rim Cap Kit (continued), Polycarbonate

Description	Lime Twist	Artic Blue	Banana Split	Orange Marmalade	Morning Mist
Caster Rim Cap Kit	60129190 See Note(s) A	60129191 See Note(s) A	60129192 See Note(s) A	60129193 See Note(s) A	60129194 See Note(s) A

NOTE: A - Includes 1 ea. Caster Rim Cap