


Drive Wheels and Mounting Hardware (14" x 3"), Polycarbonate

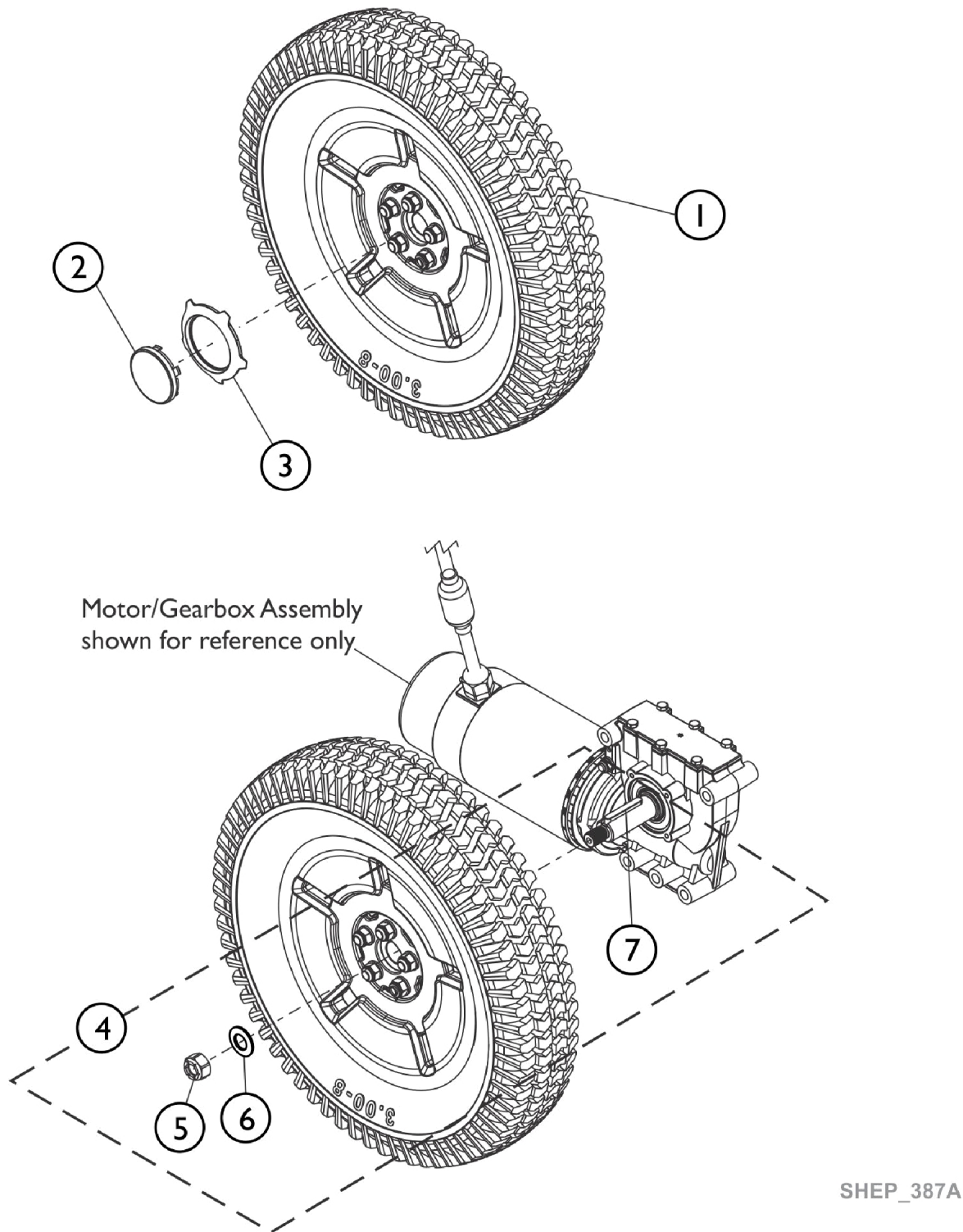

SHEP_387A

Drive Wheels and Mounting Hardware (14" x 3"), Polycarbonate

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	60124365	Kit, Drive Wheel, Flat Free w/ Hub Assembly, Black (14" x 3")	1	1
2		60129164	Kit, Drive Wheel Hub Cap, Black	1	1
3		See Chart	Item #3 Drive Wheel Rim Cap Kit, Polycarbonate	1	1
3		See Chart	Item #3 Drive Wheel Rim Cap Kit (continued), Polycarbonate	1	1
4	B	60129154	Kit, Drive Wheel Mounting Hardware	1	1
5			Locknut (M12 x 1.5)	1	1
6			Washer, Belleville (12 x 23 x 1.5mm)	1	1
7			Key (6mm x 6mm x 40mm)	1	1
			Sheet, Insert Parts Service	1	1
7		1023264	Key (6mm x 6mm x 40mm)	1	1

NOTE: A - Sold Complete.
B - Includes items 5-7.

Item #3 Drive Wheel Rim Cap Kit, Polycarbonate

Description	Obsidian Black	Ruby Red	Peacock Blue	Pink Flamingo	Purple Passion	Denim Navy
Drive Wheel Rim Cap Kit	60129173 See Note(s) A	60129174 See Note(s) A	60129175 See Note(s) A	60129176 See Note(s) A	60129177 See Note(s) A	60129178 See Note(s) A

NOTE: A - Includes 1 ea. Drive Wheel Rim Cap.

Item #3 Drive Wheel Rim Cap Kit (continued), Polycarbonate

Description	Lime Twist	Artic Blue	Banana Split	Orange Marmalade	Morning Mist
Drive Wheel Rim Cap Kit	60129179 See Note(s) A	60129180 See Note(s) A	60129181 See Note(s) A	60129182 See Note(s) A	60129183 See Note(s) A

NOTE: A - Includes 1 ea. Drive Wheel Rim Cap.