


Wheel Lock - Recliner

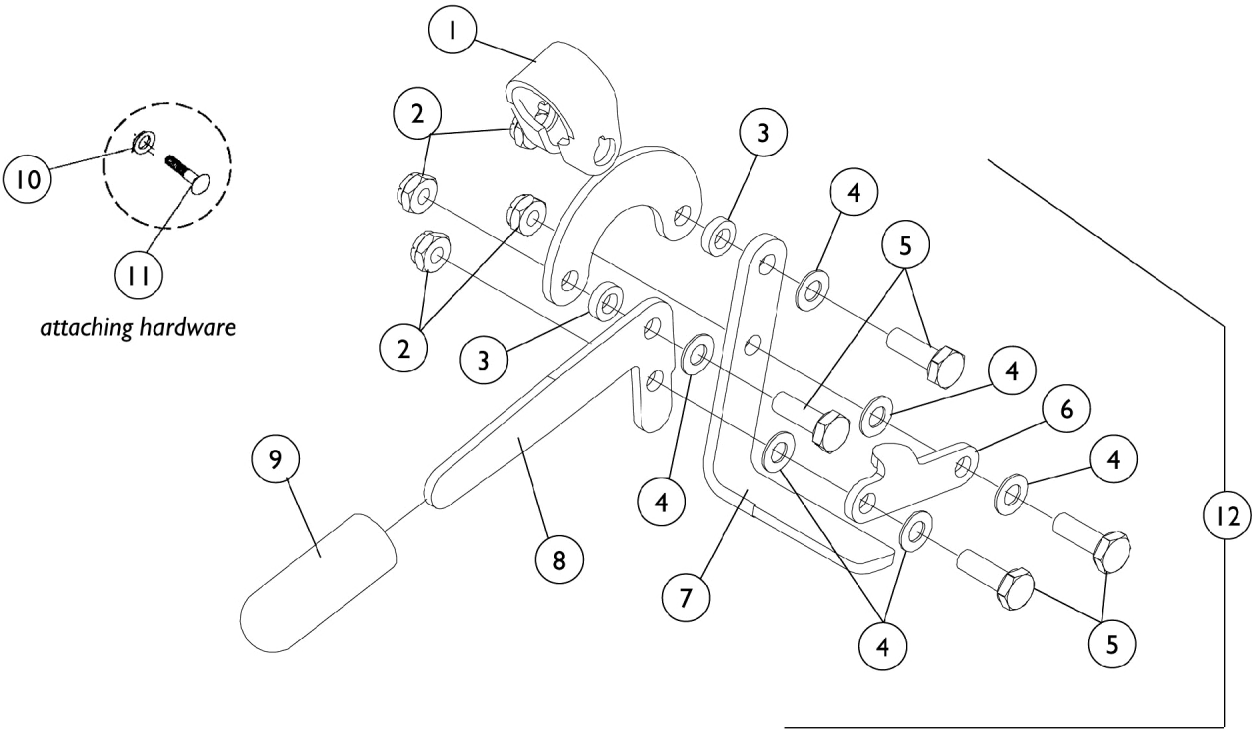

Wheel Lock - Recliner

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		04512X113-NLA	NLA - Bracket, Mounting, Chrome - Right	1	1
1		04512X114-NLA	NLA - Bracket, Mounting, Chrome - Left	1	1
2		7930367-NLA	NLA - Locknut (1/4-28)	25	4
3		40157X004	Spacer, Nylon (1/4 x 7/16 x 1/8")	1	2
4		7930357-NLA	NLA - Washer, Nylon (1/4 x 1/2 x 1/32")	25	6
5		60274X012-NLA	NLA - Screw, Hex Head (1/4-28 x 3/4")	1	4
6		47559X025-NLA	NLA - Link, Chrome - Right	1	1
6		47559X026-NLA	NLA - Link, Chrome - Left	1	1
7		1015443-NLA	NLA - Shoe	1	1
8		1023296-NLA	NLA - Handle, Wheel Lock, Chrome	1	1
9		47376X024	Tip, Rubber Wheel Lock Handle, Black	1	1
10		1107321	Nut, Low Cap (5/16"-24)	1	1
11		11232X009	Bolt, Lug (5/16-24 x 1-3/8")	1	1
12	A	1021068	Wheel Lock Assembly w/o Hardware, Black - Right	1	1
12	A	1021069	Wheel Lock Assembly w/o Hardware, Black - Left	1	1
NOTE: A - Pull to Lock (shown), Includes Items 1-9. Chrome is No Longer Available. Order right and left side in Black to match.					