


One Piece Footrest - Adjustable

89

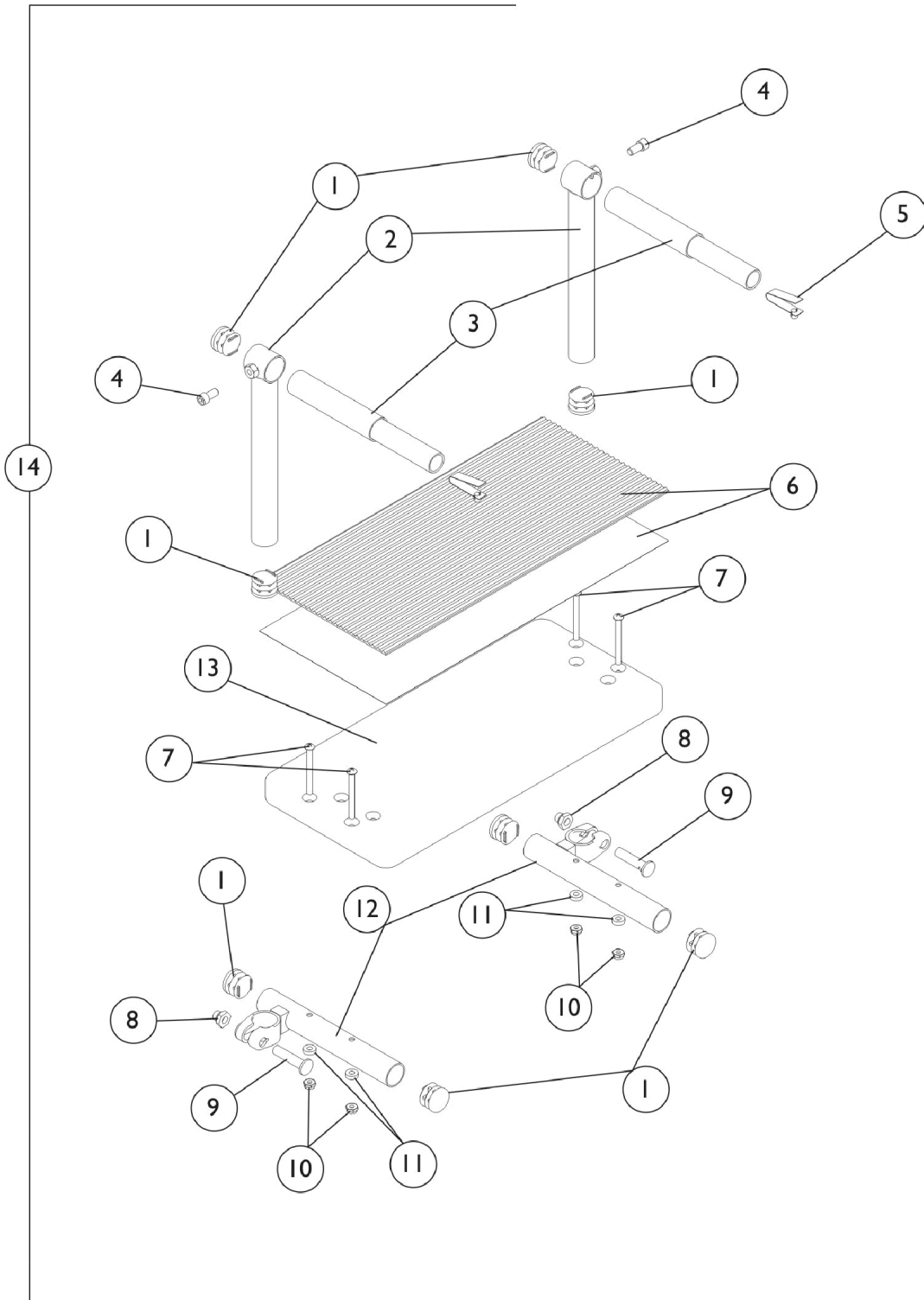

One Piece Footrest - Adjustable

89

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	04685X001-NLA	NLA - Button, Spring Push	1	1
1	B	40188X009	Button, Single Snap	1	1
2		04470X136-NLA	NLA - Tube, Pivot Spacer	1	1
3		04470X137-NLA	NLA - Tube, Pivot, Insert	1	1
4		60284X001	Screw, Socket Head (1/4-20 x 1/2")	1	1
6	C	48247X011-NLA	NLA - Footboard w/o Hardware	1	1
6	D	48247X012-NLA	NLA - Footboard w/o Hardware	1	1
6	E	48247X013-NLA	NLA - Footboard w/o Hardware	1	1
7		40183X004-NLA	NLA - Screw (#10-32 x 1-7/8")	1	2
8		1107321	Nut, Low Cap (5/16"-24)	1	1
9		11232X009	Bolt, Lug (5/16-24 x 1-3/8")	1	1
10		7930365-NLA	NLA - Locknut (#10-32)	25	2
11		16347X000-NLA	NLA - Washer, Nylon (3/16 x 7/16 x 1/8")	1	2
12		04470X138-NLA	NLA - Support, Footboard	1	1
13	C	1017528-NLA	NLA - Mat Footboard	1	1
13	D	1016951-NLA	NLA - Mat Footboard	1	1
13	E	1016952-NLA	NLA - Mat Footboard	1	1
14	C	04706X079-NLA	NLA - Footrest Assembly, One Piece Adjustable, Chrome	1	1
14	D	04706X080-NLA	NLA - Footrest Assembly, One Piece Adjustable, Chrome	1	1
14	E	04706X081-NLA	NLA - Footrest Assembly, One Piece Adjustable, Chrome	1	1
NOTE: A - Before 7/98 B - After 6/98 C - 12" Wide Chairs D - 14" Wide Chairs E - 16" Wide Chairs					