


One Piece Footrest - Non-Adjustable

95

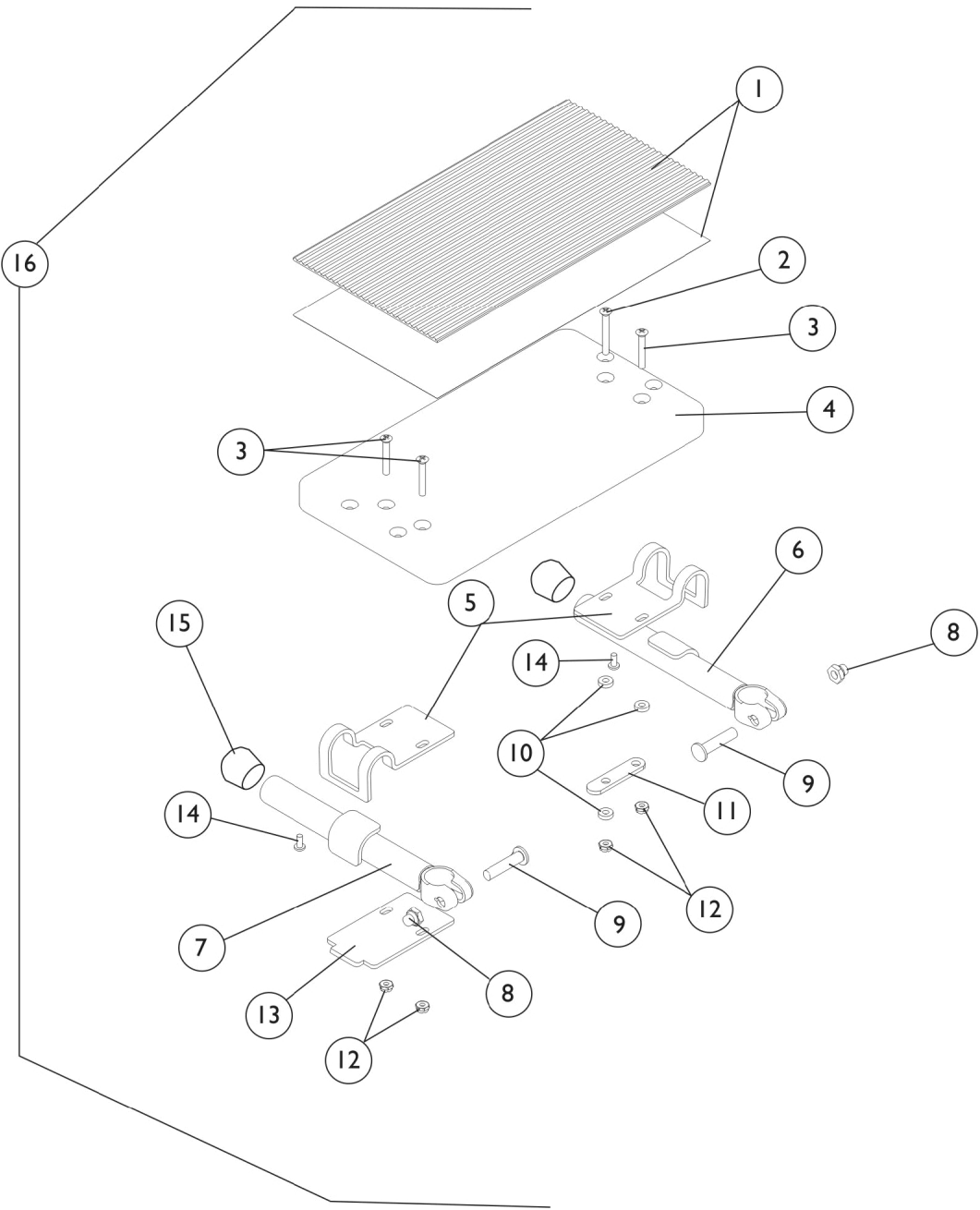

One Piece Footrest - Non-Adjustable

95

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1017528-NLA	NLA - Mat Footboard	1	1
1	B	1016951-NLA	NLA - Mat Footboard	1	1
1	C	1016952-NLA	NLA - Mat Footboard	1	1
2		18860X005-NLA	NLA - Screw, Phillips Oval Head (#10-32 x 1-1/2")	1	1
3		11008X002-NLA	NLA - Screw, Phillips Oval Head (#10-32 x 1-1/4")	1	3
4	A	48247X011-NLA	NLA - Footboard w/o Hardware	1	1
4	B	48247X012-NLA	NLA - Footboard w/o Hardware	1	1
4	C	48247X013-NLA	NLA - Footboard w/o Hardware	1	1
5		47352X004	Clamp, Black	1	2
6		04470X052-NLA	NLA - Tube, Pivot - Right	1	1
7		04470X053-NLA	NLA - Tube, Pivot - Left	1	1
8		1107321	Nut, Low Cap (5/16"-24)	1	2
9		11232X009	Bolt, Lug (5/16-24 x 1-3/8")	1	2
10		16347X000-NLA	NLA - Washer, Nylon (3/16 x 7/16 x 1/8")	1	3
11		47559X030-NLA	NLA - Link, Locking, Chrome	1	1
12		60279X003	Locknut (#10-32)	1	4
13		47352X003	Lock, Pad Plate, Legrest, Black	1	1
14		40120X001	Screw, Phillips Pan Head (#8-32 x 3/8")	1	2
15		1016346	Cap, End, Tube, Black (7/8")	1	2
16	A	1017530-NLA	NLA - Footrest Assembly, One Piece Non-Adjustable, Chrome	1	1
16	B	1016955-NLA	NLA - Footrest Assembly, One Piece Non-Adjustable, Chrome	1	1
16	C	1016956	Footrest Assembly, One Piece Non-Adjustable, Chrome	1	1

NOTE: A - 12" Wide Chairs
 B - 14" Wide Chairs
 C - 16" Wide Chairs