


Rail Shrouds and Front Shroud

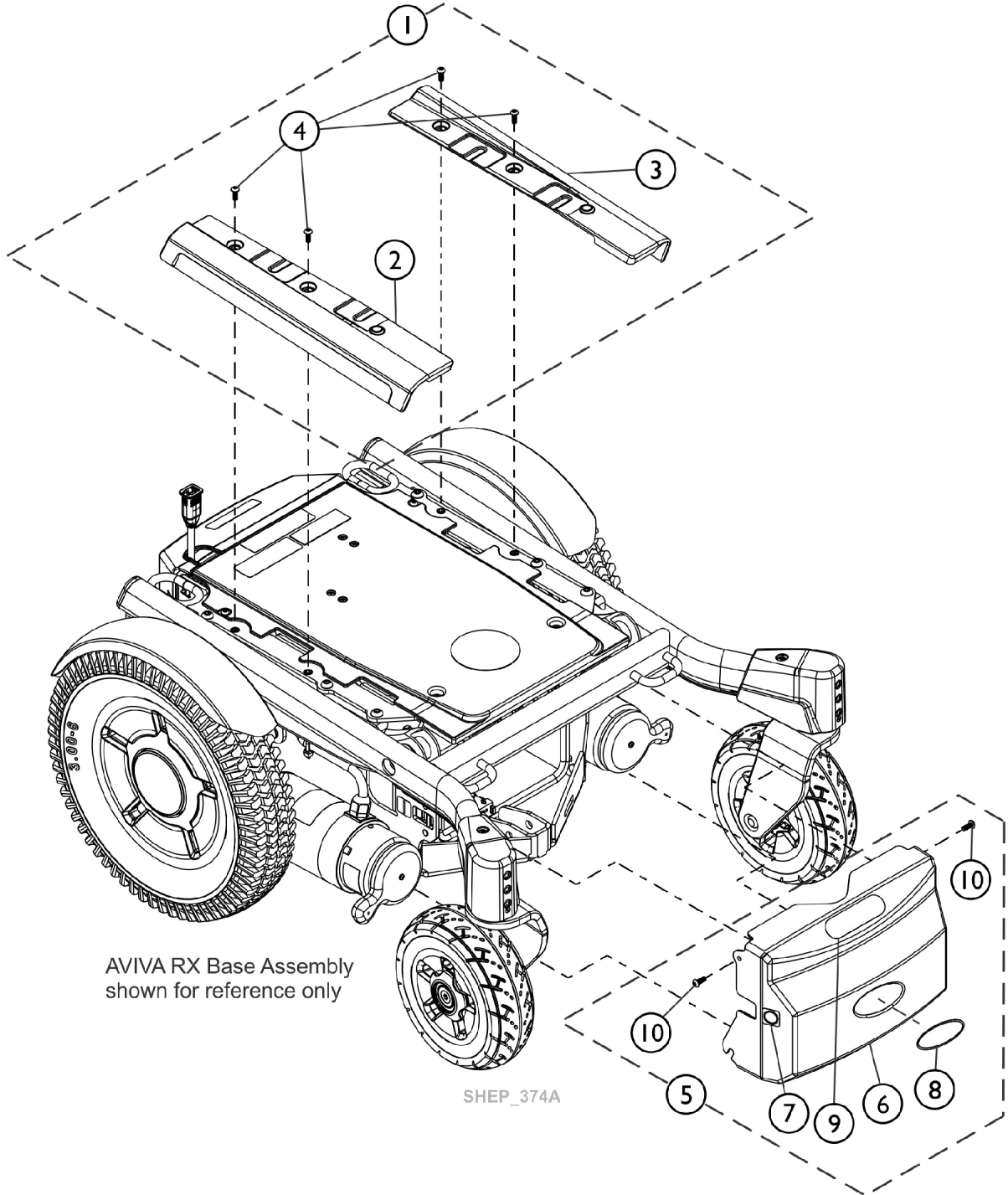

AVIVA RX Base Assembly
shown for reference only

SHEP_374A

Rail Shrouds and Front Shroud

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	See Chart	Item #1 Kit, Rail Shroud (Right and Left) w/ Hardware	1	1
1	A	See Chart	Item #1 Kit, Rail Shroud (Right and Left) w/ Hardware (continued)	1	1
5	B	60127924	Kit, Front Shroud w/ Hardware, Wide	1	1
6			Shroud, Front, Wide	1	1
7			Decal, Power Off	1	1
8			Decal, Invacare Medallion, Reflective - Large	1	1
9			Decal, Transportation	1	1
10	B	60128938	Screw, Socket Button Head Cap (M5 x 16)	1	2
			Sheet, Insert Parts Service	1	1
5			Kit, Front Shroud w/ Hardware, Narrow	1	1
6			Shroud, Front, Narrow	1	1
7			Decal, Power Off	1	1
8			Decal, Invacare Medallion, Reflective - Large	1	1
9			Decal, Transportation	1	1
10			Screw, Socket Button Head Cap (M5 x 16)	1	2
			Sheet, Insert Parts Service	1	1

NOTE: A - Includes items 2-4.
B - Includes items 6-10.

Item #1 Kit, Rail Shroud (Right and Left) w/ Hardware

Description	Obsidian Black (170P)	Ruby Red (171P)	Peacock Blue (172P)	Pink Flamingo (173P)
Kit, Rail Shroud (Right and Left) w/ Hardware	60127950	60127951	60127952	60127953

Item #1 Kit, Rail Shroud (Right and Left) w/ Hardware (continued)

Description	Purple Passion (174P)	Denim Navy (175P)	Lime Twist (176P)	Arctic Blue (177P)
Kit, Rail Shroud (Right and Left) w/ Hardware	60127954	60127955	60127956	60127957