


One Piece Wooden Footboard

1263

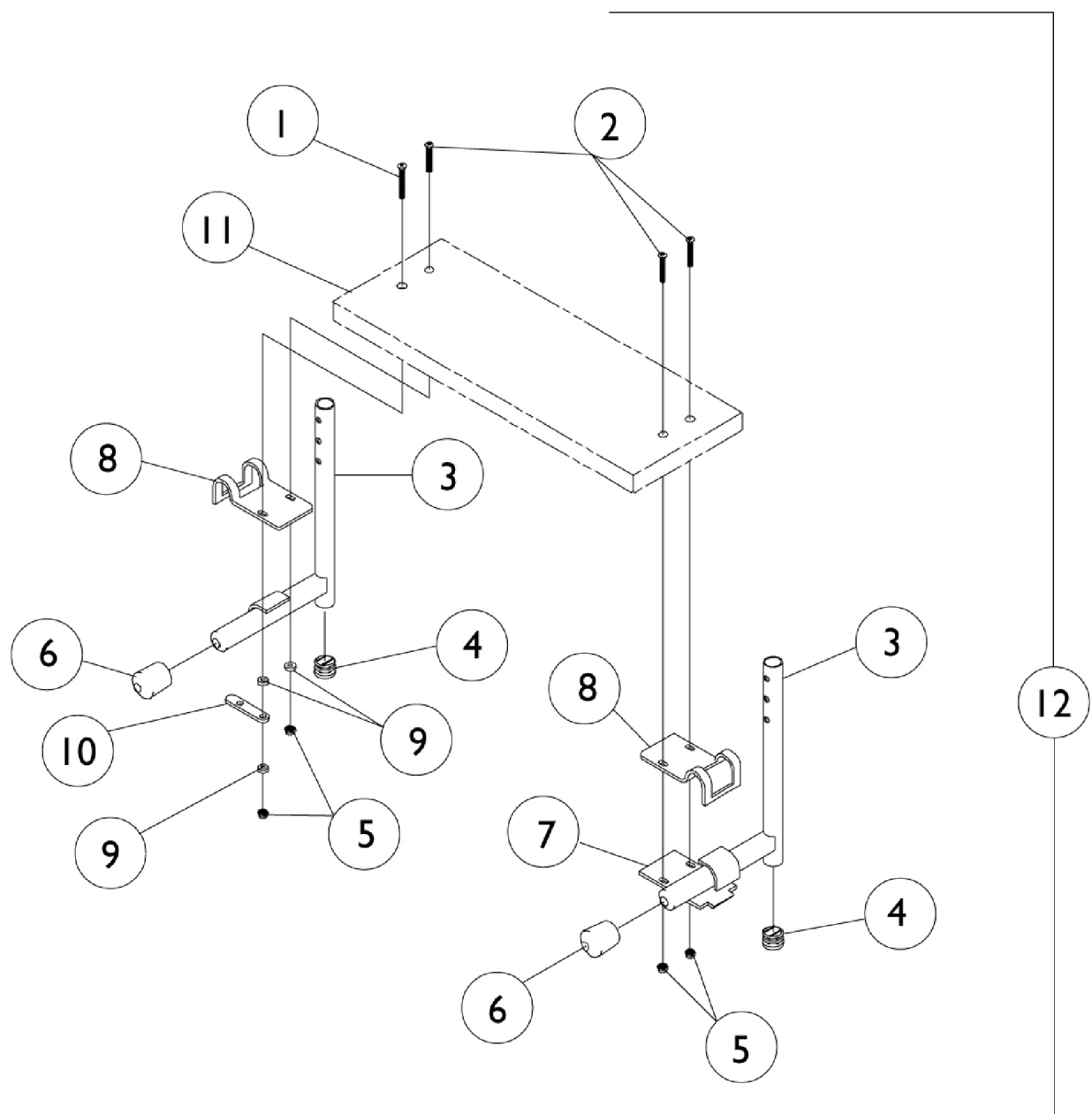

One Piece Wooden Footboard

1263

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		18860X005-NLA	NLA - Screw, Phillips Oval Head (#10-32 x 1-1/2")	1	1
2		11008X002-NLA	NLA - Screw, Phillips Oval Head (#10-32 x 1-1/4")	1	3
3		04470X050-NLA	NLA - Tube, Slide - Right	1	1
3		04470X051-NLA	NLA - Tube, Slide - Left	1	1
4		1027097	Plug Button, Black (3/4")	1	2
5		7930365-NLA	NLA - Locknut (#10-32)	25	4
6		1016346	Cap, End, Tube, Black (7/8")	1	2
7		47352X003	Lock, Pad Plate, Legrest, Black	1	1
8		47352X004	Clamp, Black	1	2
9		16347X000-NLA	NLA - Washer, Nylon (3/16 x 7/16 x 1/8")	1	3
10		47559X030-NLA	NLA - Link, Locking, Chrome	1	1
11	A	48247X002-NLA	NLA - Footboard w/o Hardware	1	1
11	B	48247X003-NLA	NLA - Footboard w/o Hardware	1	1
11	C	48247X001-NLA	NLA - Footboard w/o Hardware	1	1
12	A	1036915-NLA	NLA - One Piece Wooden Footboard	1	1
12	B	1037651-NLA	NLA - One Piece Wooden Footboard	1	1
12	C	1037650-NLA	NLA - One Piece Wooden Footboard	1	1
NOTE: A - 12" Wide Chairs B - 14" Wide Chairs C - 16" Wide Chairs					