


Footrest Support Assembly

91

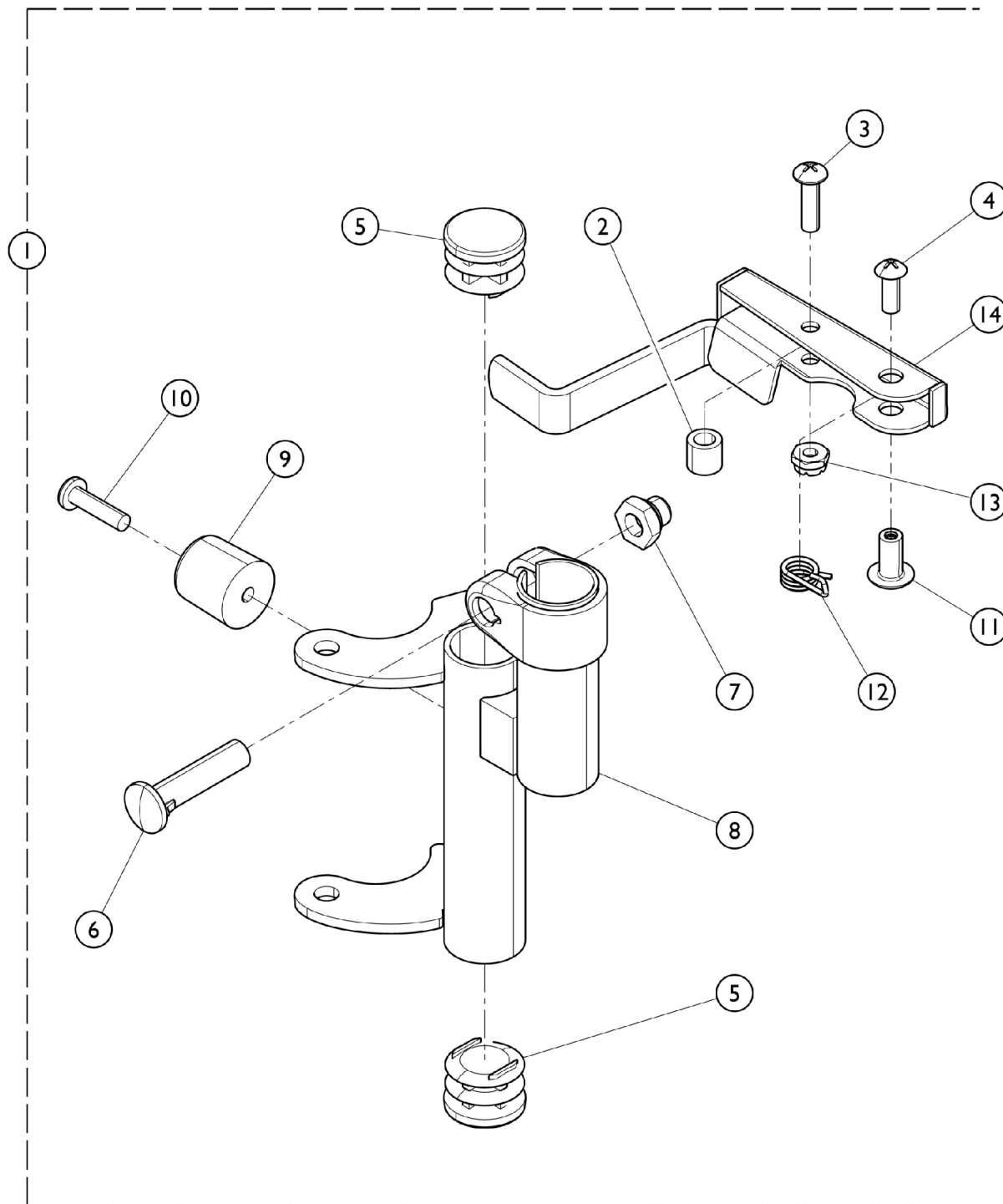

Footrest Support Assembly

91

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1017543-NLA	NLA - Footrest Support Assembly, Adult, Silver Vein - Right	1	1
1		1017544-NLA	NLA - Footrest Support Assembly, Adult, Silver Vein - Left	1	1
2		47565X057	Bushing (7/32 x 3/8 x 11/32")	1	1
3		7930360-NLA	NLA - Package, Screw, Phillips Pan Head (#10-32 x 11/16")	25	1
4		7930359-NLA	NLA - Package, Screw, Phillips Pan Head (#10-32 x 1/2")	25	1
5		1027095	Plug Button, Black (7/8")	1	2
6		11232X009	Bolt, Lug (5/16-24 x 1-3/8")	1	1
7		1107321	Nut, Low Cap (5/16"-24)	1	1
8		1016962-NLA	NLA - Tube, Footrest Support, Adult, Chrome - Right	1	1
8		1016963-NLA	NLA - Tube, Footrest Support, Adult, Chrome - Left	1	1
9		1022874	Bumper	1	1
10		1030979	Screw, Phillips Pan Head Tap (#10-32 x 3/4")	1	1
11		67005X000	Rivnut (#10-32)	1	1
12		1008935	Spring, Release - Right	1	1
12		1008936	Spring, Release - Left	1	1
13		7930365-NLA	NLA - Locknut (#10-32)	25	1
14		1035539	Handle, Release, Black - Right	1	1
14		1035540	Handle, Release, Black - Left	1	1
	A	See Chart	Pivot Slide Tube with Aluminum Footplate	1	1

NOTE: A - Not Shown.

The following pivot slide tubes with footplates are used with the above assembly. For more information on the pivot slide tube or footplates see page titled "Pivot Slide Tube with Footplate"
Due to design change, for all chairs built prior to 10/1/00 it is suggested that Release Handles be replaced in pairs.

Pivot Slide Tube with Aluminum Footplate

Chair Width	Right	Left
12" Width	1017535-NLA	1017536-NLA
14" Width	1018025-NLA	1018026-NLA
16" Width	1018023-NLA	1018024-NLA