


Battery Tray Assembly and Wire Harnesses For 22NF Post Terminals (Narrow Battery Box) After 03/03/20

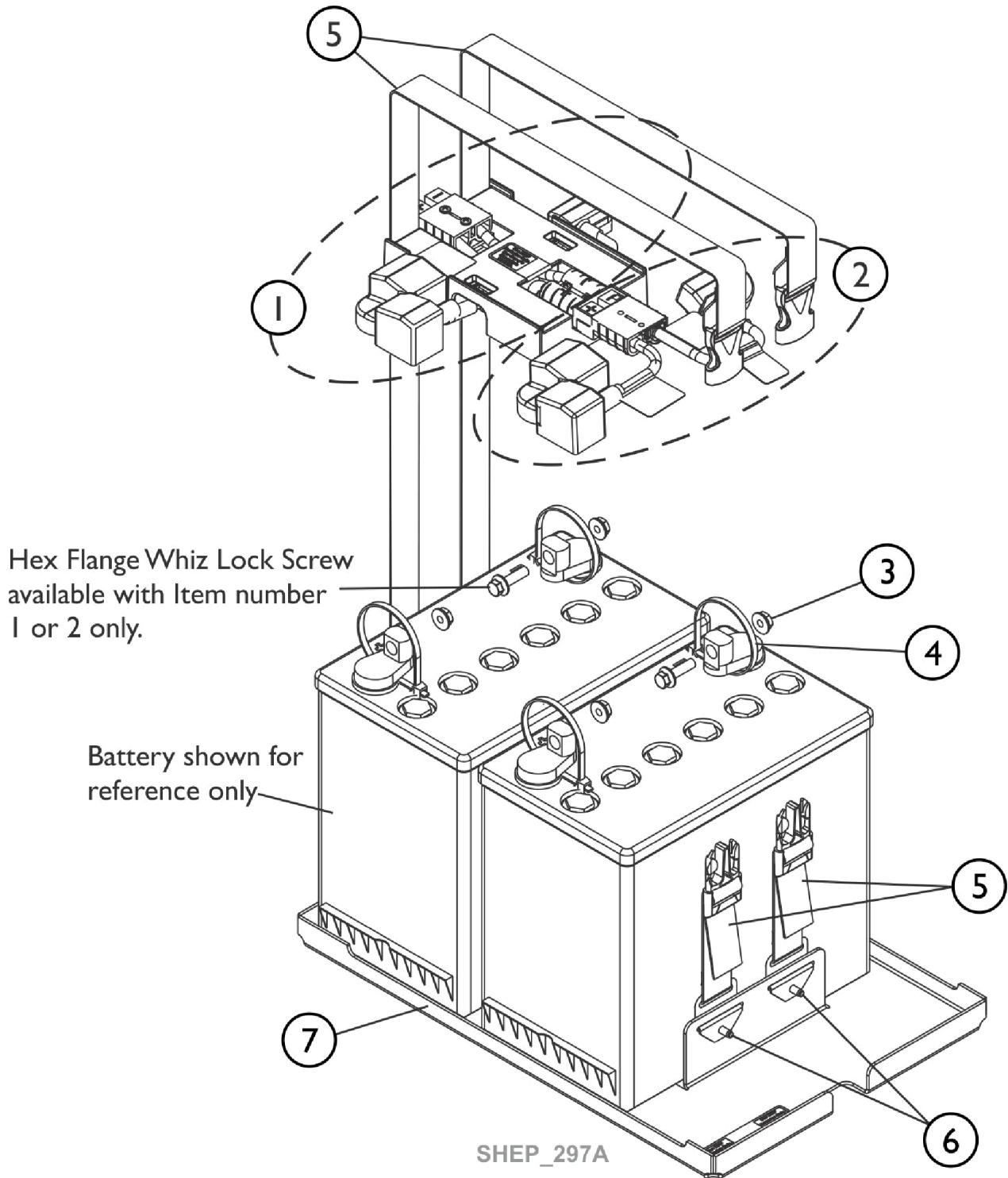

Battery Tray Assembly and Wire Harnesses

For 22NF Post Terminals (Narrow Battery Box) After 03/03/20

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A,B	60102800	Kit, Battery Wire Harness with Terminal Hardware, 22NF - Rear	1	1
2	A,B	60102799	Kit, Battery Wire Harness with Terminal Hardware, 22NF - Front	1	1
3		16721X000	Locknut (1/4-20)	1	4
4		98007X005	Tie Wrap (11-1/2" L)	1	4
5		1144682	Kit, Battery Strap Assembly, 22NF	1	2
6		16348X000-NLA	NLA - Rivet (3/16 x 7/16")	1	4
7	C	1157804	Kit, Battery Tray Assembly, 22NF	1	1
	D	1144611	Strip, Self Adhesive Wear (2-1/2' L x 1/2" W)	1	1

NOTE: A - Individual components are not available for the battery wire harnesses
 B - Includes items 3, 4 and 2 ea. Screw, Hex Flange Whiz Lock (1/4-20 x 7/8")
 C - Includes the Battery Tray and items 5 and 6
 D - Not Shown. The Self Adhesive Wear Strip is attached to the bottom side of the Battery Tray on both sides to allow removal from the Battery Box easier. One strip will accommodate both sides. Cut strip to desired length.

NOTE: ALL Power Wheelchairs with 5FAA or 10FAA Fixed Anterior Assist Option did not change and continue to use Before 03/04/20 components.