


**Backrest Bracket Mounting Hardware, Recline -
DLX-ACT400, DLX-IN500, DLX-OUT500, GLM-CONX4
(When ASL Options were ordered On-Chair)**

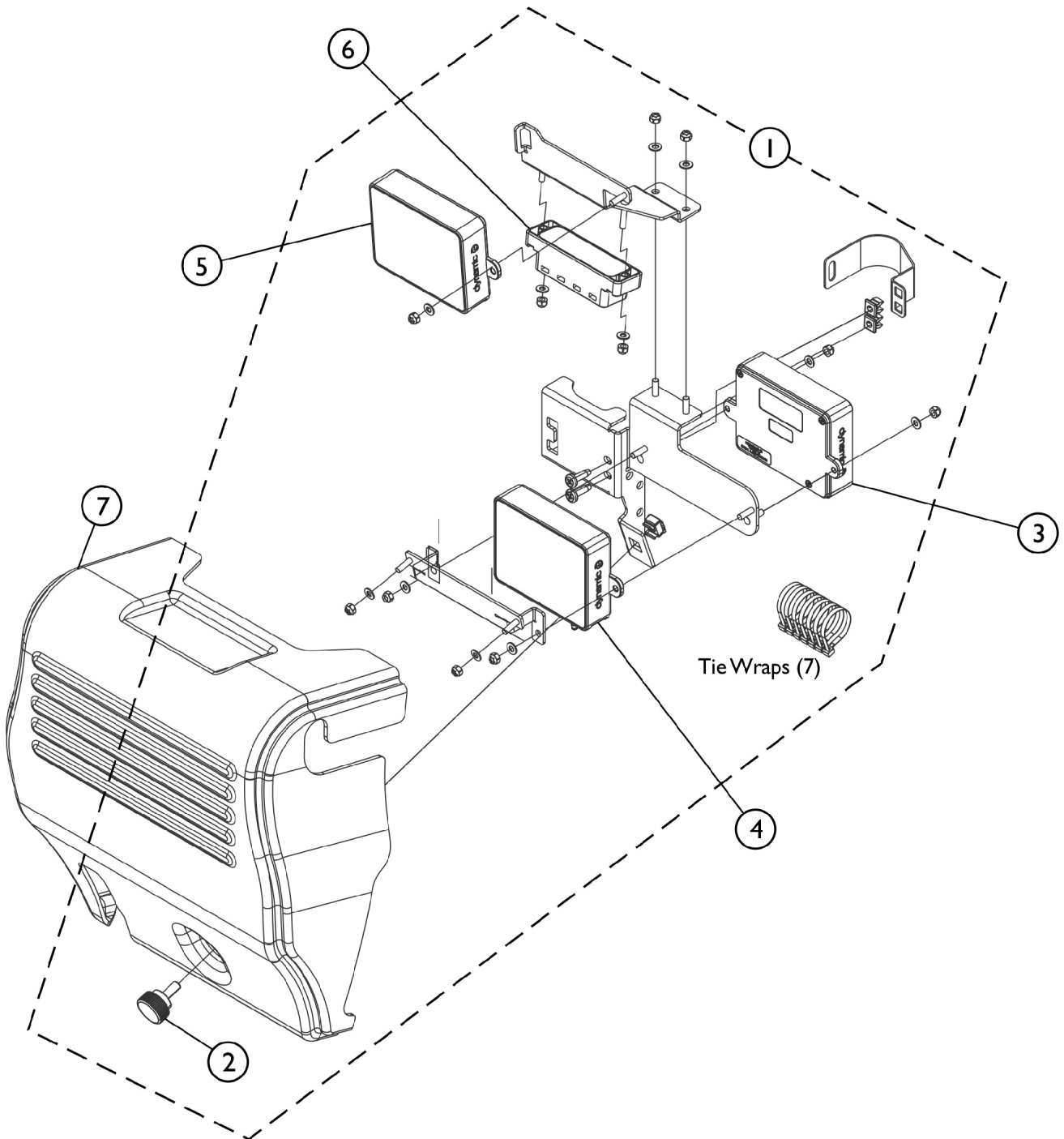

SHEP_346B

Backrest Bracket Mounting Hardware, Recline - DLX-ACT400, DLX-IN500, DLX-OUT500, GLM-CONX4 (When ASL Options were ordered On-Chair)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	60102541	Kit, Backrest Bracket Mounting Hardware, Recline for DLX-IN500, CONX4, DLX ACT400	1	1
2		60102721	Kit, Knob, Hand Wheel (M6 x 15)	1	1
3	D,E	60102474	Kit, LiNX DLX-IN500 Input Module (INPUT)	1	1
4	E	60102475	Kit, LiNX 4-Channel Actuator Module (DLX-ACT400)	1	1
5	C	60108452	Kit, LiNX DLX-OUT500 Output Module (OUT500)	1	1
6		60102473	Kit, LiNX CONX4 4-way Expansion Block	1	1
7		60102672	Kit, Shroud, Backrest Recline	1	1
	B	60102523	Kit, Sip-N-Puff, 16" Fixed (PKG32666) Includes: (3) Air Tube Straws, (2) Saliva Traps, Tubing, Pivot Shaft/Flexible Metal Gooseneck Assembly, and Extruded Figure "8" Clamp to fit 7/8" Wheelchair Frame Tubing. Extruded Back Cane Adapter Clamp.	1	1
NOTE: A - Includes all hardware shown including item #2 and Tie Wraps (7 ea.). B - Not Shown C - Item #5 includes LiNX DLX-OUT500 Output Module, 2 ea. Washer (M4 4.3 x 9 x .8), 2 ea. Locknut (M4), 1 ea. LiNX Bus Cable (PWH) 300mm 11-13/16" (GLM-BUS030) D - Item #4 LiNX DLX-IN500 Input Module does not include the LiNX Bus Cable (PWH) 300mm 11-13/16" (GLM-BUS030). E - Item #3 and #4 switched mounting locations after 01/23/22					