


Backrest Bracket Mounting Hardware, Tilt or ASBA-2- DLX-IN200, DLX-ACT400, DLX-IN500, DLX-OUT500, GLM-CONX4

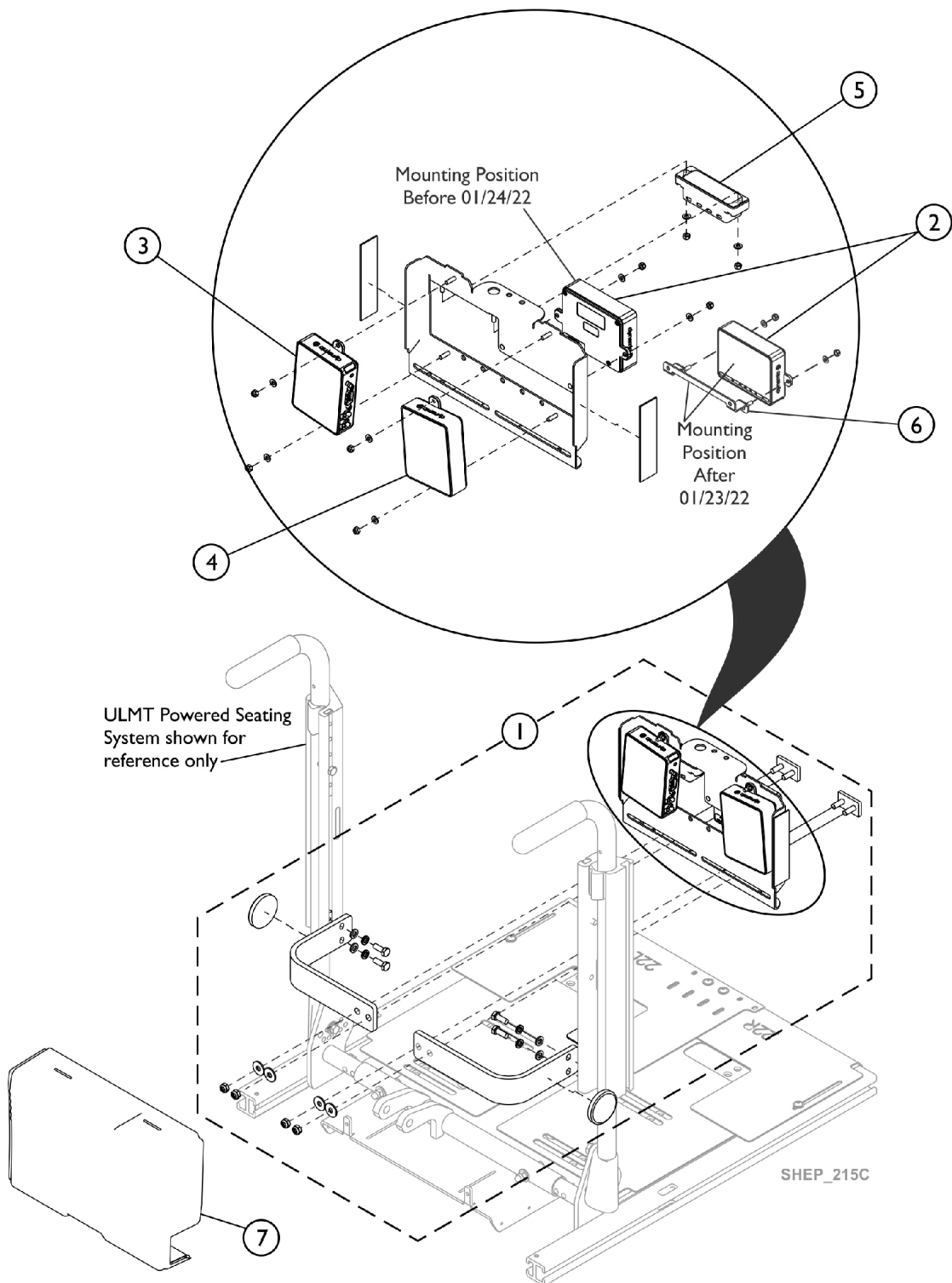

Backrest Bracket Mounting Hardware, Tilt or ASBA-2-DLX-IN200, DLX-ACT400, DLX-IN500, DLX-OUT500, GLM-CONX4

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	C	60102496	Kit, Backrest Bracket Mounting Hardware, Tilt and Sling Back- DLX-ACT400, DLX-IN500, GLM-CONX4	1	1
2	A	60102475	Kit, LiNX 4-Channel Actuator Module (DLX-ACT400)	1	1
2	A,F	60124980	Kit, LiNX DLX-IN200 Input Module (INPUT)	1	1
3	E	60102474	Kit, LiNX DLX-IN500 Input Module (INPUT)	1	1
4	D	60108452	Kit, LiNX DLX-OUT500 Output Module (OUT500)	1	1
5		60102473	Kit, LiNX CONX4 4-way Expansion Block	1	1
6	G	60132728	Kit, Bracket Assembly, LiNX, Black	1	1
7		60102472	Kit, Shroud Cover, Module	1	1
	B	60102523	Kit, Sip-N-Puff, 16" Fixed (PKG32666) Includes: (3) Air Tube Straws, (2) Saliva Traps, Tubing, Pivot Shaft/Flexible Metal Gooseneck Assembly, and Extruded Figure "8" Clamp to fit 7/8" Wheelchair Frame Tubing. Extruded Back Cane Adapter Clamp.	1	1
NOTE: A - IN200 or ACT400 are attached in the same location. B - Not Shown. C - Includes all hardware shown. D - Item #4 includes LiNX DLX-OUT500 Output Module, 2 ea. Washer (M4 4.3 x 9 x .8), 2 ea. Locknut (M4), 1 ea. LiNX Bus Cable (PWH) 300mm 11-13/16" (GLM-BUS030) E - Item #3 LiNX DLX-IN500 Input Module does not include the LiNX Bus Cable (PWH) 300mm 11-13/16" (GLM-BUS030). F - No SNP. Switch Inputs only. G - Item 6 used after 01/23/22					