


Ocean E-VIP Commode Before 10/01/21

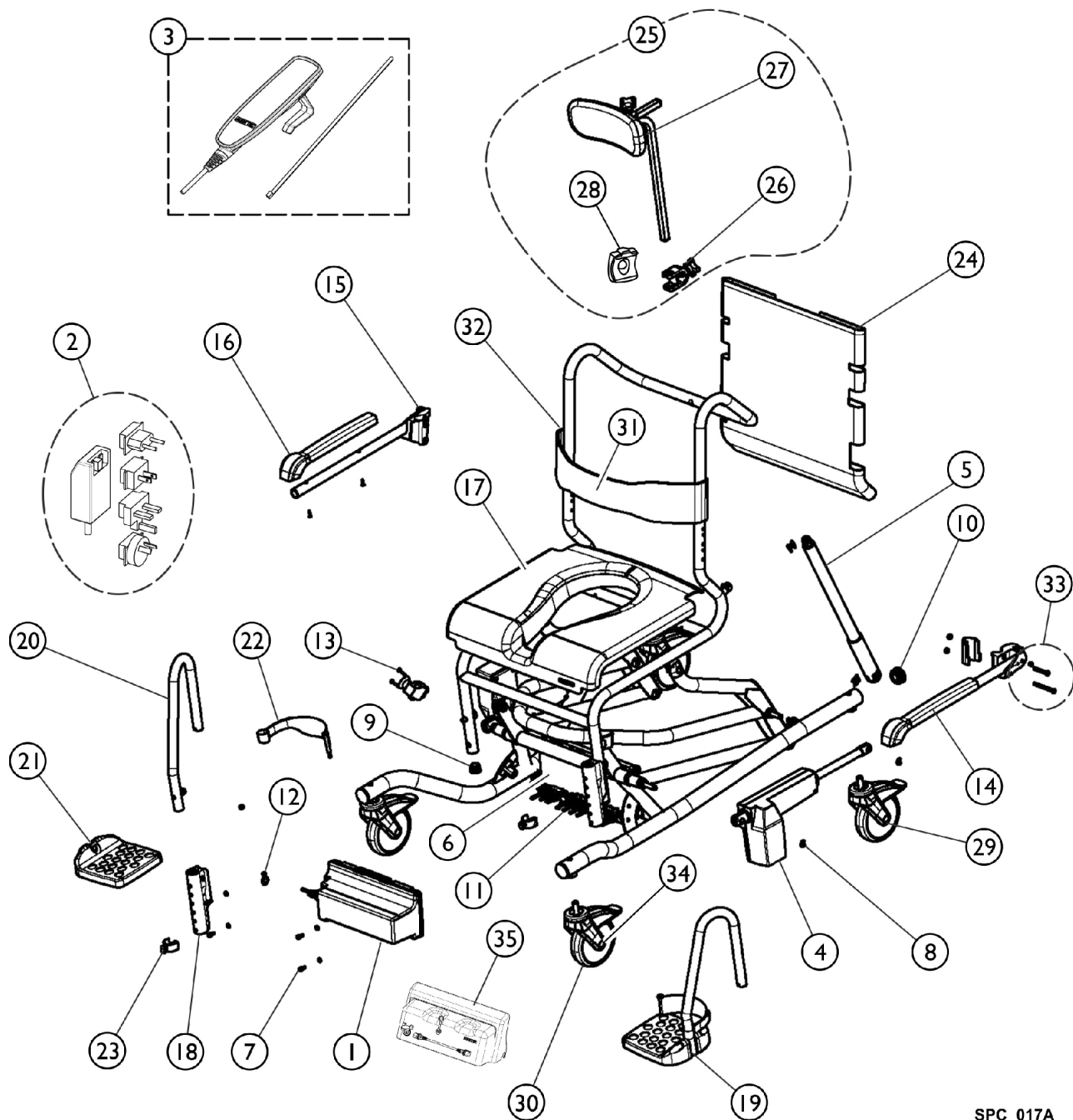

SPC_017A

Ocean E-VIP Commode Before 10/01/21

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1525920	Battery	--	1
2		1471153	Charger	--	1
3		1471154	Hand Control	--	1
4		1471155	Actuator, Height Adjustment	--	2
5		1471156	Actuator, Tilt Adjustment	--	1
6		1471157	Control Box	--	1
7		1471158	Screws, Control Box	--	4
8		1471159	Locking Rings, Actuators	--	8
9		1471160	Plug, Seating Tube	--	2
10		1471161	Plug, Side Tube	--	2
11		1471162	Cable Fixation, Control Box	--	1
12		1471207	Plug, Charging Socket	--	1
13		1471626	Emergency Release	--	1
14		1470694	Armrest Assembly Left	--	1
15		1470693	Armrest assembly, Right	--	1
16		16472	Armrest Pads Incl. Screws Kit	--	2
17		16262	Seat	--	1
18		1470929	Holder, Footrest, Ocean VIP	--	2
19		16432	Footrest, Complete - Left	--	1
20		16433	Footrest, Complete - Right	--	1
21		16260	Plates, Footrest	--	2
22		16334	Heel Loop (Pair)	--	1
23		16333	Clip, Footrest	--	2
24		1471143	Cover, Backrest	--	1
25		1470790	Headrest, Complete	--	1
26		1470971	Holder, Headrest	--	1
27		1470700	Headrest Assembly without Bracket	--	1
28		1514509	Clamp, Headrest	--	1
29		1470687	Caster, Directional Locking, Blue Tab	--	1
30		16259	Caster, Locking, Red Tab	--	3
31		1470081	Chest Belt Padded w/ Stainless Hardware	--	1
32		1522314	Bracket, Fixing, Chest Restraint	--	1
33		1471052	Ocean Armrest Kit Parts	--	1
34		1513809	Cover Cap for 5" Wheel	--	1
35		1525709	Battery Cover	--	1
		1522315	Bracket, Fixing, Pelvis Restraint	--	1
		1470082	Pelvis Belt Padded w/Stainless Hardware	--	1
		1536056	Peg, Heel Strap	--	1
		1536057	Screw, Phillips, Delta Point (K45 x 16mm)	--	1
		1470700	Headrest Assembly without Bracket	--	1