


Spare parts Ocean ErgoSP/(SPXL) Before 02/01/21

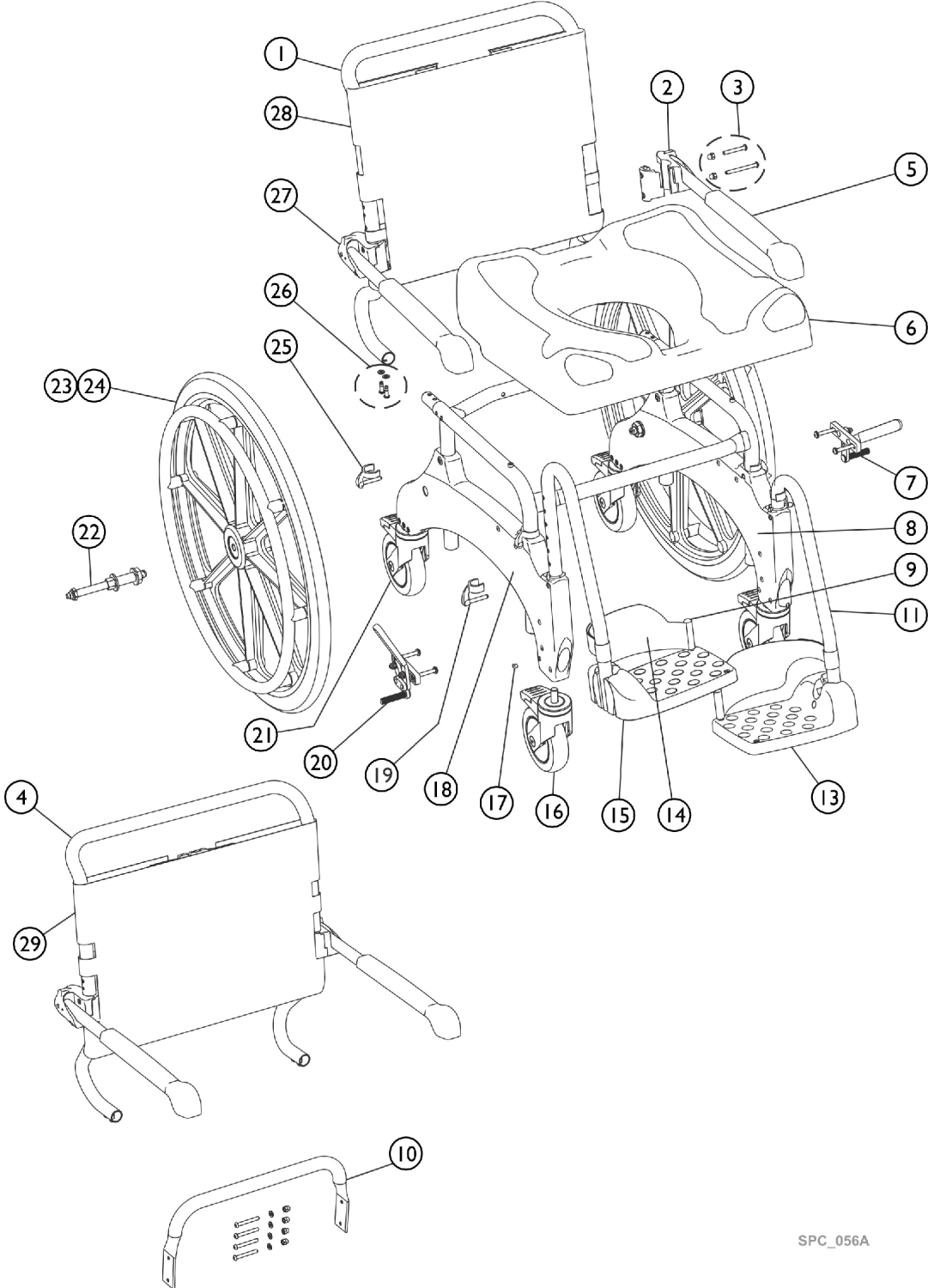

Spare parts Ocean ErgoSP/(SPXL) Before 02/01/21

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		SP1607166	Backrest w/o arm. / Hdwr. - Ocean (24") Ergo	1	1
2		1470694	Armrest Assembly Left	1	1
3		1471052	Ocean Armrest Kit Parts	1	2
4		SP1607167	Backrest XL w/o arm. / Hdwr. - Ocean (24") Ergo XL Armrest	1	1
5		16472	Armrest Pads Incl. Screws Kit	1	2
6		SP1607743	Seat plate - Ocean Ergo Series	1	1
7		16339	Lt. Brk. Lever - Ocean / Ergo Commodes	1	1
8		SP1607371	Side Frame w / 5" Casters - Ocean (24") Ergo (XL)	1	1
9		SP1607170	Hdwr., heel strap - Ocean (24") Ergo (XL)	1	2
10		SP1607168	Base stabilizer bar w / Hdwr. - Ocean (24") Ergo XL Footrest cpl.,	1	1
11		SP1643541	Footrest cpl., Left - Ocean (XL) / (Dual-)VIP / Ocean Ergo Series	1	1
13		1470689	Footrest Plate	1	2
14		16334	Heel Loop (Pair)	2	1
15		SP1643542	Footrest cpl., Right - Ocean (XL) / VIP / Ocean Ergo Series	1	1
16		SP1607628	Caster 5" - Ocean Ergo Series	1	4
17		SP1607169	Plugs, Side Frame - Ocean (24") Ergo (XL)	1	1
18		SP1607546	Side Frame w / 5" Dir. Caster - Ocean (24") Ergo (XL)	1	1
19		SP1607742	Clip, footrest - Ocean Ergo Series	1	2
20		16340	Rt. Brk. Lever Ocean / Ergo Commodes	1	1
21		SP1607741	Caster 5", dir. lock - Ocean Ergo Series	1	1
22		16418	Cpl Set Hub and QR Axle (16338 and 16337)	1	1
23		16336	24 In Wheel W / Handrim for Ocean/Ergo SP	1	2
24		16379	Self Propelling Wheels Kit complete including brakes	1	1
25		16256	Side Frame Adj. Clip	1	4
26		1542900	Bkrst Hdwr. - Ergo / Ocean (Dual-)VIP / XL	1	1
27		1470693	Armrest assembly, Right	1	1
28		AP1603031	Backrest Cover - Ocean Ergo/ SP	1	1
29		AP1603577	Backrest Cover - Ocean Ergo XL/ SPXL	1	1