


Spare parts Ocean ErgoVIP After 01/31/21

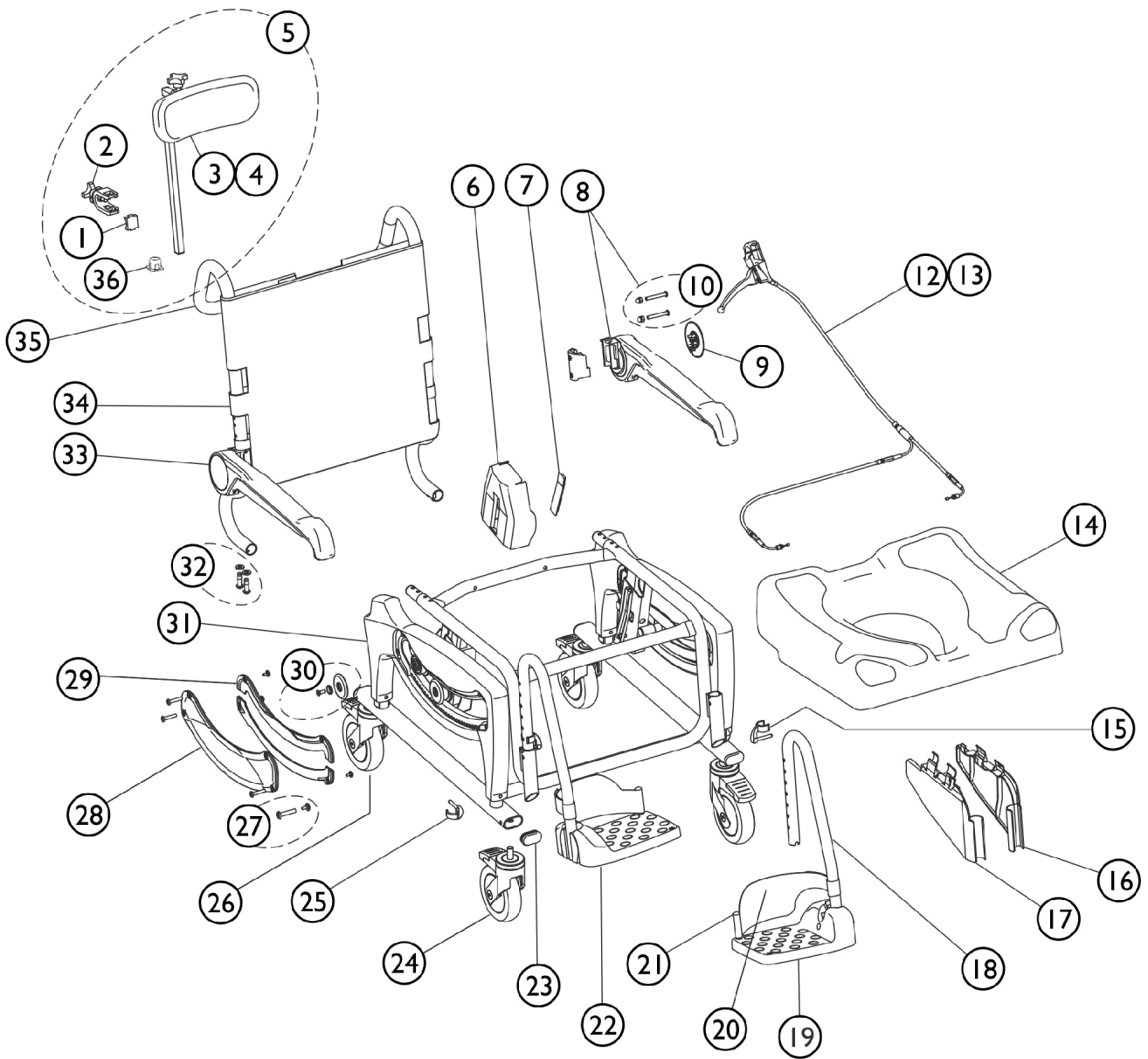

SPC_096A

Spare parts Ocean ErgoVIP After 01/31/21

| Item Number | Note | Part Number | Description | Quantity Per | |
|-------------|------|-------------|--|--------------|----------|
| | | | | Package | Assembly |
| 1 | | 1514509 | Clamp, Headrest | 1 | 1 |
| 2 | | 1470971 | Holder, Headrest | 1 | 1 |
| 3 | | 1470700 | Headrest Assembly without Bracket | 1 | 1 |
| 4 | | SP1607172 | Headrest Pad w / screws - Ocean (Dual-)VIP Ergo | 1 | 1 |
| 5 | | 1470790 | Headrest, Complete | 1 | 1 |
| 6 | | SP1607763 | Brake cover - Ocean (Dual-)VIP Ergo | 1 | 2 |
| 7 | | SP1607764 | Brake lever cover - Ocean (Dual-)VIP Ergo | 1 | 2 |
| 9 | | SP1643752 | Cover, Armrest 2.0 | 1 | 2 |
| 12 | | SP1607176 | Bowden cable w/o lever | 1 | 1 |
| 13 | | SP1607744 | Bowden cable w / lever | 1 | 1 |
| 14 | | SP1607743 | Seat plate - Ocean Ergo Series | 1 | 1 |
| 15 | | SP1607742 | Clip, footrest - Ocean Ergo Series | 1 | 2 |
| 16 | | SP1607003 | Clamping protection cover left - Ocean (Dual-)VIP Ergo | 1 | 1 |
| 17 | | SP1607807 | Clamping protection cover right - Ocean (Dual-)VIP Ergo | 1 | 1 |
| 18 | | SP1643541 | Footrest cpl., Left - Ocean (XL) / (Dual-)VIP / Ocean Ergo Series | 1 | 1 |
| 19 | | 1470689 | Footrest Plate | 1 | 2 |
| 20 | | 16334 | Heel Loop (Pair) | 2 | 1 |
| 21 | | SP1607170 | Hdwr., heel strap - Ocean (24") Ergo (XL) | 1 | 2 |
| 22 | | SP1643542 | Footrest cpl., Right - Ocean (XL) / VIP / Ocean Ergo Series | 1 | 1 |
| 23 | | SP1607765 | Frame Plug - Ocean (Dual-)VIP Ergo | 1 | 4 |
| 24 | | SP1607628 | Caster 5" - Ocean Ergo Series | 1 | 3 |
| 25 | | 16256 | Side Frame Adj. Clip | 1 | 4 |
| 26 | | SP1607741 | Caster 5", dir. lock - Ocean Ergo Series | 1 | 1 |
| 27 | | SP1607174 | Side Frame cover Hdwr. - Ocean (Dual-)VIP Ergo | 1 | 1 |
| 28 | | SP1607762 | Cover, Side Frame outside - Ocean (Dual-)VIP Ergo | 1 | 2 |
| 29 | | SP1607173 | Cover, Side Frame inside - Ocean (Dual-)VIP Ergo | 1 | 2 |
| 30 | | SP1607207 | Flanged wheels w / Hdwr. - Ocean (Dual-)VIP Ergo | 1 | 2 |
| 31 | | SP1607761 | Side Frame - Ocean (Dual-)VIP Ergo | 1 | 2 |
| 32 | | 1542900 | Bkrst Hdwr. - Ergo / Ocean (Dual-)VIP / XL | 1 | 1 |
| 34 | | AP1603027 | Backrest cover - OceanVIP Ergo | 1 | 1 |
| 35 | | AP1606985 | Backrest tube only -Ocean VIP Ergo (separate order needed for armrest, backrest cover, headrest) | 1 | 1 |