


Spare parts Ocean ErgoVIP Before 02/01/21

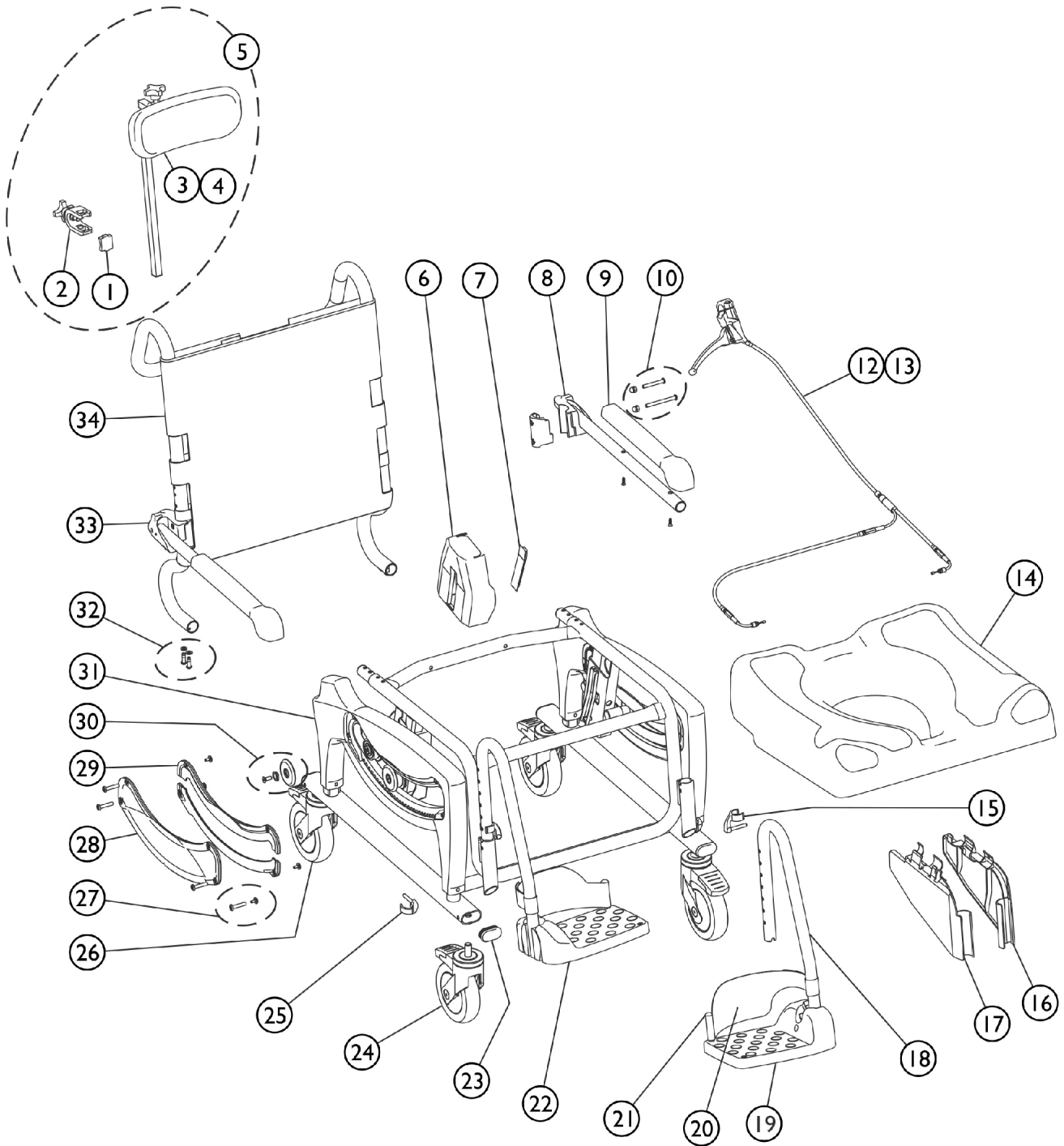

Spare parts Ocean ErgoVIP Before 02/01/21

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1514509	Clamp, Headrest	1	1
2		1470971	Holder, Headrest	1	1
3		1470700	Headrest Assembly without Bracket	1	1
4		SP1607172	Headrest Pad w / screws - Ocean (Dual-)VIP Ergo	1	1
5		1470790	Headrest, Complete	1	1
6		SP1607763	Brake cover - Ocean (Dual-)VIP Ergo	1	2
7		SP1607764	Brake lever cover - Ocean (Dual-)VIP Ergo	1	2
8		1470694	Armrest Assembly Left	1	1
9		16472	Armrest Pads Incl. Screws Kit	1	2
10		1471052	Ocean Armrest Kit Parts	1	2
12		SP1607176	Bowden cable w/o lever	1	1
13		SP1607744	Bowden cable w / lever	1	1
14		SP1607743	Seat plate - Ocean Ergo Series	1	1
15		SP1607742	Clip, footrest - Ocean Ergo Series	1	2
16		SP1607003	Clamping protection cover left - Ocean (Dual-)VIP Ergo	1	1
17		SP1607807	Clamping protection cover right - Ocean (Dual-)VIP Ergo	1	1
18		SP1643541	Footrest cpl., Left - Ocean (XL) / (Dual-)VIP / Ocean Ergo Series	1	1
19		1470689	Footrest Plate	1	2
20		16334	Heel Loop (Pair)	2	1
21		SP1607170	Hdwr., heel strap - Ocean (24") Ergo (XL)	1	2
22		SP1643542	Footrest cpl., Right - Ocean (XL) / VIP / Ocean Ergo Series	1	1
23		SP1607765	Frame Plug - Ocean (Dual-)VIP Ergo	1	4
24		SP1607628	Caster 5" - Ocean Ergo Series	1	3
25		16256	Side Frame Adj. Clip	1	4
26		SP1607741	Caster 5", dir. lock - Ocean Ergo Series	1	1
27		SP1607174	Side Frame cover Hdwr. - Ocean (Dual-)VIP Ergo	1	1
28		SP1607762	Cover, Side Frame outside - Ocean (Dual-)VIP Ergo	1	2
29		SP1607173	Cover, Side Frame inside - Ocean (Dual-)VIP Ergo	1	2
30		SP1607207	Flanged wheels w / Hdwr. - Ocean (Dual-)VIP Ergo	1	2
31		SP1607761	Side Frame - Ocean (Dual-)VIP Ergo	1	2
33		1542900	Bkrst Hdwr. - Ergo / Ocean (Dual-)VIP / XL	1	1
33		1470693	Armrest assembly, Right	1	1
34		AP1603027	Backrest cover - OceanVIP Ergo	1	1