


Footrest Support Assembly

93J

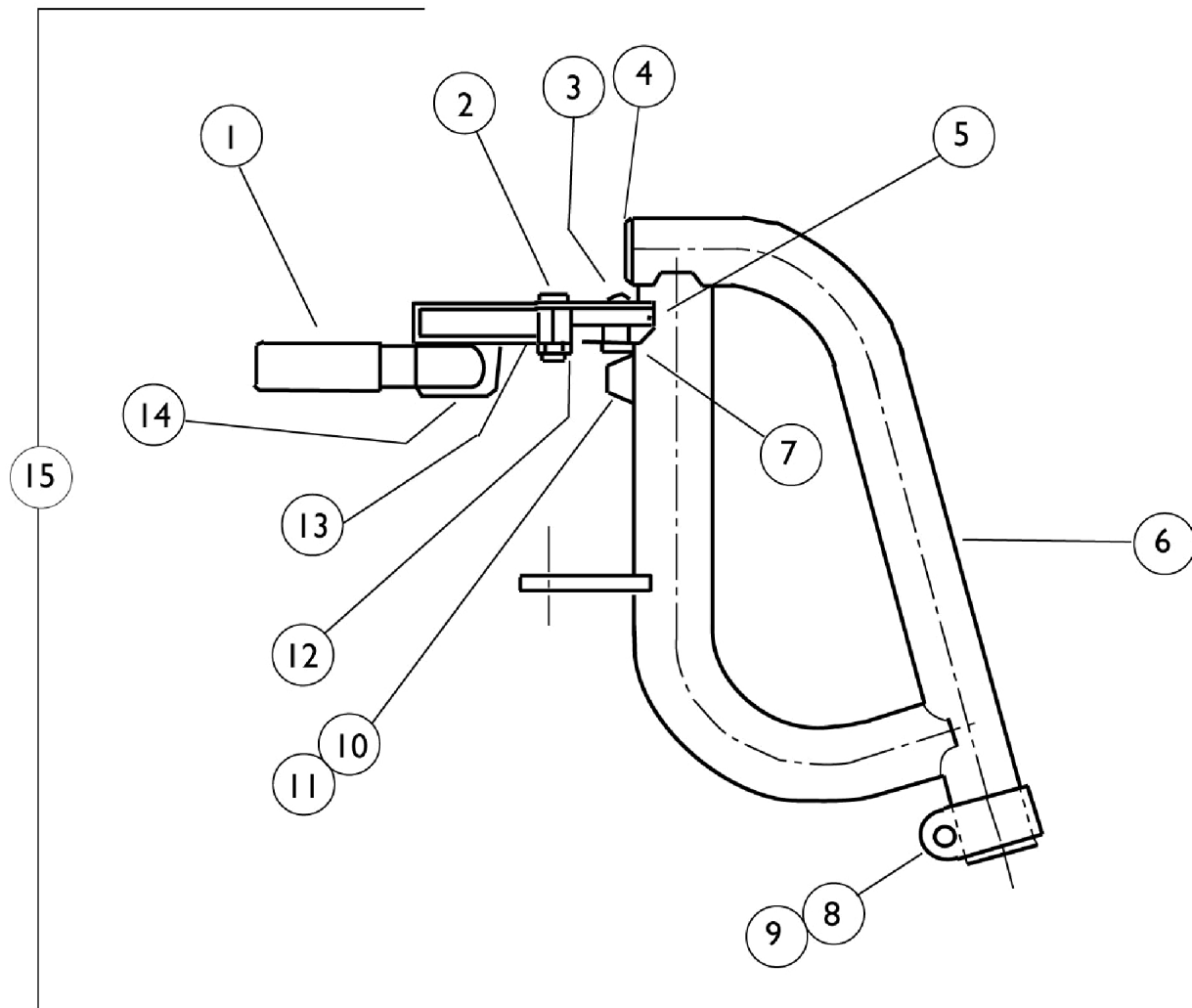

Footrest Support Assembly 93J

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1006490	Tip, Plastic, Black	1	1
2		7930360-NLA	NLA - Package, Screw, Phillips Pan Head (#10-32 x 11/16")	25	1
3		7930359-NLA	NLA - Package, Screw, Phillips Pan Head (#10-32 x 1/2")	25	1
4		1027095	Plug Button, Black (7/8")	1	1
5		1008935	Spring, Release - Right	1	1
5		1008936	Spring, Release - Left	1	1
6		04706X039-NLA	NLA - Tube, Footrest Support, Adult, Chrome - Right	1	1
6		04706X040-NLA	NLA - Tube, Footrest Support, Adult, Chrome - Left	1	1
7		67005X000	Rivnut (#10-32)	1	1
8		7930363-NLA	NLA - Package, Locknut (5/16"-24)	25	1
9		11232X009	Bolt, Lug (5/16-24 x 1-3/8")	1	1
10		1022874	Bumper	1	1
11		1030979	Screw, Phillips Pan Head Tap (#10-32 x 3/4")	1	1
12		7930365-NLA	NLA - Locknut (#10-32)	25	1
13		47565X057	Bushing (7/32 x 3/8 x 11/32")	1	1
14		1035539	Handle, Release, Black - Right	1	1
14		1035540	Handle, Release, Black - Left	1	1
15		1031330-NLA	NLA - Footrest Support w/ Release Handle (Tip, Bolt and Locknut) Right	1	1
15		1031331-NLA	NLA - Footrest Support w/ Release Handle (Tip, Bolt and Locknut) Left	1	1
	B	See Chart	Pivot Slide Tube with Aluminum Footplate	1	1

NOTE: A - Before 10/1/00
B - Not Shown

The following pivot slide tubes with footplates are used with the above assembly. For more information on the pivot slide tube or footplates see page titled "Pivot Slide Tube with Footplate"

Due to design change, for all chairs built prior to 10/1/00 it is suggested that Release Handles be replaced in pairs.

Pivot Slide Tube with Aluminum Footplate

Chair Width	Standard 9-1/2"	2" Longer 11-1/2"	4" Longer 13-1/2"
14"	1020232-NLA See Note(s) A	N/A	N/A
14"	1020233-NLA See Note(s) B	N/A	N/A
16"	1017243-NLA See Note(s) A	1086486-NLA See Note(s) A	1086487-NLA See Note(s) A
16"	1017244-NLA See Note(s) B	1086488-NLA See Note(s) B	1086489-NLA See Note(s) B

NOTE: A - Right
B - Left