


Head Support

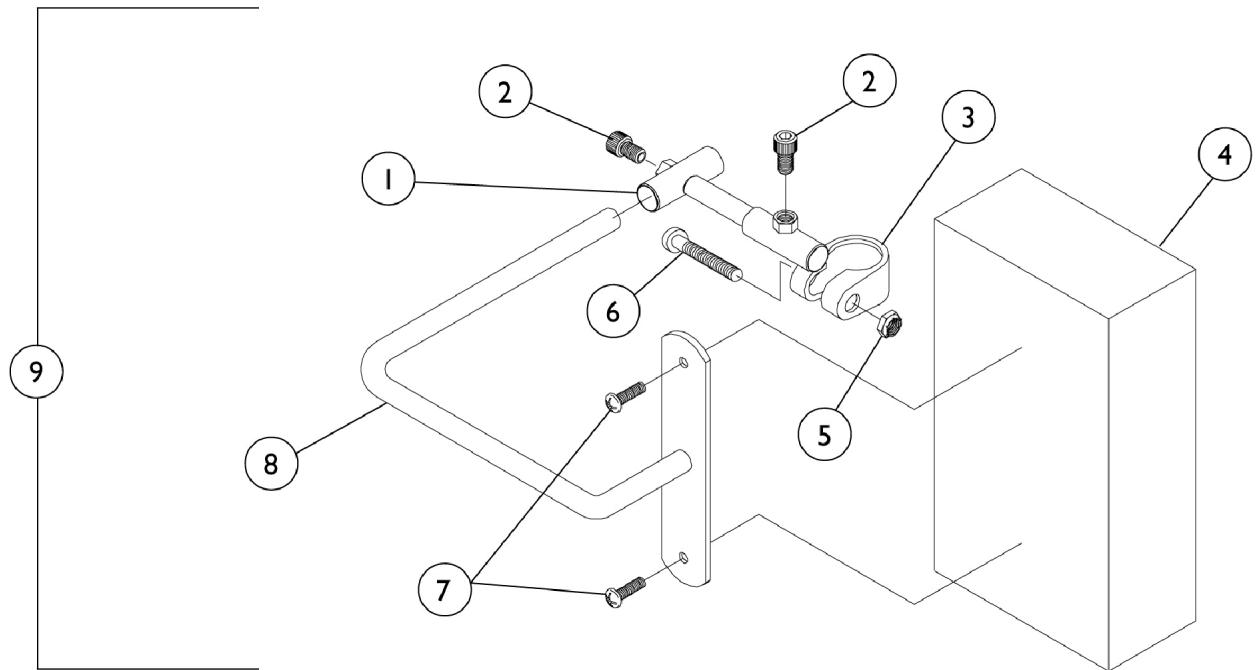

Head Support

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		03018X042	Wing Bracket Assembly, Chrome	1	1
2		60284X001	Screw, Socket Head (1/4-20 x 1/2")	1	1
3		03018X043	Bracket, Clamp, Black- Right	1	1
3		03018X046	Bracket, Clamp, Black - Left	1	1
4	A	88804529X040-NLA	NLA - Pad, Headrest	1	1
5		7930363-NLA	NLA - Package, Locknut (5/16"-24)	25	1
6		11232X009	Bolt, Lug (5/16-24 x 1-3/8")	1	1
7		7930360-NLA	NLA - Package, Screw, Phillips Pan Head (#10-32 x 11/16")	25	1
8	D	03018X045	Bracket, Pad Support, Chrome	1	1
8	C	03018X044-NLA	NLA - Bracket, Pad Support, Black	1	1
9	A,D	8881050191-NL A	NLA - Head Support Assembly (Pair)	1	1
9	A,C	8881050192-NL A	NLA - Head Support Assembly (Pair)	1	1
	B,E	1023610-NLA	NLA - Instruction Sheet	1	1

NOTE: A - Specify upholstery color, see page titled "Packaging and Upholstery Colors"
 B - Not Shown
 C - Recliner
 D - Non-Recliner
 E - Item is no longer available from the Invacare Service Parts Department. Refer to the Invacare Website under "Product Literature Search". Enter the part number in the "Keywords" field and download a free PDF version copy.