


Crossbraces and Hardware

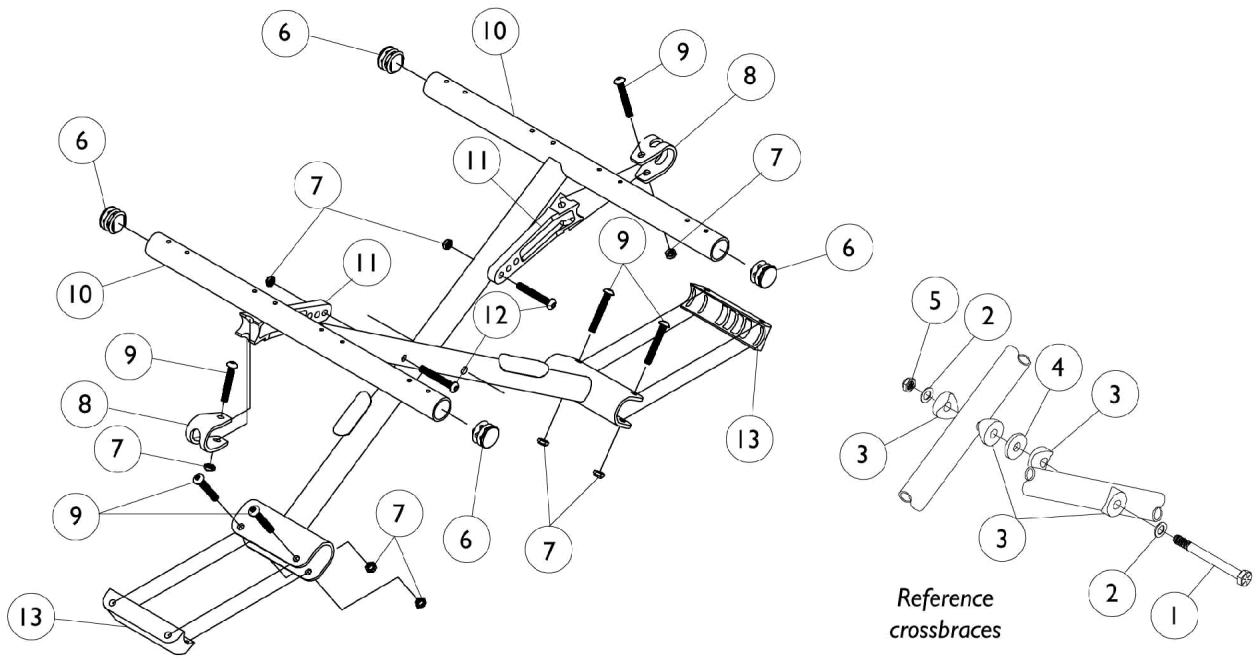

Crossbraces and Hardware

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1016261	Screw, Hex Head (5/16-24 x 3-1/4")	1	1
2		1016652	Washer (5/16 x 9/16 x 1/32")	1	2
3		1016263	Spacer, Coved (3/4 x 1 x 1/2")	1	4
4		7930313	Package, Washer, Nylon (5/16 x 7/8 x 1/8")	25	1
5		7930363-NLA	NLA - Package, Locknut (5/16"-24)	25	1
6		1029501	Plug Button, Black (1")	1	4
7		7930364-NLA	NLA - Package, Locknut (1/4-20)	25	8
8		1012180	Bracket, Pivot, Black	1	2
9		1008495	Screw, Socket Button Head (1/4-20 x 1-1/2")	1	6
10	D	See Chart	#10 Crossbraces without Hardware	--	--
11	A	1046501	Link, Pivot, Black (14 - 15" W)	1	2
11	B	1012173	Link, Pivot, Black (16 - 18" W)	1	2
11	C	1018238	Link, Pivot, Black (20 - 22" W)	1	2
12		7930301-NLA	NLA - Package, Button Head Cap Screw (1/4-20 x 1-3/4")	25	2
13		1012172-NLA	NLA - Clamp, Saddle	1	2
	D,E	1046925	Hardware, Crossbrace (14 - 15" W)	2	1
	D,E	1034077	Hardware, Crossbrace (16 - 18" W)	2	1
	D,E	1034078	Hardware, Crossbrace (20 - 22" W)	2	1

NOTE: A - 14" Wide Chairs
 B - 16" Wide Chairs
 C - 20" Wide Chairs
 D - Not Shown
 E - Includes Items 1-12

#10 Crossbraces without Hardware

(Individual Crossbraces Fit Both Right and Left Sides)

Depth	14" Width	16" Width	17" Width	18" Width	20" Width
16" Depth	1046107-NLA	1016250	1037547-NLA	1016251	1038381-NLA
18" Depth	--	1053344-NLA	1053345-NLA	1053346	1053347