


# Fork and Hardware

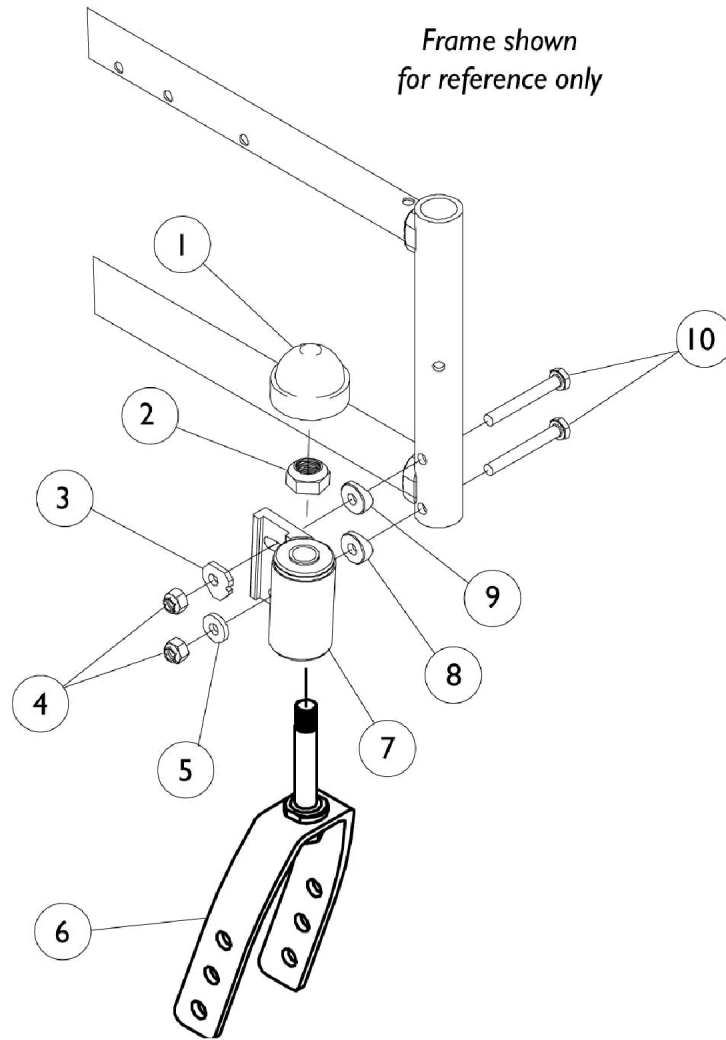

## Fork and Hardware

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1001843	Cap, Head Tube, Black	1	1
1		1131189	Package, Head Tube Cap	10	1
2		1001979	Locknut (1/2-20)	1	1
3		1080318-NLA	NLA - Cam, Headtube Adjustable	1	1
4		1065362	Locknut (1/4-28)	1	2
5		1018739	Washer (1/4 x 5/8 x 3/32")	1	1
6		1037420	Fork and Stem Assembly, Black (6 x 1" Caster)	1	1
6		1056640	Fork and Stem Assembly, Black (8" & 6 x 2" Caster)	1	1
6		1153529	Fork and Stem Assembly, Short, Black (2 Hole)	1	1
6		1157707	Fork and Stem Assembly, Short, Black (3 Hole)	1	1
7		1080315	Headtube, Caster w/ Flanged Bearings, Black - Right	1	1
7		1080316	Headtube, Caster w/ Flanged Bearings, Black - Left	1	1
8		1009861	Spacer, Coved (1/4 x 11/16 x 1/2")	1	1
9		1026099	Spacer, Coved (1/4 x 11/16 x 1/2")	1	1
10		1055349	Screw, Hex Head (1/4-28 x 2")	1	2