


# Elevating Actuator, Sensors and Seat Support Tray For Elevating Seat

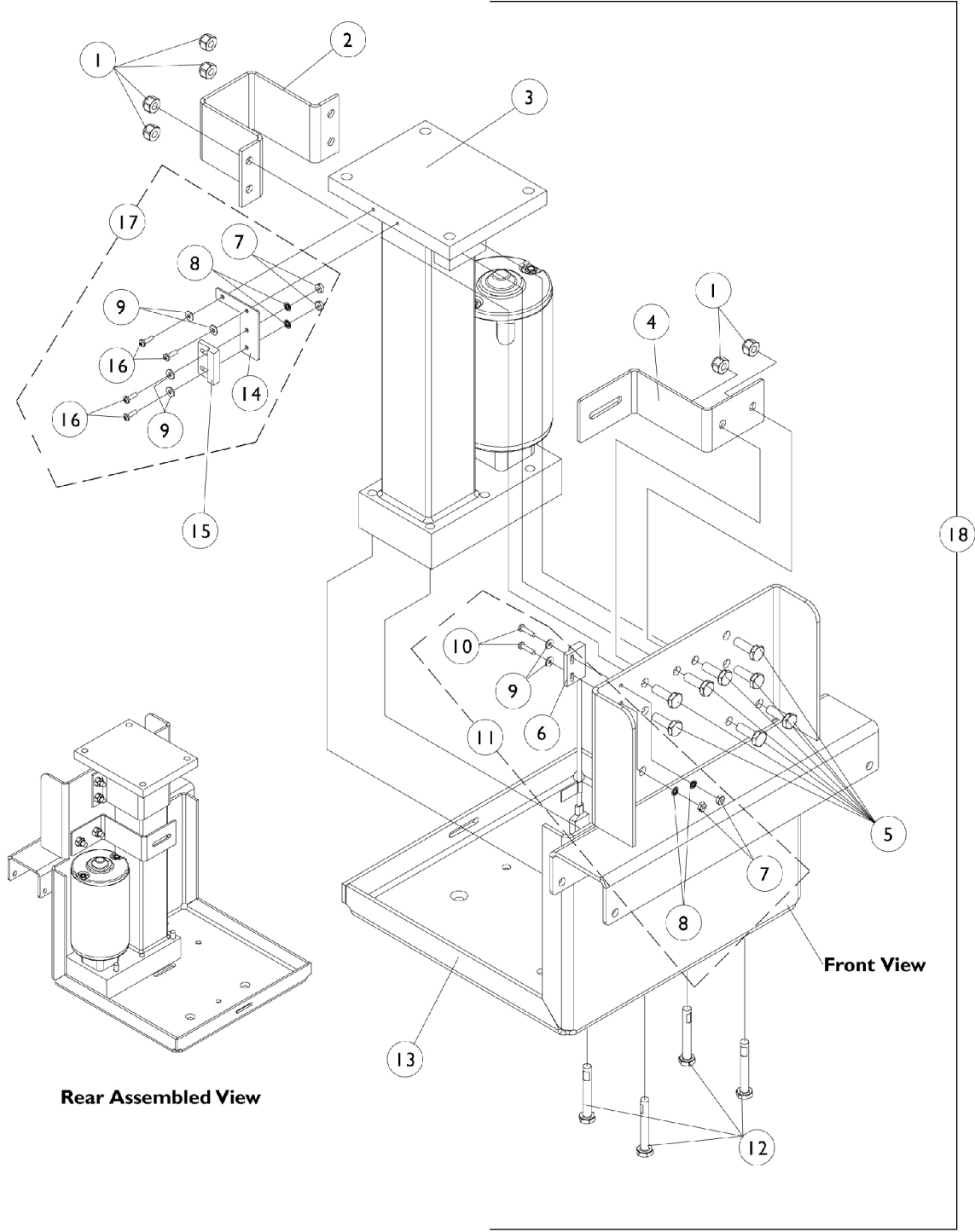

## Elevating Actuator, Sensors and Seat Support Tray For Elevating Seat

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		7930323-NLA	NLA - Package, Locknut (1/4-20)	25	6
2		1108436-NLA	NLA - Bracket, Actuator Support, Black	1	1
3		1109334-NLA	NLA - Actuator Motor Assembly, Elevate	1	1
4		1108435-NLA	NLA - Bracket, Battery Strap, Black	1	1
5		7930326-NLA	NLA - Package, Screw, Hex Head (1/4-20 x 3/4")	25	8
6		1105715-NLA	NLA - Sensor, Proximity Switch	1	1
7		1005889	NLA - Locknut (#4-40)	1	4
8		1008525-NLA	NLA - Washer, Internal Tooth Lock (#4-40)	1	4
9		1011011-NLA	NLA - Washer, Plain (1/8 x 5/16 x 1/32")	1	6
10		7930318-NLA	NLA - Package, Screw, Phillips Pan Head (#4-40 x 1/2")	25	2
11	A	1108774-NLA	NLA - Hardware, Proximity Switch Sensor and Mounting	1	1
12		1088898-NLA	NLA - Screw, Hex Head w/ Patch (1/4-20 x 1-3/4")	1	4
13		1106520-NLA	NLA - Tray, Seat Support	1	1
14		1106518-NLA	NLA - Bracket, Actuator Switch Sensor, Black	1	1
15		1024001	Sensor, Actuator Switch	1	1
16		1027704-NLA	NLA - Screw, Phillips Pan Head (#4-40 x 3/8")	1	4
17	B	1108773-NLA	NLA - Hardware, Actuator Switch Sensor and Mounting	1	1
18		1108768-NLA	NLA - Elevating Actuator, Sensors and Seat Support Tray Assembly	1	1

NOTE: A - Includes items 6-10  
B - Includes items 7-9, 14 and 16