


Rear Wheel Hardware

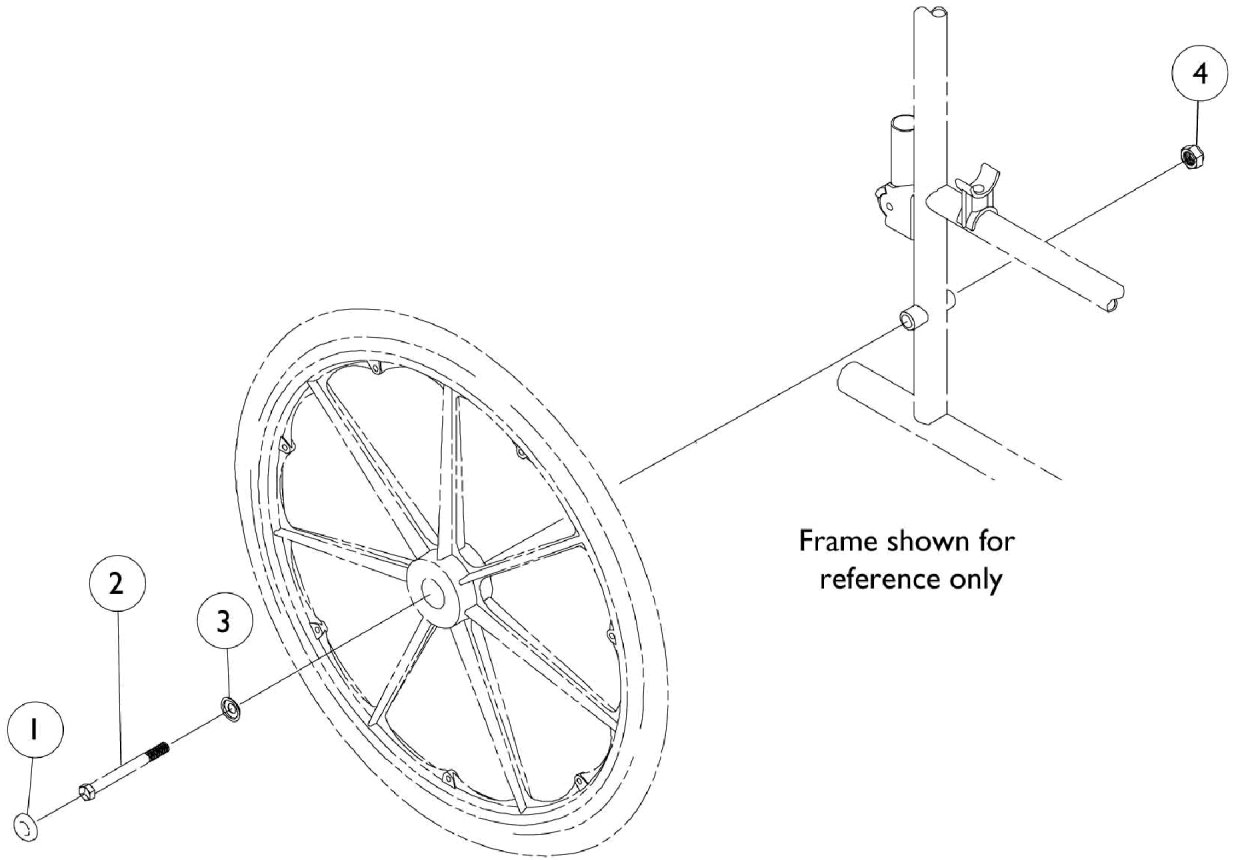

Rear Wheel Hardware

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A,B	1027600-NLA	NLA - Cap, Black	1	1
1	C	1016291	Cap, Hub, Black	1	1
2	D	47203X029	Screw, Hex Head (7/16-20 x 4-1/2")	1	1
2	D	1015436	Axle, Quick Release (7/16" x 4-1/2")	1	1
3	C	1016289	Washer, Caplugs (15/32" I.D. x 1-3/32" O.D.)	1	1
4		1045608	Locknut (7/16-20)	1	1
	E,G	1047943	Kit, Rear Wheel Hub Hardware	1	1
			Cap, Hub, Black	1	1
			Washer, Caplugs (15/32" I.D. x 1-3/32" O.D.)	1	1
	F,G	1034106	Kit, Rear Wheel Hardware	1	1
			Screw, Hex Head (7/16-20 x 4-1/2")	1	1
			Locknut (7/16-20)	1	1
	F,G	1030936	Plug, Valve Stem	1	1
	F,G,H	1034109	NLA - Kit, Rear Wheel Hardware, Conventional	1	1
			Bolt, Hex Head Cap (7/16-20 x 5-1/8")	1	1
			Locknut (7/16-20)	1	1
			Spacer (7/16 x 5/8 x 1/2")	1	1
		1085454	Bolt, Hex Head Cap (7/16-20 x 5-1/8")	1	1
		1045608	Locknut (7/16-20)	1	1
		1022671	Spacer (7/16 x 5/8 x 1/2")	1	1

NOTE: A - U2222 Wheel Only
 B - Before 11/20/98
 C - U2222 only After 11/19/98
 D - Space Saver Arms
 E - Includes items 1 and 3
 F - Includes items 2 and 4
 G - Not Shown.
 H - Components contained in Kit Hardware 1034109 must now be ordered separately. Order part# 1085454, 1045608, 1022671.