


Wheel Locks

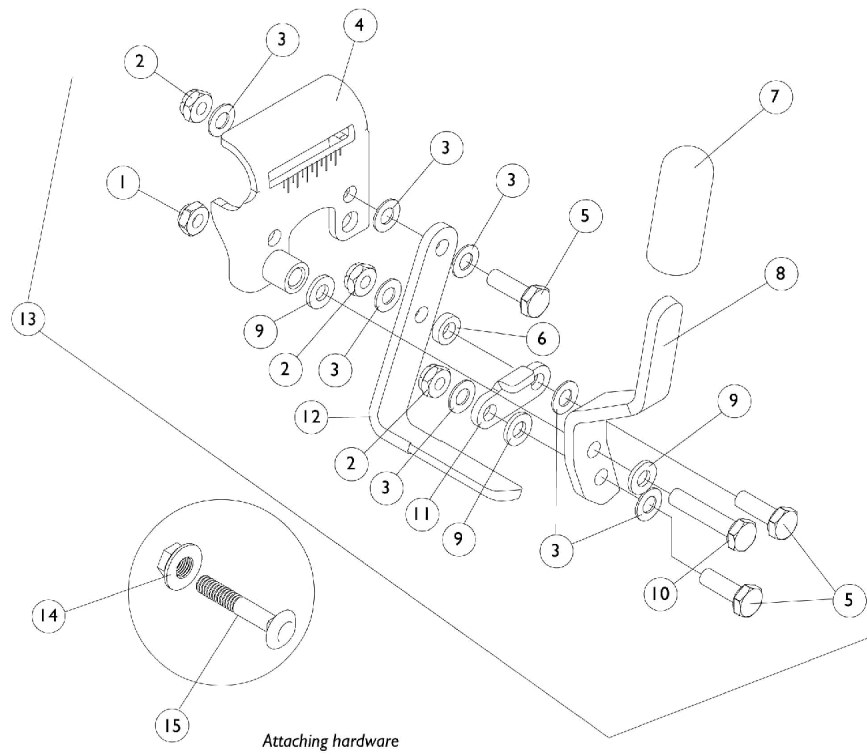

Wheel Locks

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	7930364-NLA	NLA - Package, Locknut (1/4-20)	25	1
1	B	7930323-NLA	NLA - Package, Locknut (1/4-20)	25	1
2	A	7930367-NLA	NLA - Locknut (1/4-28)	25	3
2	B	7930334-NLA	NLA - Locknut (1/4-28)	25	3
3	A	7930357-NLA	NLA - Washer, Nylon (1/4 x 1/2 x 1/32")	25	7
3	B	1026103	Washer, Nylon (1/4 x 1/2 x 1/32")	1	7
4		See Chart	#4,8, & 13 Mounting Bracket, Handle & Wheellock Assembly	1	1
5	A	60274X012-NLA	NLA - Screw, Hex Head (1/4-28 x 3/4")	1	3
5	B	1028687	Screw, Hex Head (1/4-28 x 3/4")	1	3
6	A	40157X004	Spacer, Nylon (1/4 x 7/16 x 1/8")	1	1
6	B	1026105-NLA	NLA - Spacer, Nylon (1/4 x 7/16 x 1/8")	1	1
7		47376X024	Tip, Rubber Wheel Lock Handle, Black	1	1
8		See Chart	#4,8, & 13 Mounting Bracket, Handle & Wheellock Assembly	1	1
9	A	1010398	Washer, Nylon (1/4 x 1/2 x 1/16")	1	3
9	B	1026033	Washer, Nylon (1/4 x 1/2 x 1/16")	1	3
10	A	1007017-NLA	NLA - Screw, Hex Head (1/4-20 x 1-1/4")	1	1
10	B	1021406	Screw, Hex Head (1/4-20 x 1-1/4")	1	1
11	A	1016592-NLA	NLA - Link, Chrome	1	1
11	B	1021404-NLA	NLA - Link, Black	1	1
12	A	1015443-NLA	NLA - Shoe	1	1
12	B	1021405-NLA	NLA - Shoe, Wheel Lock	1	1
13		See Chart	#4,8, & 13 Mounting Bracket, Handle & Wheellock Assembly	1	1
14	A	7930351-NLA	NLA - Nut, Hex Flange (1/4-20)	25	1
14	B	1021403	Nut, Hex Flang (1/4-20)	1	1
15	A	7930309-NLA	NLA - Bolt, Carriage (1/4-20 x 1-1/2")	25	1
15	B	1021402	Bolt, Carriage (1/4-20 x 1-1/4")	1	1
	C,D,E, F	7400109	Kit, Wheel Lock Shoe Spacer (Pair)	2	2
			Screw, Set Socket Cup Point (#8-32 x 3/16")	1	4
			Spacer, Wheel Lock Shoe (2-7/16")	1	2
	C,D,E, G	1037422	Kit, Wheel Lock Shoe Spacer, 22" Rear Wheels (Pair)	2	2
			Spacer, Wheel Lock Shoe (1-45/64")	1	2
			Screw, Set Socket Cup Point (#8-32 x 3/16")	1	4
	C	7930306-NLA	NLA - Package, Screw, Set Socket Cup Point (#8-32 x 3/16")	25	2

NOTE: A - Chrome Chairs
 B - Black or Color Chairs
 C - Not Shown
 D - Includes set screws
 E - These items are not included with wheel lock, they may be needed if chair has 20 inch or 22 inch rear wheels
 F - 20" Rear Wheels
 G - 22" Rear Wheels

Wheel Locks

#4,8, & 13 Mounting Bracket, Handle & Wheellock Assembly

Wheellock Type	Side	4) Mounting Bracket	8) Wheel Lock Handle	13) Wheel Lock Assembly
Push to Lock: Chrome	Right	1015445-NLA	1016588-NLA	1045861-NLA See Note(s) A
	Left	1015446-NLA	1016589-NLA	1045862-NLA See Note(s) A
Push to Lock: Black	Right	N/A	N/A	1023493
	Left	N/A	N/A	1023494
Push to Lock: Color	Right	1031787-NLA	1021408-NLA	1031792
	Left	1031788-NLA	1021409-NLA	1031793
Pull to Lock: Chrome	Right	1015445-NLA	1016590-NLA	1045857
	Left	1015446-NLA	1016591-NLA	1045858
Pull to Lock: Color	Right	1031787-NLA	1031780-NLA	1031794
	Left	1031788-NLA	1031781-NLA	1031795

NOTE: A - No Longer Available. Use part #'s 1023493 and 1023494 as replacements.