


Support Assemblies

For Models P60, P70, P70T, P30E

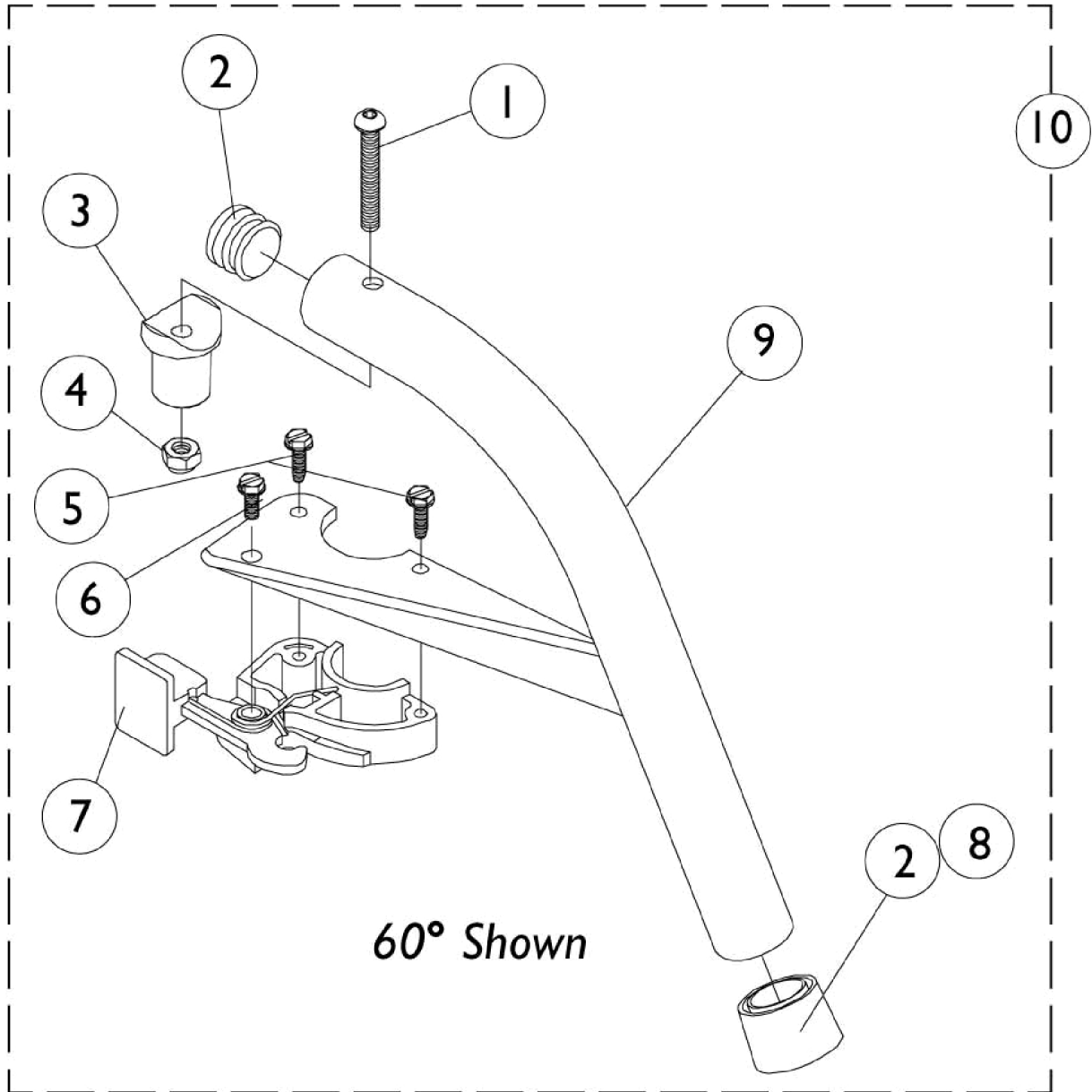

Support Assemblies For Models P60, P70, P70T, P30E

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		7930301-NLA	NLA - Package, Button Head Cap Screw (1/4-20 x 1-3/4")	25	1
2	B	1056328	Plug Button, Black (1")	1	2
3		1017148	Pivot, Footrest	1	1
4		7930333	Locknut (1/4-20)	25	1
5		1025451	Screw, Slotted Hex Washer Head (#10-32 x 5/8")	1	2
6		7930314	Package, Screw, Slotted Hex Washer Head (#10-32 x 1/2")	25	1
7		1025454	Housing, Latch - Right	1	1
7		1025455	Housing, Latch - Left	1	1
8		1026124	Collar, Anti-Rattle	1	1
	A	1026987	Guard, Impact (8" L)	1	1
9	C	See Chart	#9 & 10 Support Assemblies (Include Items 2-8)	1	1
10	D	See Chart	#9 & 10 Support Assemblies (Include Items 2-8)	1	1

NOTE: A - Not Shown. For 70 deg. Tapered Footrest.
 B - 2 each on 60° and 70°
 C - Support Tube and Bracket
 D - Support Assembly (Includes Items 1-9)

Support Assemblies (Includes Items 2-8)

Side	Item #9 60° Swingaway	Item #10 60° Swingaway	Item #9 70° Non-Tapered	Item #10 70° Non-Tapered	Item #9 70° Tapered	Item #10 70° Tapered
Right	1082432-NLA See Note(s) A	1082440-NLA See Note(s) A	1082434-NLA See Note(s) A	1082442-NLA See Note(s) A	1082436-NLA See Note(s) A	1082444-NLA See Note(s) A
Left	1082433-NLA See Note(s) A	1082441-NLA See Note(s) A	1082435-NLA See Note(s) A	1082443-NLA See Note(s) A	1082437-NLA See Note(s) A	1082445-NLA See Note(s) A
Right	8881026831-NL A See Note(s) B	8881026992 See Note(s) B	8881026833-NL A See Note(s) B	8881026994 See Note(s) B	8881025447-NL A See Note(s) B	8881026996 See Note(s) B
Left	8881026832-NL A See Note(s) B	8881026993 See Note(s) B	8881026834-NL A See Note(s) B	8881026995 See Note(s) B	8881025448-NL A See Note(s) B	8881026997 See Note(s) B

NOTE: A - Specify Black.
 B - Specify frame color, see page titled "Frame Finishes, Upholstery Colors and Packaging".
 Specify frame color when ordering. Specify Black for chrome frames. See page titled "Frame Finishes, Upholstery Colors and Packaging".