



# Legrest Calfpads and Ratchet Assemblies

**DETAIL A**  
(see Legrest Support  
Less Pad page.)

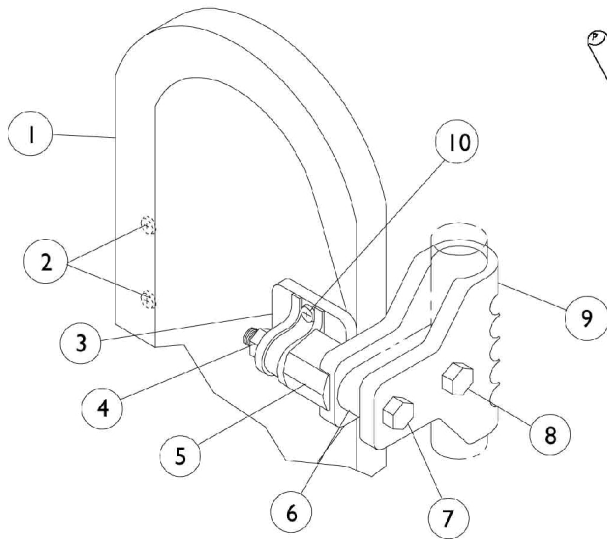

**DETAIL B**  
(see Legrest Support  
Less Pad page.)

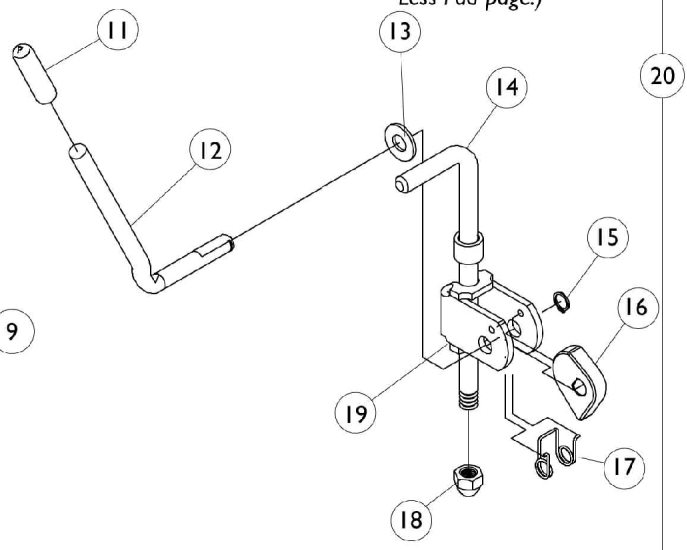

## Legrest Calfpads and Ratchet Assemblies

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	88804529X003	Pad, Legrest, Vinyl - Right	1	1
1	A	88804529X004	Pad, Legrest, Vinyl - Left	1	1
1	A,C	7500003	Pad, Calf with Cover	1	1
1	A,C	01415X000-NLA	NLA - Cover, Legrest Pad	1	1
1	D	8881091759	Pad w/ Cover and Screws, Legrest Vinyl (Pair)	2	1
2		7930360-NLA	NLA - Package, Screw, Phillips Pan Head (#10-32 x 11/16")	25	2
3		04526X019	Plate, Legrest Pad	1	1
4		7930330-NLA	NLA - Locknut (5/16-24)	25	1
5		1025133	Spacer, Legrest Pad, Black	1	1
6		1032267	Spacer, Legrest Pad Mounting, Black	1	1
7		1037110	Screw, Hex Head (5/16-24 x 3-1/2")	1	1
8		1028514	Screw, Hex Head (5/16-24 x 1-1/2")	1	1
9		1032266	Bracket, Legrest Pad Mounting, Black	1	1
10		7930328-NLA	NLA - Package, Screw, Phillips Pan Head (#10-32 x 11/16")	25	2
11		1040060	Tip, Plastic, Black	1	1
12		1037126	Handle, Release	1	1
13		7930310-NLA	NLA - Washer, Nylon (5/16 x 11/16 x 1/16")	25	1
14		42326X001	Bar, Ratchet	1	1
15		1040831	Ring, Snap	1	1
16		1037125	Cam, Legrest	1	1
17		1037485	Spring, Release	1	1
18		7930336-NLA	NLA - Nut, Acorn (5/16-24)	25	1
19		04528X002	Bracket, Release, Black	1	1
20	E	1046328-NLA	NLA - Kit, Ratchet	2	1
	B,F	1042798-NLA	NLA - Instruction Sheet, Ratchet Kit	1	1

NOTE: A - Specify U67 upholstery color when ordering, see page titled "Frame Finishes, Upholstery Colors and Packaging"  
 B - Not Shown  
 C - 16" and Wider (before 9/5/00)  
 D - After 9/5/00  
 E - Includes Items 11-19 and Instruction sheet  
 F - Item is no longer available from the Invacare Service Parts Department. Refer to the Invacare Website under "Product Literature Search". Enter the part number in the "Keywords" field and download a free PDF version copy.