



Seat Pan and Hardware

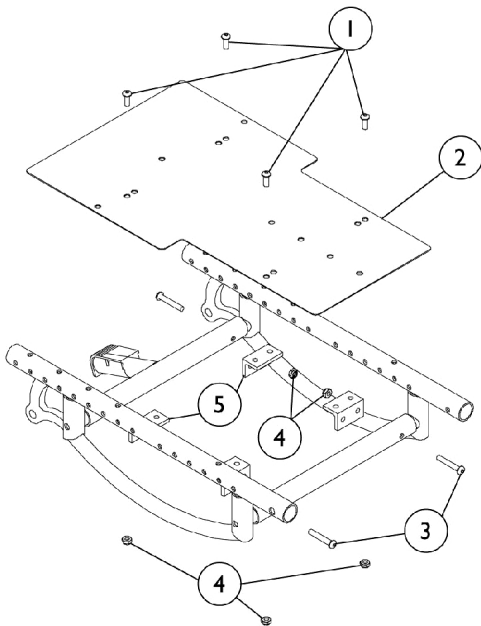

Drop Mount Seat Pan

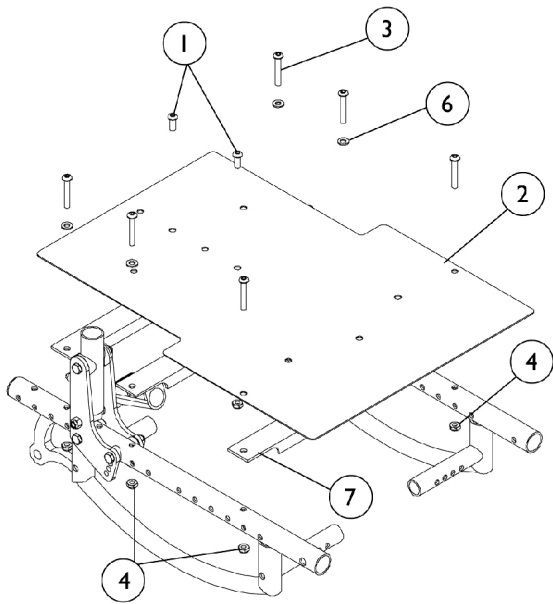

Flush Mount Seat Pan

Seat Pan and Hardware

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1055304	Screw, Socket Button Head (1/4-28 x 5/8")	1	4
2		See Chart	#2 Seat Pan w/ Hook and Loop Fastener	1	1
		1086270-NLA	NLA - Hook Fastener with Adhesive Back (2" W x 9-1/2" L)	1	2
		1086346	Loop Fastener (2" Wide)	1	2
3		1083099	Screw, Socket Button Head (1/4-28 x 1-1/2")	1	6
4		1025765	Locknut (1/4-28)	1	8
5		1108536-NLA	NLA - Bracket, Seat Drop, Black	1	4
6		1026907	Washer (1/4 x 1/2 x 1/16")	1	4
7	B	See Chart	#7 Seat Pan Supports	1	3
NOTE: A - Not Shown B - Required for Flush Mount Only					

#2 Seat Pan w/ Hook and Loop Fastener

Seat Mounting Type	10" Wide	11" Wide	12" Wide	13" Wide	14" Wide	15" Wide	16" Wide
Drop Mount	N/A	N/A	1104636-NLA	1104637-NLA	1104638-NLA	1104639-NLA	1104640-NLA
Flush Mount	1104636-NLA	1104637-NLA	1104638-NLA	1104639-NLA	1104640-NLA	1104641-NLA	1104642-NLA

#7 Seat Pan Supports

(3 ea. Required)

10" Wide	11" Wide	12" Wide	13" Wide	14" Wide	15" Wide	16" Wide
1104496-NLA	1104497	1104498	1104499	1104500	1104501	1104502-NLA