
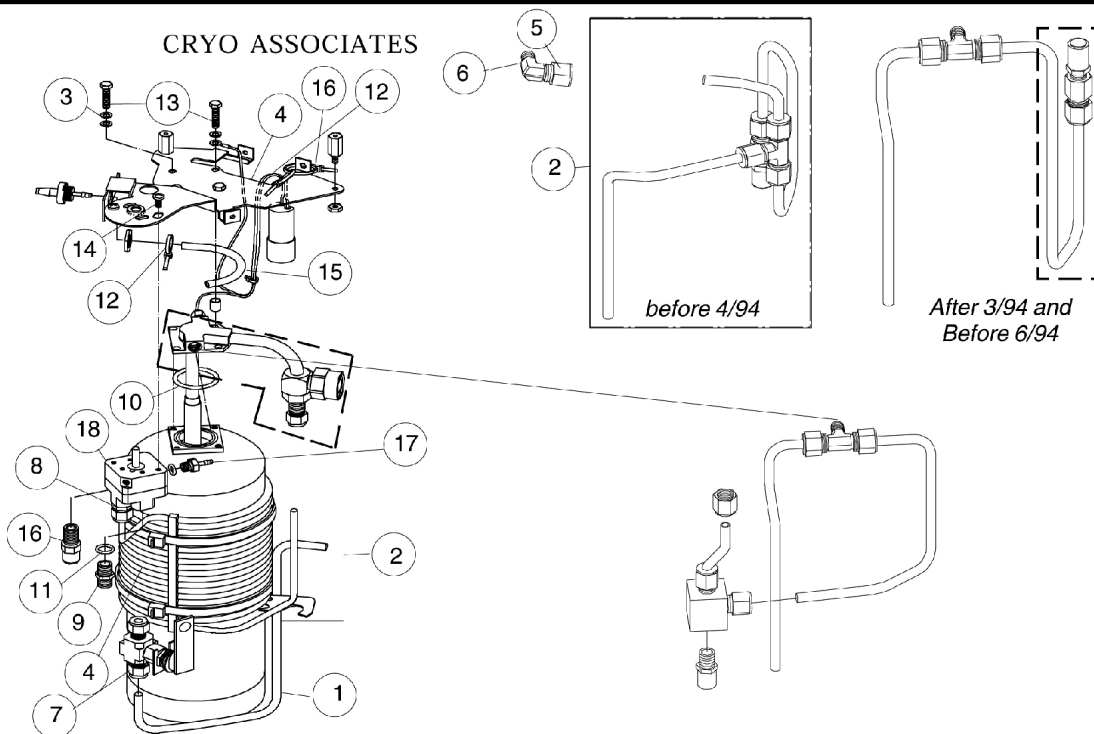


SYSTEM NEUTRAL, CONTINUOUS, PORTABLE

1 and 1.2 Standard, Low and High Flows

All Models Except
Cryo Associates

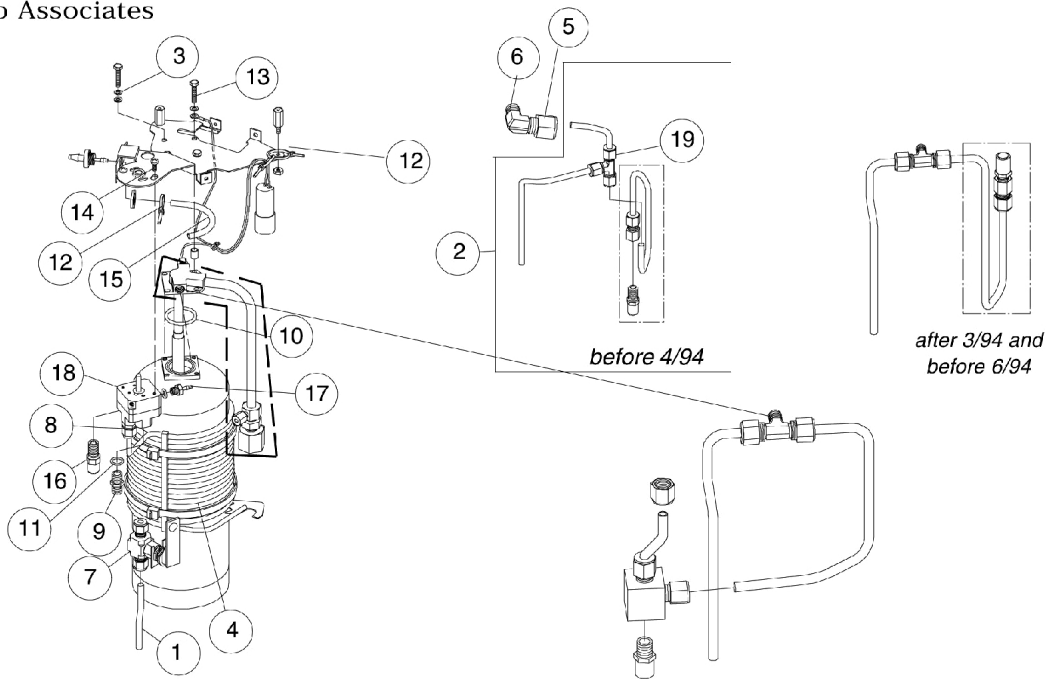

SYSTEM NEUTRAL, CONTINUOUS, PORTABLE

1 and 1.2 Standard, Low and High Flows


Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	210022M-NLA	NLA - Tube, Outlet Vent	1	1
1	B	210072M-NLA	NLA - Tube, Outlet Vent	1	1
1	C	210079M-NLA	NLA - Tube, Outlet Vent	1	1
2	D	210031M-NLA	NLA - Flange Tube, Vent to Fill (VTF)	1	1
2	C	210076M-NLA	NLA - Flange Tube, Vent to Fill (VTF)	1	1
3		210033M-NLA	NLA - Washer, Lock (#10)	1	4
4	A	210052M-NLA	NLA - Heat Exchanger Assembly	1	1
4	B	210070M-NLA	NLA - Heat Exchanger Assembly	1	1
5	E	210075M-NLA	NLA - Fitting, Hose	1	1
6	E	211071M-NLA	NLA - Fitting, Elbo, Brass	1	1
7		211210M-NLA	NLA - Vent to Fill (VTF) Valve with Handle	1	1
8		211219M-NLA	NLA - Fitting, Inlet	1	1
9	F,H	110051M-NLA	NLA - Fitting, Union (7/16-20)	1	1
9	G,H	211346M-NLA	NLA - Fitting, Plug (7/16-20)	1	1
10	H	2002147-NLA	NLA - O-Ring, Spring Loaded (1-1/4" x 1-7/16")	1	1
11	H,I	212018M-NLA	NLA - O-Ring (3/8" I.D. x 1/16")	1	1
12		214004M-NLA	NLA - Ty-Wrap (4")	1	5
13		214056M-NLA	NLA - Screw, Hex Head (#10-24 x 7/8")	1	4
14		214067M-NLA	NLA - Screw, Phillips Pan Head (#8-32 x 3/8")	1	2
	I	214095M-NLA	NLA - Screw, Hex Head (#10-24 x 1/2")	1	2
15		110030M-NLA	NLA - Tygon Tubing	1	1
16	D	801033M-NLA	NLA - Relief Valve, 30 PSI with O-Ring	1	1
16	C	801240M-NLA	NLA - Relief Valve, 75 PSI with O-Ring	1	1
	J	2003068-NLA	NLA - Filter, Inlet - For Relief Valves	1	1
17		111201M-NLA	NLA - Fitting, Barbed with O-Ring - Flow Selector	1	1
18	D	110008M01-NLA	NLA - Flow Selector-Standard 0-6 LPM	1	1
18	C	110008M04-NLA	NLA - Flow Selector-Standard 0-6 LPM	1	1
18	D	110008M03-NLA	NLA - Flow Selector-Low 0-2.5 LPM	1	1
18	C	110008M06-NLA	NLA - Flow Selector-Low 0-2.5 LPM	1	1
18	D	110008M02-NLA	NLA - Flow Selector-High 0-15 LPM	1	1
18	C	110008M05-NLA	NLA - Flow Selector-High 0-15 LPM	1	1

NOTE: A - All Models Except Cryo Associates & Linde
B - Cryogenic Associates (Caire)
C - Linde (Mark)
D - All Models Except Linde (Mark)
E - Before 4/94
F - After 5/94
G - Before 6/94
H - It is recommended when replacing to use Invacare products only
I - 1 Liter Only
J - Not Shown


SYSTEM NEUTRAL, CONTINUOUS, PORTABLE

1 and 1.2 Standard, Low and High Flows

All Models Except
Cryo Associates

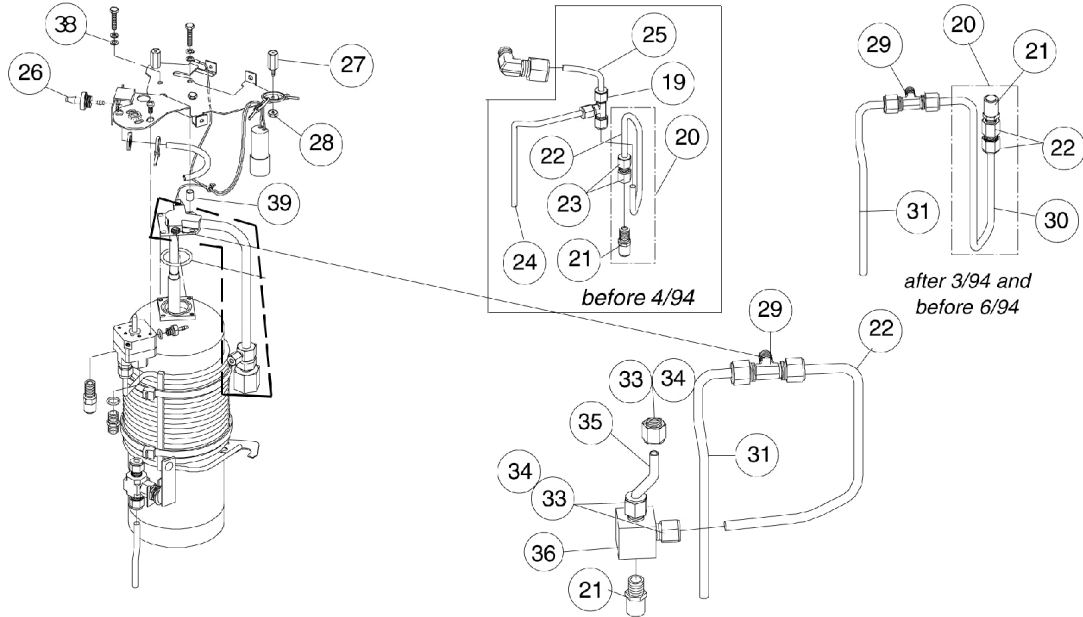

CRYO ASSOCIATES

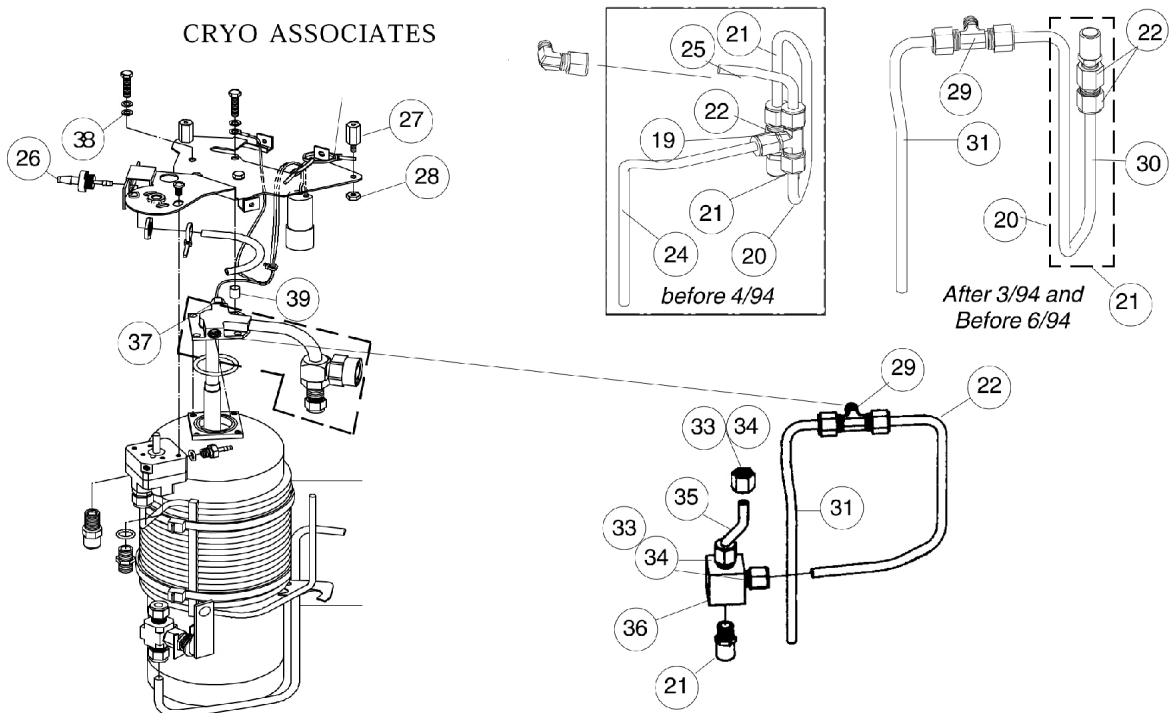

SYSTEM NEUTRAL, CONTINUOUS, PORTABLE

1 and 1.2 Standard, Low and High Flows

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
19	A,B	211085M-NLA	NLA - Tee Fitting (1/4" CPI)	1	1
20	A,C	210044M-NLA	NLA - Valve, Primary Relief Valve - 23 PSI	1	1
20	A,D	210058M-NLA	NLA - Valve, Primary Relief Valve - 55 PSI	1	1
20	E	110042M-NLA	NLA - Valve, Primary Relief Valve - 23 PSI	1	1
20	E	110043M-NLA	NLA - Valve, Primary Relief Valve - 55 PSI	1	1
21	C	211016M-NLA	NLA - Primary Relief Valve - 23 PSI	1	1
21	D	211238M-NLA	NLA - Primary Relief Valve - 55 PSI	1	1
22		210048M-NLA	NLA - Tubing, Primary Relief Valve	1	1
23		211186M-NLA	NLA - Fitting, Primary	1	1
24		210061M-NLA	NLA - Tubing, Inlet Vent to Fill (VTF)	1	1
25		210059M-NLA	NLA - Tubing, Flange Vent to Fill (VTF)	1	1
26		111202M-NLA	NLA - Fitting, Portable Outlet	1	1
27		210012M-NLA	NLA - Stand-Off (3/8 x 5/8")	1	2
28		211230M-NLA	NLA - Nut, Hex (#10-32)	1	2
29		110040M-NLA	NLA - Fitting, Tee (1/4 CPI 1/8 NPT)	1	1
30		110038M-NLA	NLA - Tube, Primary Relief	1	1
31		110039M-NLA	NLA - Tube, Vent to Fill (VTF)	1	1
32		110052M-NLA	NLA - Tube, Flange	1	1
33		211344M-NLA	NLA - Nut, CPI (1/4")	1	1
34		211062M-NLA	NLA - Ferrule, 1/4" CPI	1	1
35		110053M-NLA	NLA - Tube, Economizer	1	1
36	C	110049M-NLA	NLA - Economizer Valve - 20 PSI	1	1
36	D	110050M-NLA	NLA - Economizer Valve - 50 PSI	1	1
37		420054M-NLA	NLA - Fitting, Bulkhead Adapter	1	1
38		2002184-NLA	NLA - Washer, Flat	1	3
39		214093M-NLA	NLA - Spacer	1	4
	F,G	520074M-NLA	NLA - Kit, Economizer (20 PSI)	1	1
	F,G	520075M-NLA	NLA - Kit, Economizer (50 PSI)	1	1

NOTE: A - Before 4/94
 B - After 5/94
 C - All Models Except Linde (Mark)
 D - Linde (Mark)
 E - After 3/94
 F - Includes: (1) Each Economizer Tube - 110054M, (1) Each Union Fitting - 110051M, Items 33, 34, 35 and 36.
 G - Not Shown