


Solid Folding Seats

NOTE: Solid Folding Seats measure two inches narrower than actual seat width.

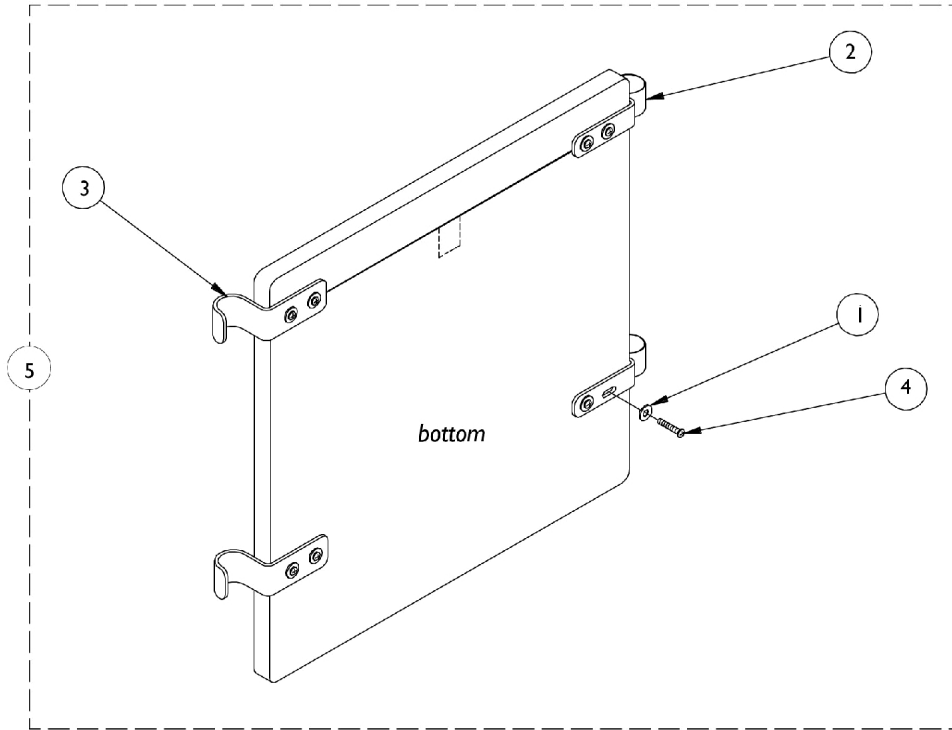

Solid Folding Seats

NOTE: Solid Folding Seats measure two inches narrower than actual seat width.

| Item Number | Note | Part Number | Description | Quantity Per | |
|--|------|--------------------|---|--------------|----------|
| | | | | Package | Assembly |
| 1 | | 1013218 | Washer (#10 Flat) | 1 | 8 |
| 2 | | 1016770-NLA | NLA - Bracket, Seat, Chrome | 1 | 2 |
| 3 | | 1016771-NLA | NLA - Hook, Seat | 1 | 2 |
| 4 | | 40163X008 | Screw, Phillips Pan Head (#10-32 x 11/16") | 1 | 1 |
| 5 | A | 8881035937 | Solid Folding Seat w/ Hardware, Vinyl (14" W x 16" D) | 1 | 1 |
| 5 | A | 8881110378-NL A | NLA - Solid Folding Seat w/ Hardware, Vinyl (16" W x 16" D) | 1 | 1 |
| 5 | A | 8881110379 | Solid Folding Seat w/ Hardware, Vinyl (18" W x 16" D) | 1 | 1 |
| 5 | A | 8881110372 | Solid Folding Seat w/ Hardware, Nylon (20" W x 16" D) | 1 | 1 |
| 5 | A | 8881035936 | Solid Folding Seat w/ Hardware, Nylon (20" W x 18" D) | 1 | 1 |
| NOTE: A - Specify upholstery color when ordering. See page titled "Frame Finishes, Touch-Up Paints and Upholstery Colors." | | | | | |