


Seat Frame & Hardware

Basic Recline 4XP

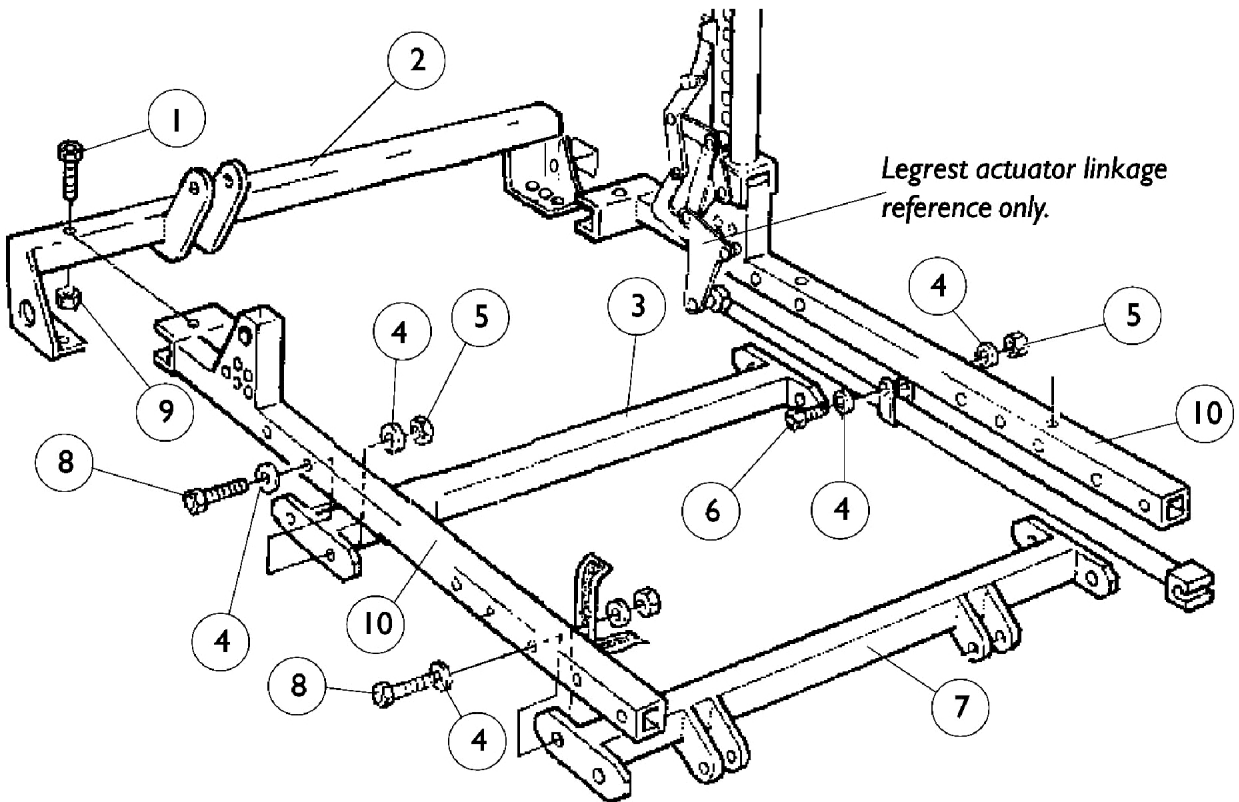

Seat Frame & Hardware

Basic Recline 4XP

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		0055306	Screw, Hex Head (5/16-18 x 1-3/4")	1	2
2		0216302-NLA	NLA - Frame Assembly, Rear, 4XP (16" W)	1	1
2		0216402-NLA	NLA - Frame Assembly, Rear, 4XP (18" W)	1	1
3		0158402-NLA	NLA - Frame Assembly, Center, Black, 4XP (16" W)	1	1
3		0158502-NLA	NLA - Frame Assembly, Center, Black, 4XP (18" W)	1	1
4		0050406	Washer, Flat (1/4 x 5/8 x 1/16")	1	20
5		1025736	Locknut (1/4-20)	1	10
6		0051006-NLA	NLA - Screw, Hex Head Full Thread (1/4-20 x 1-1/2")	1	2
7		0216702-NLA	NLA - Frame Assembly, Front, Black, 4XP (16" W)	1	1
7		0216802-NLA	NLA - Frame Assembly, Center, Black, 4XP (18" W)	1	1
8		1024793-NLA	NLA - Screw, Hex Head (1/4-20 x 1-3/4")	1	8
9		0054906	Locknut, High Profile (5/16-18)	1	2
10		0216602-NLA	NLA - Frame, Seat Side, 4XP - Right	1	1
10		0216502-NLA	NLA - Frame, Seat Side, 4XP - Left	1	1