


Wheel Lock Extension Handle

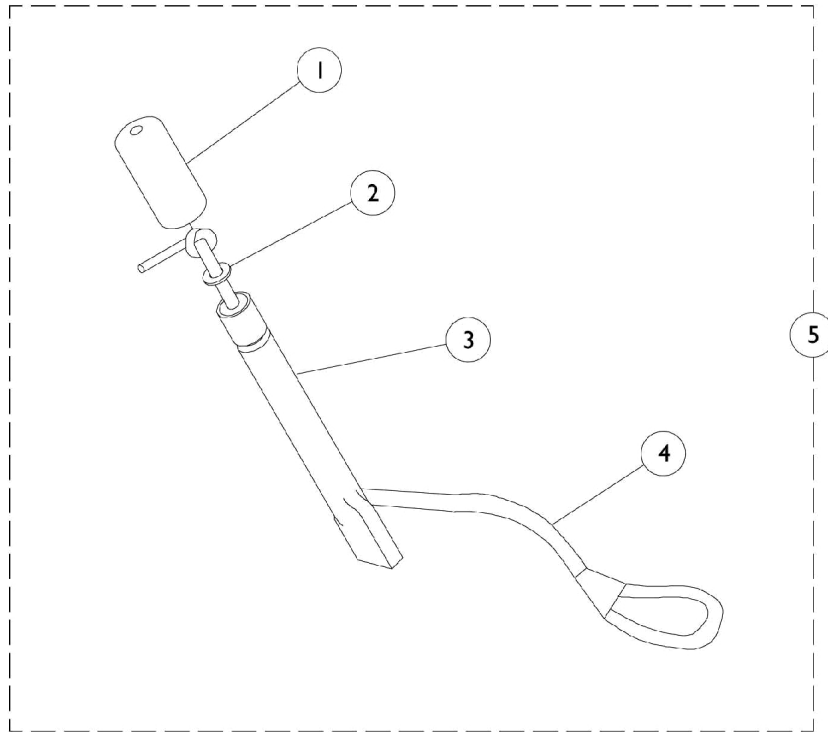

Wheel Lock Extension Handle

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1128817	Package, Tip, Rubber, Black (Pkg. of 10)	1	1
2		1013218	Washer (#10 Flat)	1	1
3		1038597-NLA	NLA - Handle, Extension, Black (6")	1	1
4		1038662-NLA	NLA - Cord, Elastic	1	1
5	B	1063125	Wheel Lock Extension Handle Assembly	1	1
5	B,C	1038599	Wheel Lock Extension Handle Assembly, Black (6") (Pair)	2	1
	D	1039644-NLA	NLA - Instruction Sheet, Wheel Lock Extensions	1	1
NOTE: A - After 8/02 B - Includes items 1-4 C - Includes Instruction Sheet D - Not Shown					