



Captain Seat Frame and Actuator Hardware After 10/31/01

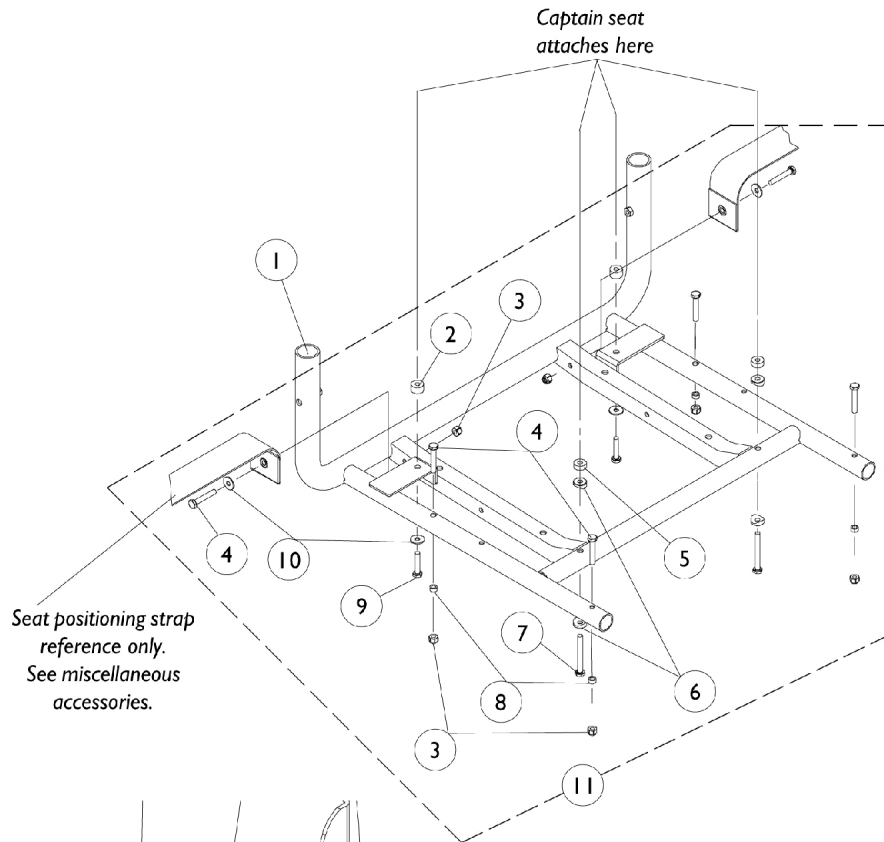


Full recline captain seat reference only.

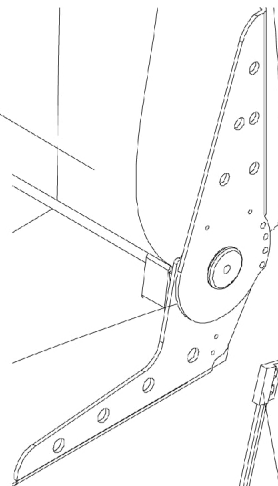

**Actuator Attaching Hardware
(For Full Recline Models Only)**

Sensor cable reference only. See harnesses and cables page.

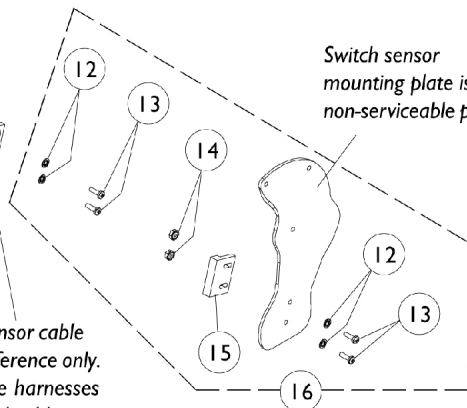

Captain Seat Frame and Actuator Hardware After 10/31/01

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1070455-NLA	NLA - Frame, Seat, Black (18" W)	1	1
1		1108099-NLA	NLA - Frame, Seat, Black (20" W)	1	1
1		1101913-NLA	NLA - Frame, Seat, Black (22" W)	1	1
2		1108411-NLA	NLA - Spacer (1/4 x 5/8 x 3/8")	1	2
3		7930333	Locknut (1/4-20)	25	6
4		1030949	Screw, Hex Head (1/4-20 x 1-1/2")	1	6
5		1108410	Spacer (1/4 x 5/8 x 1/4")	1	2
6		1026099	Spacer, Coved (1/4 x 11/16 x 1/2")	1	4
7		1026154-NLA	NLA - Screw, Hex Head (1/4-20 x 2-1/4")	1	2
8		1047449-NLA	NLA - Spacer (1/4 x 3/8 x 3/16")	1	4
9		1021406	Screw, Hex Head (1/4-20 x 1-1/4")	1	2
10		1026900	Washer (1/4 x 23/32 x 1/16")	1	4
11	A	1079208-NLA	NLA - Hardware, Captain Seat Mounting	1	1
12		1008525-NLA	NLA - Washer, Internal Tooth Lock (#4-40)	1	4
13		7930339-NLA	NLA - Package, Screw, Phillips Pan Head (#4-40 x 3/8")	25	4
14		P0870X000	Nut, Hex Keps (#4-40)	1	2
15		1024001	Sensor, Actuator Switch	1	1
16	B	1107242-NLA	NLA - Hardware, Actuator Switch Sensor Mounting	1	1

NOTE: A - Includes items 2-10
B - Includes items 12-15