


Mechanical Legrest Actuator Linkage w/ M.S.R. Basic Recline 4XP

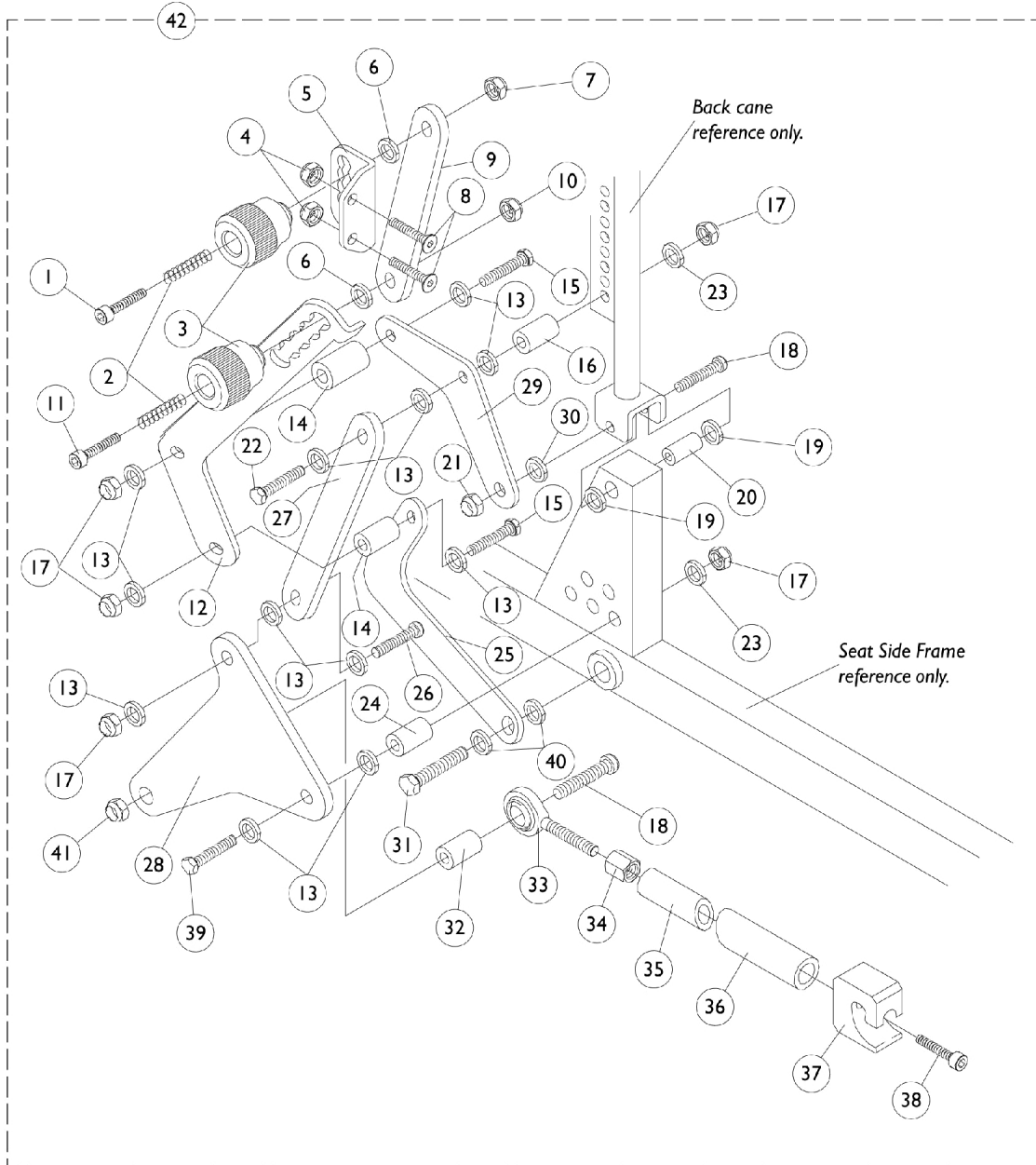

Mechanical Legrest Actuator Linkage w/ M.S.R. Basic Recline 4XP

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		0258406-NLA	NLA - Screw, Socket Head, Cap (#10-32 x 1-1/4")	1	1
2		0146500-NLA	NLA - Spring, Compression (1/32 WT x 15/16" L)	1	2
3		0146622-NLA	NLA - Knob, MSR Adjustment	1	2
4		7930323-NLA	NLA - Package, Locknut (1/4-20)	25	2
5		0149624-NLA	NLA - Bracket, MSR Back Pan, Black	1	1
6		0146700-NLA	NLA - Washer, Nylon (3/16 x 5/8 x 1/16")	1	2
7		7930332-NLA	NLA - Locknut, High Profile (#10-32)	25	1
8		0195606-NLA	NLA - Screw, Socket, Cap, Flat Head (1/4-20 x 3/4")	1	2
9		0148806-NLA	NLA - Link, Upper Adjustment, Black	1	1
10		317601-NLA	NLA - Locknut, Low Profile (#10-32)	1	1
11		0149506-NLA	NLA - Screw, Socket Head, Hex (#10-32 x 1")	1	1
12		0175524-NLA	NLA - Link, Lower Adjustment, Black - Right	1	1
12		0175424-NLA	NLA - Link, Lower Adjustment, Black - Left	1	1
13		0050100-NLA	NLA - Washer, Nylon Flat (1/4 x 1/2 x 1/16")	1	12
14		0216901-NLA	NLA - Bushing, Upper/ Lower	1	2
15		0052606-NLA	NLA - Screw, Hex Head (1/4-20 x 2")	1	2
16		0146422-NLA	NLA - Spacer, Leg Link	1	1
17		7930333	Locknut (1/4-20)	25	5
18		0055506-NLA	NLA - Screw, Button Head, Socket (5/16-18 x 2")	1	2
19		0055800-NLA	NLA - Washer, Nylon (7/16 x 3/4 x 3/32")	1	2
20		0060300-NLA	NLA - Bushing, Bronze (5/16 x 7/16 x 1-1/4")	1	1
21		0055006-NLA	NLA - Locknut, Low Profile (5/16-18)	1	1
22		1024793-NLA	NLA - Screw, Hex Head (1/4-20 x 1-3/4")	1	1
23		0050406	Washer, Flat (1/4 x 5/8 x 1/16")	1	2
24		0217024-NLA	NLA - Bushing, Back Post/Leg Link	1	1
25		0165424-NLA	NLA - Link, MSR Lower, Black	1	1
26		0051606-NLA	NLA - Screw, Cap, Button Head, Socket (1/4-20 x 1")	1	1
27		0217324-NLA	NLA - Link, Small, Black	1	1
28		0217124-NLA	NLA - Link, Back Post Leg, Black	1	1
29		0217224-NLA	NLA - Link, MSR Back Post Lower, Black	1	1
30		0054806-NLA	NLA - Washer, Flat (11/32 x 11/16 x 1/16")	1	1
31		0139606-NLA	NLA - Screw, Hex Head (5/16-18 x 1")	1	1
32		0220924-NLA	NLA - Spacer, Rod End	1	1
33		0083800-NLA	NLA - Rod, End, Male (5/16-24)	1	1
34		0141405-NLA	NLA - Nut, Hex (5/16-24)	1	1
35	B	0035902-NLA	NLA - Rod, Push, Rear Tube, Short, Black (12-1/2")	1	1
35	C	0222502-NLA	NLA - Rod, Push, Rear Tube, Long, Black (14")	1	1
36	D	0035702-NLA	NLA - Rod, Push, Front Tube, AT5544 Legrest, Black	1	1
36	E	0036402	NLA - Rod, Push, Front Tube, Genius Legrest, Black	1	1
37		0026400-NLA	NLA - Receptacle, Power Legrest	1	1
38		0053306-NLA	NLA - Screw, Allen Head (1/4-28 x 1")	1	1
39		0233506-NLA	NLA - Screw, Hex Head (1/4-20 x 2-1/2")	1	1
40		0054700-NLA	NLA - Washer, Nylon (5/16 x 1/2 x 1/16")	1	2

Mechanical Legrest Actuator Linkage w/ M.S.R. Basic Recline 4XP

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
41		0054906	Locknut, High Profile (5/16-18)	1	1
42	A	1077514-NLA	NLA - Mechanical Legrest Actuator Linkage Assembly w/ M.S.R. - Right	1	1
42	A	1077515-NLA	NLA - Mechanical Legrest Actuator Linkage Assembly w/ M.S.R. - Left	1	1
NOTE: A - Less items 35, 36 B - Short 12-1/2" C - Long 14" D - AT5544 Legrest E - Genius Legrest					