


Motor Actuators

Genius and AT5544 Legrest

Tilt Only or Tilt/Recline & Recline Only

NOTE: See AT5544 Legrest (Power) page for mounting hardware.

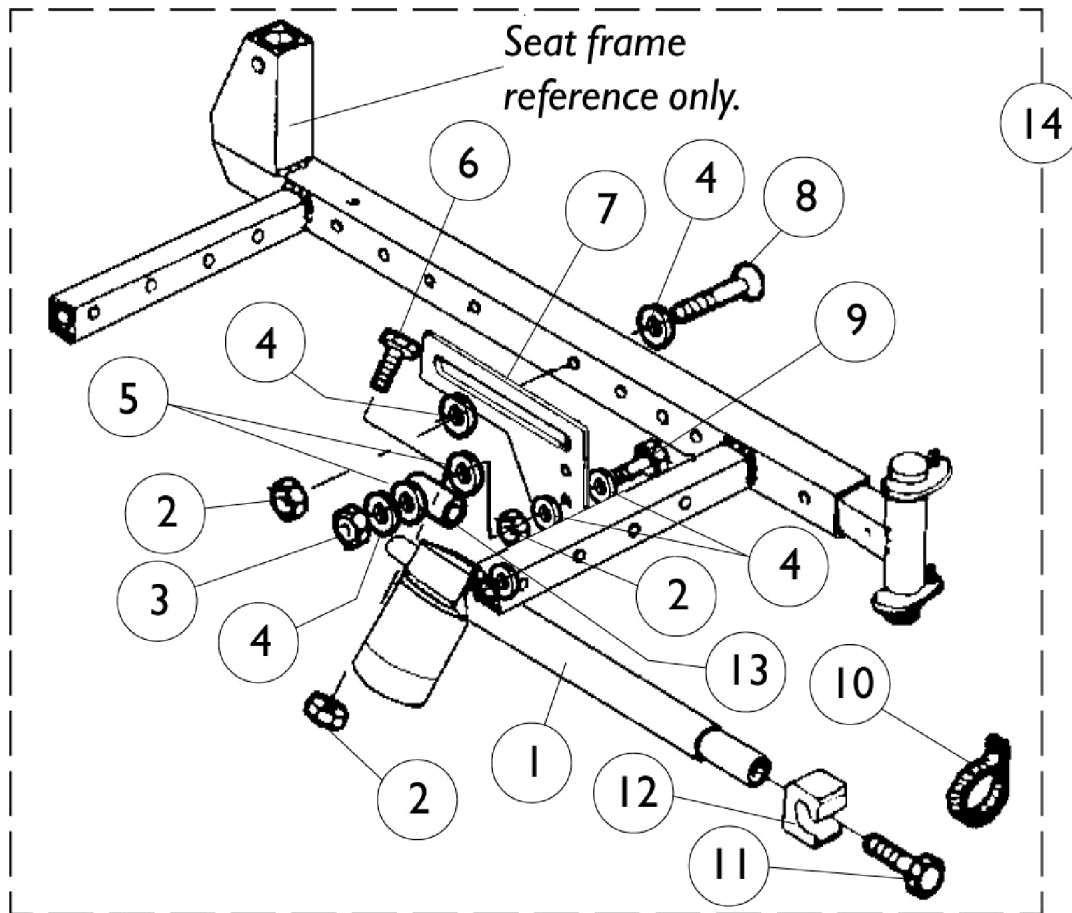

Motor Actuators

Genius and AT5544 Legrest

Tilt Only or Tilt/Recline & Recline Only

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	B	0204100-NLA	NLA - Actuator Motor, Legrest - Right (4" Stroke)	1	1
1	B	0204000-NLA	NLA - Actuator Motor, Legrest - Left (4" Stroke)	1	1
1	C	0001000-NLA	Actuator Motor, Legrest - Right (5" Stroke)	1	1
1	C	0000900-NLA	Actuator Motor, Legrest - Left (5" Stroke)	1	1
2		7930323-NLA	NLA - Package, Locknut (1/4-20)	25	4
3		7930333	Locknut (1/4-20)	25	1
4		0050406	Washer, Flat (1/4 x 5/8 x 1/16")	1	7
5		0050100-NLA	NLA - Washer, Nylon Flat (1/4 x 1/2 x 1/16")	1	2
6		0051706-NLA	NLA - Screw, Hex Head (1/4-20 x 1")	1	1
7		0116602-NLA	NLA - Plate, Legrest Actuator Mounting	1	1
8		1024793-NLA	NLA - Screw, Hex Head (1/4-20 x 1-3/4")	1	2
9		0224306-NLA	NLA - Screw, Hex Head (1/4-20 x 1-3/4")	1	1
10		0157600-NLA	NLA - Strap, Support, Actuator	1	1
11		0053306-NLA	NLA - Screw, Allen Head (1/4-28 x 1")	1	1
12		0026400-NLA	NLA - Receptacle, Power Legrest	1	1
13	D	0203802-NLA	NLA - Bushing, Extension	1	1
13	E	0116502-NLA	NLA - Bushing, Anti-Rotation	1	1
14	A,B	1072855-NLA	NLA - Kit, Actuator Motor w/ Hardware, Legrest - Right (4" Stroke)	1	1
14	A,B	1072856-NLA	NLA - Kit, Actuator Motor w/ Hardware, Legrest - Left (4" Stroke)	1	1
14	B,D	1072857-NLA	NLA - Kit, Actuator Motor w/ Hardware, Legrest - Right (4" Stroke)	1	1
14	B,D	1072858-NLA	NLA - Kit, Actuator Motor w/ Hardware, Legrest - Left (4" Stroke)	1	1
14	A,C	1072859-NLA	NLA - Kit, Actuator Motor w/ Hardware, Legrest - Right (5" Stroke)	1	1
14	A,C	1072860	Kit, Actuator Motor w/ Hardware, Legrest - Left (5" Stroke)	1	1
14	C,E	1072861	Kit, Actuator Motor w/ Hardware, Legrest - Right (5" Stroke)	1	1
14	C,E	1072862	Kit, Actuator Motor w/ Hardware, Legrest - Left (5" Stroke)	1	1

NOTE: A - Item 13 is not used with this legrest actuator attaching hardware.

B - Genius

C - Invacare, AT5544

D - 20" Seat Depth Only

E - 16"-17" Seat Depth Only